

Conquest DICOM Server 1.4.16 CONFORMANCE STATEMENT and detailed software revision history

Revision 1; 19961103	As taken from the 'developers cut' distribution of MicroPACS
Revision 2; 19980703	Conquest additions; pre-release 1.3.1.
Revision 3; 19980713	Conquest additions; release 1.3.1
Revision 4; 19990415	Updated (mainly for RT), release 1.3.2
Revision 5; 19990527	Small software updates, birthday, sex, bolus, release 1.3.3
Revision 6; 20000131	NKI sops updated, new compression mode, mirroring, release 1.3.4
Revision 7; 20000316	Added operation as NT service, pre-release 1.3.5
Revision 8; 20000403	Unlimited servers/services, auto-login, self-test, sop choices, fixes for GE Hi-Speed, improved display, fixed hang concurrent C-ECHO's, final release 1.3.5
Revision 9; 20000701	Improved browser menu, other ODBC drivers, release 1.3.6
Revision 10; 20001129	Multithread UNC path fixes, edit ID, sendto, edit, release 1.3.7
Revision 11; 20010318	Hostname fix, runs 4 services, show incoming, release 1.3.8
Revision 12; 20010502	GUI additions database fixing; faster compression; fixed leaks, release 1.3.9
Revision 13; 20011109	better forward/export, extended DB/sop, zip, drag/drop, rel 1.3.10
Revision 14; 20020508	reliability and error handling fixes, UID configurable, dbrev 6
Revision 15; 20020613	StudyModality, db rev 7, find local missing pats, release 1.3.11
Revision 16; 20020822	FilenameSyntax for DICOM-Works; printing, release 1.3.12
Revision 17; 20021215	Various fixes. Non-ODBC driver; read-ahead thread; release 1.4.0
Revision 18; 20030303	Bug fix release 1.4.1
Revision 19; 20030710	Release 1.4.2, flexible compression and transfer fixes, db rev 11
Revision 20; 20030922	release 1.4.3, registry/administrator bug/doc; long filenames
Revision 21; 20040406	Release 1.4.4, mysql fix / large disks / db rev 13 / jpeg fixes / GUI uses MAG0..9 / system DSN
Revision 22; 20040426	Release 1.4.4a, mysql fix broke MS-access and SQL server, added DoubleBackSlashToDB (see 4.1.3)
Revision 23; 20040615	Release 1.4.5, built-in DbaseIII suitable for huge archives; better deletion tools; auto-move
Revision 24; 20040722	Release 1.4.6, Added FixKodak; Added "nj" compression; FixPhilips/FixKodak defaults 0 (!)
Revision 25; 20040805	Release 1.4.6a, Fix in move for single image
Revision 29; 20041129	Release 1.4.7, JPEG fix, bugreport button; split/merge, 'k' compress, uid list matching, AE with ~xx
Revision 30; 20050131	Release 1.4.8, Fixed delete, nightlymove, 4-server; hold failed forwards/copies; distribution compiles, initial linux release and initial web interface
Revision 31; 20050912	Release 1.4.9, Fixes in web and Linux, modality worklist
Revision 32; 20051024	Release 1.4.9a, Fix crashes due to no patch in controls.pas
Revision 33; 20060103	Release 1.4.10, VirtualServerFor, fixes leaks, linux, postgres
Revision 34; 20060328	Release 1.4.11, K-Pacs viewer, native MySQL, worklist fix, virtualserver fix, flexible filenamesyntax, WEBReadOnly, Fix forwarder retry logic
Revision 35; 20060708	Release 1.4.12alpha: bug fixes, db performance improved, and appendix 5-7
Revision 36; 20061222	Release 1.4.12: bug fixes in dbaseIII driver and in deleting, forwarding and grabbing, resizeable GUI
Revision 37; 20070127	Release 1.4.12b: importconverters, converter scripting, fixes in dbaseIII driver and web access, no nki compression in .dcm

Revision 38; 20070201 Release 1.4.12c: “storage” importconverter, fix ExportFilter and problem in most GUI commands

Revision 39; 20070305 Release 1.4.13alpha: “defer” exportconverter, stuff for k-pacs web viewer; manual split into user and technical part by radtraveller – this is the technical part

Revision 39; 20070707 Release 1.4.13beta: native mysql fixes, delayed converters, sqlite

Revision 40; 20070709 Release 1.4.13beta2: some last fixes

Revision 41; 20071116 Release 1.4.13: more fixes and parameters

Revision 42; 20080322 Release 1.4.14beta: uj compression; database changes; fixes

Revision 43; 20080902 Release 1.4.14: fixes, improved web pages; added query on ‘Number of Patient Related Studies’ items etc.

Revision 44; 20090412 Release 1.4.15alpha: 64 bits, postgres, jpg graphic, etc.

Revision 45; 20090929 Release 1.4.15: mostly bug fixes and a few scripting extensions

Revision 46; 20100905 Release 1.4.16beta: JPEG and JPEG2000 added by Bruce Barton
Further extended scripting options; bug fixes; GUI prints to AE

Revision 47; 20101023 Release 1.4.16beta5: bug fixes, WADO server and client, more converters, improved serversideviewer, caching repetitive queries, enabled MAG0\incoming folder, upload from web server, optional overlap of get and send in virtualservers, animated GIF support.

Revision 48; 20110105 Release 1.4.16rc1; service cleans and zips, fixes and more commands

Revision 49; 20110106 Release 1.4.16rc2 bug fixes

Revision 49; 20110208 Release 1.4.16rc4 added lua scripting

Revision 20; 20110402 Release 1.4.16 full release, typo in study table

Contact, ConQuest DICOM server and many MicroPACS extensions

Lambert Zijp or Marcel van Herk; Radiotherapy department; The Netherlands Cancer Institute; Amsterdam, the Netherlands; Fax: +31-20-6691101 / Email: zijp@nki.nl or portal@nki.nl

Copyright (c) 2011 The Netherlands Cancer Institute.

Developed by Marcel van Herk and Lambert Zijp; the Netherlands Cancer Institute; RT Department

Server core based upon:

Copyright (c) 1995 Regents of the University of California. All rights reserved.

Developed by: Mark Oskin, mhoskin@ucdavis.edu; University of California, Davis Medical Center; Department of Radiology with a Solaris port done and maintained by: Terry Rosenbaum; Michigan State University; Department of Radiology.

Original MicroPACS developer (not active anymore)

Mark Oskin; UC Davis Medical Center; PACS Research and Development Lab.
(916)734-0308 / FAX (916)734-0316 / Email: mhoskin@ucdavis.edu

Administrative / Licensing Contact, original MicroPACS components

Richard L. Kennedy; UC Davis Medical Center
(916)734-7267 / FAX (916)734-0316 / Email: rlkennedy@ucdavis.edu

Redistribution and use in source and binary forms are permitted provided that the above copyright notice and this paragraph are duplicated in all such forms and that any documentation, advertising materials, and other materials related to such distribution and use acknowledge that the software was developed by the University of California, Davis and The Netherlands Cancer Institute, Amsterdam. The name of the University may not be used to endorse or promote products derived from this software

without specific prior written permission. THIS SOFTWARE IS PROVIDED ``AS IS" AND WITHOUT ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, WITHOUT LIMITATION, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

TABLE OF CONTENTS

SECTION 1 INTRODUCTION.....	6
SECTION A(2) CONFORMANCE STATEMENT.....	9
A.0.....INTRODUCTION	9
A.1.....IMPLEMENTATION MODEL	9
A.1.1.....Application Data Flow Diagram	9
A.1.2.....Functional Definition of AE's	10
A.1.3.....Sequencing of Real - World Activities	10
A.2.....AE SPECIFICATIONS	15
A.2.1.....AE1 Specification	15
A.2.1.1.....Association Establishment Policies	15
A.2.1.1.....General	15
A.3.....COMMUNICATION PROFILES	20
A.3.1.....Supported Communication Stacks (parts 8,9)	20
A.3.2.....OSI Stack	20
A.3.3.....TCP/IP Stack	20
A.3.3.1.....API	20
A.3.3.2.....Physical Media Support	20
A.3.4.....Point-to-Point Stack	20
A.4.....EXTENSIONS / SPECIALIZATION / PRIVATIZATIONS	20
SECTION 3 DEFAULT QUERY/RETRIEVE INFORMATION MODEL	
DEFINATION.....	22
3.0.....INTEROPERABILITY SCHEMA	22
3.0.1.....PATIENT ROOT QUERY/RETRIEVE ENTITY RELATIONSHIP	22
3.0.2.....STUDY ROOT QUERY/RETRIEVE ENTITY RELATIONSHIP	23
3.0.1.....PATIENT/STUDY ONLY QUERY/RETRIEVE ENTITY RELATIONSHIP	23
3.1.....ENTITY DESCRIPTIONS	23
3.2.....PATIENT ROOT QUERY/RETRIEVE INFORMATION OBJECT DEFINATION	24

3.2.1.....	Patient Level Keys for Patient Root Query/Retrieve Information Model	24
3.2.2.....	Study Level Keys for Patient Root Query/Retrieve Information Model	24
3.2.3.....	Series Level Keys for Patient Root Query/Retrieve Information Model	24
3.2.4.....	Image Level Keys for Patient Root Query/Retrieve Information Model	25
3.3.....	STUDY ROOT QUERY/RETRIEVE INFORMATION OBJECT DEFINATION	25
3.3.1.....	Study Level Keys for Study Root Query/Retrieve Information Model	25
3.3.2.....	Study Level Keys for Study Root Query/Retrieve Information Model	26
3.3.2.....	Image Level Keys for Study Root Query/Retrieve Information Model	26
3.4.....	PATIENT/STUDY ONLY QUERY/RETRIEVE INFORMATION OBJECT DEFINATION	26
3.4.1.....	Patient Level Keys for Patient/Study Only Query/Retrieve Information Model	26
3.4.1.....	Study Level Keys for Patient/Study Only Query/Retrieve Information Model	26
3.3.....	STUDY ROOT QUERY/RETRIEVE INFORMATION OBJECT DEFINATION	27
3.3.1.....	Study Level Keys for Study Root Query/Retrieve Information Model	27
4.2.....	FILES.....	28
4.4.....	THREADING ARCHITECTURE.....	31
APPENDIX 1: CONQUEST MODIFICATIONS TO MICROPACS.....		33

SECTION 1. INTRODUCTION

The MicroPACS is a Windows or Linux based PACS system that has, at its core, the UCDCM DICOM Network Transport libraries. This system has been combined with a complete user interface (Windows only), which also acts as installation program (written in Borland Delphi) to form the Conquest DICOM server. The Information Definition is designed to be field/run-time programmable. Below the DICOM interface is a database connectivity class that uses a stable built-in SQLite or DBASEIII driver, or talks to ODBC compatible data sources (Windows only), MySQL, or PostGres. This combination permits a PACS system with the following features:

- Complete DICOM Interface. Including SCP's for run-time programmable storage IOD's, and SCP for DICOM Queries and Retrieves.
- Programmable SQL Database tables. This user-programmable feature allows the MicroPACS to be custom tailored to a particular Clinical/Research area. For instance, in a CR setting, the PACS system can be programmed to allow users to query on kvp and ma or in a CT setting, the PACS can be programmed to allow queries on slice-distance.
- Communication to database is done via a SQLite (default and advised for small archives of up to 1000.000 images), a built-in dBaseIII driver, any ODBC database (Windows only), MySQL or PostGres. This allows decoupling PACS and SQL technology. ODBC was tested with:
 - Microsoft Access
 - SQL server
 - Some users have reported successful operation using Interbase and Oracle. Oracle requires manual editing of the DICOM.SQL file, where the names of fields 'rows' and 'columns' are changed to, e.g., 'grows' and 'qcolumns'.

Note: The built-in dBaseIII driver (Conquest addition) is not a full SQL server and poses limitations on query keys: only queries like 'key' = exact match; 'key*' = value starts with key; and '*key*' = value contains key, are supported, as well as date-range queries and multiple UID matching queries. Only common hierarchical queries are supported with fields that are listed in the single de-normalized table for the selected query level (see file DICOM.SQL). Regular queries passing PatientID, StudyUID, and/or SeriesUID will be (very) fast, even for huge archives. Other (image) queries in large archives (>1000.000 images) may be very slow. Server startup time for huge archives may be long due to in-memory index creation (about 1 minute per 1000.000 images). During indexing the server is read-only and only shows indexed images.

- (Conquest addition) Fast and safe (CRC checked) error free compression (>2x) of image data on disk. Do not use this option if you want to read the image files directly from disk yourselves using third party software.
- (Conquest addition) Easy installation of many servers on a single PC. Servers may run as service(s).
- (Conquest addition) A database browser and slice viewer (Windows only) integrated in the PACS system with options for: viewing the DICOM information in a slice, creating BMP files (ideal for slides), sending

selected images, printing, and database fix tools such as changing patient IDs, and deleting and anonymizing studies and series. Also tools to merge or split series. Drag and drop to load DICOM or HL7 files, directories, or a variety of zip files (you then need to place 7za.exe in the server directory).

- (Conquest addition) A simple query/move user interface (Windows only) for diagnostic purposes, to improve your knowledge of DICOM, and to grab missing data from another server.
- (Conquest addition) Fully integrated functionality in one user interface.
- (Conquest addition) Simple print server (Windows) - to default printer.
- (Conquest addition) Log files, which are daily zipped (Windows only). We use the TZipMaster VCL by Chris Vleghert and Eric W. Engler.
- (Conquest addition) Correct display of JPEG(2000) and RLE compressed images in browser.
- (Conquest addition) Flexible configuration of JPEG and NKI private compression with optional (de)compression of incoming, dropped, transmitted and archived files. Since version 1.4.16, JPEG and JPEG2000 engines are built-in using the International JPEG group and Jasper code (Interfacing by Bruce Barton). JPEG (de)compression used to be done (and this can still be configured) using executables from the OFFIS DICOM toolkit (DCMTK version 3.5.3), developed by Kuratorium OFFIS e.V..
- (More conquest additions) Highly improved performance (e.g., using a read-ahead thread), and image forwarding/action capability.
- The archive is well suited as DICOM server for the DICOMWORKS viewer by Phillipe Puech.
- If the BDE is not installed, we use the MiTeC DBFTable component by MichaL MutL. For other data sources ADO is used (Windows only). Native drivers use the DLL's from the database system (MySQL or Postgres).
- Version 1.4.12 up can use a native MySQL driver (based on Rangel Gustavo Reale's TMySQLDataset and Matthias Fichtner's mysql.pas) and includes a preliminary advanced series viewer based on EZDicom / K-Pacs (many thanks to Chris Rorden and Andreas Knopke). Fixes: strip group 2 information of any files sent, retry logic, worklist query.
- Version 1.4.12 improves database performance, has some important bug fixes (rare crashes, incomplete deletion and grabbing, and rare database corruption on dbaseIII). Further it has the possibility to forward multiple images on a single association, and improved documentation
- Version 1.4.12b and c add importconverters and bug fixes in dbaseIII driver and web access and do not allow .dcm with nki compression
- Version 1.4.13 has a web viewer based on K-PACS, SQLite is now included, and more import and export converter options were added such as delayed forwarding and preretrieval. Then small fixes were made in native mysql (also the driver is now included in the package), and several other aspects of the server, such as handling of compression errors, out of memory conditions, and others.
- Version 1.4.14 has a more web options (move, delete and viewers), more exportconverters and 'Number of Patient Related Studies' etc. query items

- Version 1.4.15alpha fixes small bugs; and adds QueryConverters, color jpeg decoding with built-in code, frame control, several command line options, an anonymization script, postgres support, and jpeg web export graphics
- Version 1.4.15 fixes more small bugs and adds more scripting options
- Version 1.4.16beta adds internal JPEG and JPEG2000 codecs adapted by Bruce, more scripting options, bug fixes and print to AE from GUI
- Version 1.4.16rc1 adds WADO, virtual query cache, zipping and cleanup at night also for a service, animated gif support, more commands and fixes
- Version 1.4.16rc2 fixes some bugs
- Version 1.4.16rc4 adds lua as very fast and flexible scripting language for converters (with access to configuration, connection, dicom objects, pixel data, database, queries) and web page design
- Full release 1.4.16 fixes several bugs

- SECTION A(2). CONFORMANCE STATEMENT


A.0 INTRODUCTION

This conformance statement (CS) details the MicroPACS compliance to DICOM v3.0. It covers all DICOM Service Class roles in this product:

- Various Storage Service Class (SCP) Roles
- Morphing Storage Service Class (SCU) Roles
- Patient Root Query & Retrieve (SCP) Roles
- Study Root Query & Retrieve (SCP) Roles
- Patient/Study Root Query & Retrieve (SCP) Roles
- Verification Service Class (SCP) Role
- Print Management (SCP) Role
- Worklist Query (SCP) Role
- WADO Server (http) Role

A.1 IMPLEMENTATION MODEL

A.1.1 Application Data Flow Diagram

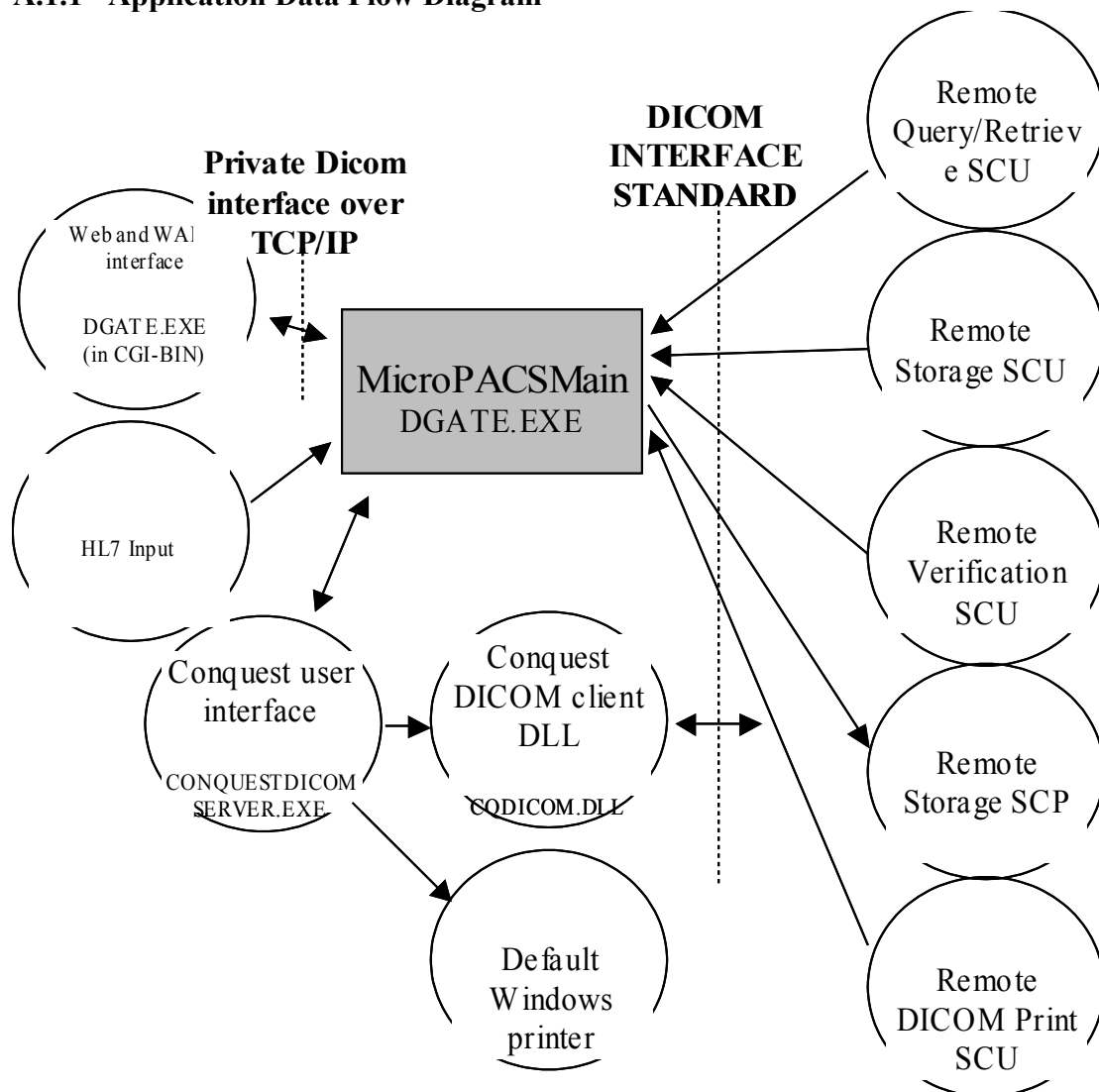

A.1.2 Functional Definition of AE's

Application Entity 1, MicroPACSMMain

Supports the following functions:

- Has access to patient demographics and pixel data in the local database.
- Responds to DICOM associations containing image and worklist query requests using the information in the local database.
- Moves IOD modules to any configured DICOM AE when requested to do so by a remote DICOM AE.
- Responds to DICOM associations containing storage requests and places the IOD in the local database.
- Responds to DICOM associations containing verification requests.
- Responds to DICOM associations containing print requests and prints items on the default local Windows printer.
- Responds to private DICOM associations to support the WEB interface and maintenance functions.

NOTE: The Conquest Applications (not part of the server release) or other DICOM network viewers (e.g., KPacs) will use separate AE's which **MUST** be configured in ACRNEMA.MAP (use the "**Known DICOM providers**" page) to allow access of images from the DICOM server.

A.1.3 Sequencing of Real - World Activities

Image Store:

- The remote AE will initiate a DICOM association.
- The MicroPACSMMain AE will select the appropriate Abstract and Transfer Syntax's from those proposed by the remote AE.
- The remote AE will initiate a C-Store to send the IOD.
- The MicroPACSMMain AE will respond with a C-Store-RSP upon receipt of the IOD.
- The following processing occurs using the WorkList database (can be enabled/disabled using 'WorkListMode' in DICOM.INI):
 - WorkListMode=0: no processing occurs.
 - WorkListMode=1: The AccessionNumber is looked up in the local WorkList database, if it is found, any element in the DICOM object that is also present (and non-NULL) in the WorkList database, will be replaced by the value from the WorkList database. These changes are made both in the database and in the image that is stored on disk.
 - WorkListMode=2: As mode 1, but the image will be refused if the AccessionNumber is not found.
 - Note: there is no DICOM method of filling the worklist database (see the description of WorkListMode).
- The following processing (can be enabled/disabled by defining 'FixPhilips = 0/1' in DICOM.INI) of the patient ID occurs to conform patient ID's generated by a Philips scanner with NKI policy:
 - From a patient ID of 10 digits (i.e., only exactly 10 digits) and a numeric value larger than 0001000000, starting with at least 2 zeros, the first 2 or 3 leading zeros are stripped. I.e., '0123456789' is not changed, '0020101234' is replaced by '20101234', '0009901234' is replaced by '9901234', and '0000012345' is replaced by 0012345. The

result is that a 10 digit ID from Philips that consists of a valid NKI patient ID with extra leading zeros is converted to a valid NKI patient ID. These changes are made both in the database and in the image that is stored on disk.

- The following processing (can be enabled/disabled by defining 'FixKodak = 0/1' in DICOM.INI) of the patient ID occurs to conform patient ID's generated by a Kodak RIS worklist with NKI policy:
 - From a patient ID of 8 digits (i.e., only exactly 8 digits) and a numeric value larger than 01000000, starting with at least 1 zero, the leading zero is stripped. I.e., '0123456789' is not changed, '09901234' is replaced by '9901234', and '00012345' is replaced by 0012345. The result is that a 8 digit ID from Kodak RIS that consists of a valid NKI patient ID of before 2000 with a superfluous leading zero, is converted to a valid NKI patient ID. These changes are made both in the database and in the image that is stored on disk.
- Trailing space are discarded from the patient ID.
- Up to 99 importConverters are called as scripts or rules to modify, delete or log images or VR's in them. These scripts can also start external programs and delete/retrieve/forward parts of the patient's information. All scripts can optionally be programmed in lua with full access to server configuration and dicom objects.
- The pixel data is (re or un)compressed if this option is enabled.
- The image is stored and disk and image header data is (re-)entered in the database at all four levels (patient, study, series, and image). The following consistency checking is performed on the data entered in the database (without changing the image information that is stored):
 - Inconsistent link information (e.g., two images of the same series belong to different patients), lead to a reject to store the new image with reported failure to the sending client. The rejected images are passed through script 'RejectedImageConverter0' to allow extensive logging or repair actions.
 - Filled items will not be overwritten by empty items.
 - Known sex (M or F) in the patient database will not be overwritten with any other value than M or F.
 - A known date of birth in the database will not be overwritten with an empty date or with a date on the 1st of January (which has a high probability to be wrong). When the original date of birth is empty, any value will be accepted.
 - In case of any other inconsistency, the newer values will be written in the database, and the change will be logged as a warning in serverstatus.log. Inconsistencies in the birthdate are also logged in PacsTrouble.log.
 - The (series) Modality field is appended to the Study Modality field in the database if it does not already contain this Modality.
 - The PatientName, PatientBirthDate and PatientSex items are duplicated in the study table (database rev8 and up), to allow detection of patient ID mix-ups.
- Optionally the image is processed or forwarded (compressed or uncompressed) if Modality and StationName match with values specified in dicom.ini and the optional ExportFilter test is passed (see appendix 7).
- Some logging of activity occurs.

Query/Retrieve:

- The remote AE will initiate a DICOM association.
- The MicroPACSMMain AE will select the appropriate Abstract and Transfer Syntax's from those proposed by the remote AE.
- Queries and retrieves can be inspected, modified or rejected by a scripts 'QueryConverter0' and 'Retrieve Converter0'. The script can also trigger move or deletes of associated data.
- Queries can be forwarded to up to 10 VirtualServerFor entries. The received data will be merged with the data from the server's database and cleaned of duplicates (see appendix 7). Query results may be cached to speed up repetetive quiriies.
- Optionally the 'Number of Patient Related Studies'-like items will be computed (this executes another query for each result of the first query) and is somewhat slow.
- The returned query results can be processed by a script called 'QueryResult'.
- Upon receipt of a C-Move request, the MicroPACSMMain AE will initiate an SSC/SCU association morphing to the stored IOD SOP Class to the specified and configured DICOM AE. Compressed pixel data will be decompressed or recompressed prior to transmission. Optionally a script 'RetrieveResultConverter0' will process the retrieved image data. A C-Move response message will be generated synchronously with the associated C-Store.
- Retrieval of data stored on one or more of the VirtualServerFor entries and not on the local server will initiate automatic transfer from the listed servers in the VirtualServerFor table to the local server, followed by (or overlapping) a transfer to the C-MOVE destination. After the retrieval data can be optionally deleted again (see appendix 7).
- Some logging of activity occurs.

Worklist Query:

- The remote AE will initiate a DICOM association.
- The MicroPACSMMain AE will select the appropriate Abstract and Transfer Syntax's from those proposed by the remote AE.
- Queries can be inspected, modified or rejected by a script called 'WorkListQueryConverter0'.
- The MicroPACSMMain AE will query the Worklist database and respond with zero or more modality worklist items. The sequence structure of the responses duplicates that of the query.
- The query results can be processed with a script called 'ModalityWorkListQueryResultConverter0'.
- Some logging of activity occurs.
- Note: there is no DICOM method of filling the worklist database. It can be filled through the web interface, by drag and dropping hl7 files or programatically (see the description of WorkListMode).

Verification:

- The remote AE will initiate a DICOM association.
- The remote AE will initiate a C-ECHO.
- The MicroPACSMMain AE will respond with a C-ECHO-RSP.
- Many private command options can be added to the C-ECHO command.
- Some logging of activity occurs.

DICOM Print:

- The remote AE will initiate a DICOM association.
- The remote AE will create a basic film session using N-CREATE.
- The MicroPACSMMain AE will ignore the information but will respond with a N-CREATE-RSP.
- The remote AE will create a basic film box using N-CREATE.
- The MicroPACSMMain AE extracts the Image Display Format (only “STANDARD\#rows,#cols” is accepted), and the film orientation (LANDSCAPE or PORTRAIT) and passes this information to the CONQUEST user interface. All other information is ignored.
- The MicroPACSMMain AE creates the correct amount of Basic Grayscale or Color Image Box objects for the film page and transmits their UIDs to the remote AE in the N-CREATE-RSP. The UIDs contain information about the page number, number of rows and columns, and the image location on the page that will be used by the CONQUEST user assemble the printed page.
- The remote AE will use N-SET to fill each Image Box object.
- The MicroPACSMMain AE will store each incoming Image Box onto disk (in directory “printer_files” on device MAG0) and responds with N-SET-RSP. The name (UID) of the files is passed to the CONQUEST user interface.
- The CONQUEST user interface (Windows only) will queue incoming images and will asynchronously convert each DICOM file into a BMP file, load it in memory and assemble the pictures to be printed on a page. Processed DICOM files and BMP files are deleted. *Note: the basic print support in the CONQUEST user interface will not handle multiple simultaneous print requests correctly!*
- The remote AE will request printing of each film or of the complete session using an N-ACTION command for a basic film session or a basic film box.
- The MicroPACSMMain AE passes these requests onto the CONQUEST user interface and responds with an N-ACTION-RSP.
- The CONQUEST user interface (Windows only) prints the pages on the default Windows printer. The only way to configure this printer is to change its default document settings in Windows. Printing progress is shown using a simple progress bar on the server status page.
- The remote AE may query the printer status with a N-GET request on the printer object.
- The MicroPACSMMain AE will always respond with a N-GET-RSP with a “NORMAL” status and the name of the printer, which is pre-set to “Conquest dicom printer”.
- Other N-DELETE, N-SET, and N-EVENTREPORT requests are acknowledged with an adequate RSP and ignored.
- Some logging of activity occurs.

Web and WADO access:

- The remote client will request a web page from cgi-application dgate.exe
- The cgi-application will fill a page with information. Patient information is requested over a private DICOM protocol (e.g., query2 or patientlist) from the main server to fill tables. Furthermore links may be added to:
 - GIF, BMP or JPG images
 - Viewer controls
- Links to GIF/BMP/JPG images will access the cgi-application, which in turn will request them from the main server over a private DICOM

protocol. The main server is responsible for converting DICOM to GIF/BMP/JPG.

- The CGI application can generate arbitrary web pages based on a simple scripting language and/or lua scripts.
- Since 1.4.16rc1, the server can honor WADO requests.
- The links to the viewer controls (OCX or Java) have two purposes. First the control will be downloaded and installed in the web viewer. This requires reduced security settings (trusted site), download of Java and/or installation of the OCX control with regsrv32.exe. Then the viewer controls will request DICOM images as needed from the cgi-application, which in turn will request these from the main server over a private DICOM protocol. The main server is responsible for (de)compressing and optionally downsizing the DICOM images.

```
client    -> webserver          url      http://127.0.0.1/scripts/dgate.exe?mode=seriesviewer&series=...
webserver -> dicomserver        query   imagelister:CONQUESTSRV1|.....
client    <- webserver <- dicomserver query results      (to build list of urls of dicom slices)

client    <- webserver          activex control http://127.0.0.1/ActiveFormProj1.ocx

Then for each slice that is required:

control   -> webserver          url      http://127.0.0.1/scripts/dgate.exe?mode=dicom&slice=.....
webserver -> dicomserver        request  convert_to_dicom:....
control   <- webserver <- dicomserver dicom data
```

A.2 AE SPECIFICATIONS

A.2.1 AE1 Specification

This Application Entity provides Standard Conformance to the following DICOM v3.0 SOP Classes as an SCU:

SOP Class Name	SOP Class UID
Verification (Echo)	1.2.840.10008.1.1
* Unknown IOD Storage	* See note

NOTE: This MicroPACS will initiate outgoing DICOM C-STORE requests masquerading as any stored IOD module. The behavior of this outgoing association link will be like the DICOM defined SCU role: Storage Service Class.

This Application Entity provides Standard Conformance to the following DICOM v3.0 SOP Classes as SCP:

SOP Class Name	SOP Class UID
Verification (Echo)	1.2.840.10008.1.1
Patient Root Query/Retrieve Info. Model -FIND	1.2.840.10008.5.1.4.1.2.1.1
Patient Root Query/Retrieve Info. Model - MOVE	1.2.840.10008.5.1.4.1.2.1.2
Study Root Query/Retrieve Info. Model –FIND	1.2.840.10008.5.1.4.1.2.2.1
Study Root Query/Retrieve Info. Model - MOVE	1.2.840.10008.5.1.4.1.2.2.2
Patient Study Only Query/Retrieve Info. Model -FIND	1.2.840.10008.5.1.4.1.2.3.1
Patient Study Only Query/Retrieve Info. Model - MOVE	1.2.840.10008.5.1.4.1.2.3.2
Basic Grayscale Print Management Meta SOP Class	1.2.840.10008.5.1.1.9
Basic Color Print Management Meta SOP Class	1.2.840.10008.5.1.1.18
Modality WorkList Query	1.2.840.10008.5.1.4.31
* Unknown IOD Storage	* See note

NOTE: This MicroPACS will accept any incoming DICOM C-STORE request sent using the DICOM defined SCP role: Storage Service Class. The dgatesop.lst file (see configuration section and dicom.ini file) can be used to selectively restrict this ability.

A.2.1.1 Association Establishment Policies

A.2.1.1.1 General

The DICOM Application Context Name (ACN) that is always proposed is:

Application Context Name	1.2.840.10008.3.1.1
--------------------------	---------------------

The Maximum Length PDU negotiation is included in all association establishment requests. The maximum length PDU for an association initiated by the NetMain AE is:

Maximum Length PDU	16Kbytes
---------------------------	-----------------

The SOP class Extended Negotiation is not supported and ignored.

There is no limit on the maximum number of Presentation Contexts Items that will be proposed. In this implementation, each Abstract syntax will be proposed with either a single Transfer Syntax, or with a few JPEG(2000) transfer syntaxes, depending on the configuration in *acrnama.map*.

The user info items sent by this product are:

- Maximum PDU Length
- Implementation UID
- Implementation Version

Note: Max PDU length is not configurable at run time.

A.2.1.1.2 Number of Associations

MicroPACSMMain AE will initiate one DICOM association to perform image store for each concurrently incoming C-MOVE request.

There is no artificial maximum placed on the number of simultaneous DICOM associations open at one time. It should be noted that system response time will be degraded, and this could possibly adversely effect a time-out period on other remote AE's.

The Print Management function will correctly handle multiple simultaneous associations, but the elementary print support in the CONQUEST user interface will not correctly print (i.e., images on the pages will be mixed up) when multiple print jobs are sent simultaneously.

A.2.1.1.3 Asynchronous Nature

Asynchronous mode is not supported. All operations will be performed synchronously.

A.2.1.1.4 Implementation Identifying Information

The Implementation UID allows unique identification of a set of products that share the same implementation.

The Implementation UID for this ID/Net v3.0 Implementation is:

Storage & Q/R UID	1.2.826.0.1.3680043.2.135.1066.101
------------------------------	---

A.2.1.2 Association Initiation by Real-World Activity

A.2.1.2.1 Proposed Presentation Contexts

Presentation Context Table – Proposed					
Abstract Syntax & configuration		Transfer Syntax		Role	Expanded Negotiation
*Unknown	un,as,n1..n4,uj,nj	Little Endian	1.2.840.10008.1.2	SCU	None
*Unknown	j3..j6	JPEGBaseLine1	1.2.840.10008.1.2.4.50	SCU	None
*Unknown	j3..j6	JPEGExtended2and4	1.2.840.10008.1.2.4.51	SCU	None
*Unknown	j5	JPEGSpectralNH6and8	1.2.840.10008.1.2.4.53	SCU	None
*Unknown	j6	JPEGFullNH10and12	1.2.840.10008.1.2.4.55	SCU	None
*Unknown	j2	JPEGLosslessNH14	1.2.840.10008.1.2.4.57	SCU	None
*Unknown	j1, j2	JPEGLossless	1.2.840.10008.1.2.4.70	SCU	None
*Unknown	jk	JPEG2000 Lossless	1.2.840.10008.1.2.4.90	SCU	None
*Unknown	jl	JPEG2000 Lossy	1.2.840.10008.1.2.4.91	SCU	None

Note: Due to the morphing nature of the outgoing SSC-SCU engine, the specific Abstract Syntax that is proposed depends upon the nature of the stored image. The actual proposed Transfer Syntaxes depend on the configuration in *acrnema.map* and are the same for each class of stored images.

A.2.1.2.1.2 SOP Specific Conformance Statement for Image Storage SOP Class

This implementation can perform multiple C-STORE operations over a single association.

Upon receiving a C-STORE confirmation containing a successful status, this implementation will perform the next C-STORE operation. The association will be maintained.

Upon receiving a C-STORE confirmation containing an Error, Refused or Warning status, this implementation will fail the specific IOD in question. If more images need to be sent, they will be sent in the same association.

A.2.1.2.2 Association Acceptance Policy

A.2.1.2.2.1 Real-World Activity

This AE accepts associations for the Query/Retrieve (Q/R) SOP using the Patient Root, Study Root, and Patient/Study Only Query Model.

This AE accepts associations for the Image Storage Class using any defined IOD class.

This AE accepts associations for the Verification Service Class.

This AE accepts associations for the Print Service Class.

This AE accepts associations for the WorkList Query Service Class.

A.2.1.2.2.2 Real-World Activity

This AE is indefinitely listening for Q/R, Storage Class, Verification and Print Management associations

A.2.1.2.2.3 Proposed Presentation Contexts

Presentation Context Table – Accepted					
Abstract Syntax		Transfer Syntax		Role	Expanded Negotiation
Patient Root Query/Retrieve Info. Model – FIND	1.2.840.10008.5.1.4.1.2.1.1	Little Endian	1.2.840.10008.1.2	SCP	None
Patient Root Query/Retrieve Info. Model – MOVE	1.2.840.10008.5.1.4.1.2.1.2	Little Endian	1.2.840.10008.1.2	SCP	None
Study Root Query/Retrieve Info. Model – FIND	1.2.840.10008.5.1.4.1.2.2.1	Little Endian	1.2.840.10008.1.2	SCP	None
Study Root Query/Retrieve Info. Model – MOVE	1.2.840.10008.5.1.4.1.2.2.2	Little Endian	1.2.840.10008.1.2	SCP	None
Patient/Study Only Query/Retrieve Info. Model – FIND	1.2.840.10008.5.1.4.1.2.3.1	Little Endian	1.2.840.10008.1.2	SCP	None
Patient/Study Only Query/Retrieve Info. Model – MOVE	1.2.840.10008.5.1.4.1.2.3.2	Little Endian	1.2.840.10008.1.2	SCP	None
Modality WorkList Query	1.2.840.10008.5.1.4.31	Little Endian	1.2.840.10008.1.2	SCP	None
Basic Grayscale Print Management Meta SOP Class	1.2.840.10008.5.1.1.9	Little Endian	1.2.840.10008.1.2	SCP	None
Basic Color Print Management Meta SOP Class	1.2.840.10008.5.1.1.18	Little Endian	1.2.840.10008.1.2	SCP	None
*Unknown	*Unknown	Little Endian*	1.2.840.10008.1.2	SCP	None
Verification	1.2.840.10008.1.1	Little Endian	1.2.840.10008.1.2	SCP	None

Note: Due to the morphing nature of the incoming SSC-SCP engine, the specific Abstract Syntax accepted will depend upon the nature of the stored image, and the *dgatesop.lst* configuration file (of which a default version is automatically created when installing the Conquest DICOM server).

*The server can accept many transfer syntaxes as configurable by *dgatesop.lst*.

A.2.1.2.2.4 SOP Specific Conformance for Query/Retrieve FIND SOP Class SCP

The C-FIND response status values are supported as defined in DICOM v3.0 Part 4.

All Required (R) and Unique (U) Study, Series, and Image Level Keys for the Patient Root, Study Root, and Patient/Study Only Query/Retrieve Information Model are supported. Many optional (O) Keys are supported, as described later in this document.

A.2.1.2.2.5 SOP Specific Conformance for Query/Retrieve MOVE SOP Class SCP

Prioritization of C-FIND & C-MOVE requests is all set to normal - 0.

All images requested in a single C-MOVE will be sent over a single association (the association will not be re-established for each image).

A.2.1.2.2.6 SOP Specific Conformance for “Unknown” Storage SCP

The specific Storage SCP classes accepted are programmable (by the user) at runtime, and cannot be explicitly stated here.

No optional elements are discarded.

The duration of the storage is temporary. Least recently added patients are deleted when the disk space is less than the amount specified in the “Cleanup disk space below (MB)” field in the Conquest DICOM server. This amount is run-time configurable. When the DICOM server is connected to a, e.g., jukebox archival system, the duration of storage can be made permanent.

A.2.1.2.2.7 Presentation Context Acceptance Criterion

No criterion.

A.2.1.3 Transfer Syntax Selection Policies

The server can accept most transfer syntaxes as configurable by *dgatesop.lst*. Outgoing connections can be made over uncompressed, loss-less or lossy JPEG(2000) transfer syntaxes. If so, the images are recompressed to conform to the accepted transfer syntax. The configuration is done through *acrnama.map*.

Configuration	Proposed transfer syntaxes	Name
un ¹ , as ² , n1..n4 ³ , nj ⁴ , uj ⁴	1.2.840.10008.1.2	ImplicitLittleEndian
ub	1.2.840.10008.1.2.1/2	ExplicitLittle/BigEndian
ul	1.2.840.10008.1.2.1	ExplicitLittleEndian
u2	1.2.840.10008.1.2.2	ExplicitBigEndian
j1	1.2.840.10008.1.2.4.70	JPEG Lossless sv1
	1.2.840.10008.1.2	ImplicitLittleEndian
j2	1.2.840.10008.1.2.4.57	JPEG Lossless sv 6
	1.2.840.10008.1.2.4.70	JPEG Lossless sv1
	1.2.840.10008.1.2	ImplicitLittleEndian
j3, j4	1.2.840.10008.1.2.4.51	JPEG extended (12 bits)
	1.2.840.10008.1.2.4.50	JPEG baseline (8 bits)
	1.2.840.10008.1.2	ImplicitLittleEndian
j5	1.2.840.10008.1.2.4.53	JPEG spectral selection
	1.2.840.10008.1.2.4.51	JPEG extended (12 bits)
	1.2.840.10008.1.2.4.50	JPEG baseline (8 bits)
	1.2.840.10008.1.2	ImplicitLittleEndian
j6	1.2.840.10008.1.2.4.55	JPEG progressive
	1.2.840.10008.1.2.4.51	JPEG extended (12 bits)
	1.2.840.10008.1.2.4.50	JPEG baseline (8 bits)
jk	1.2.840.10008.1.2	ImplicitLittleEndian
	1.2.840.10008.1.2.4.90	JPEG2000 Lossless
jl	1.2.840.10008.1.2.4.91	JPEG2000 Lossy

Note: The transfer syntaxes are listed in order of priority. I.e., if a host is configured as j1 and it accepts JPEG lossless, the image will be lossless JPEG compressed before transmission, even if it was not stored in that way.

- 1) un = uncompressed. Images will be decompressed prior to transmission.
- 2) The configuration “as” will transmit images as-is. Independent of how images are stored on disk (with JPEG or NKI compression), they will be transmitted over an ImplicitLittleEndian connection. This behavior does not conform to the DICOM standard and for many hosts this may therefore not work. NKI clients will work, though.
- 3) Configurations n1..n4 will transmit images with NKI private compression, which can only be read by NKI clients.
- 4) nj = nki compressed, leave jpeg as is; uj = uncompressed, leave jpeg as is

A.3. COMMUNICATION PROFILES

A.3.1 Supported Communication Stacks (parts 8,9)

DICOM Upper Layer (Part 8) is supported using TCP/IP.

A.3.2 OSI Stack

OSI stack not supported.

A.3.3 TCP/IP Stack

The TCP/IP stack is inherited from the Windows/Linux operating System. (Conquest addition): We have used the excellent Delphi TCP/IP components from François Piette (<http://www.overbyte.be>) in the CONQUEST user interface.

A.3.3.1 API

Developers can base client programs on **TEST.EXE** and **DICOMP.EXE** that are included with source code in the **DICOMLIB1416.ZIP** release file. The actual DICOM server (with many options) is **DGATE.EXE** that is included with source code in release file **DGATE1416.ZIP** and **DICOMLIB1416.ZIP**. Source code of the Windows user interface, client DLL (used for queries only) and web viewer OCX is *not* included. Developers can also add scripts in a private scripting language and now also lua to several core server tasks to provide extensive processing of DICOM objects such as images and queries.

A.3.3.2 Physical Media Support

Any Windows (Linux) supported physical media.

A.3.4 Point-to-Point Stack

A 50 pin ACR-NEMA connection is not supported.

A.4. EXTENSIONS / SPECIALIZATION / PRIVATIZATIONS

VR (0x7fdf, 0x0010) contains compressed pixel data if NKI compression is used. In the normal configuration, the server will never transmit NKI compressed data, but will compress and decompress the data on the fly. The following presentation contexts have been added to allow NKI private compressed data to be transmitted and to allow retrieval of downsized images for higher speed:

1.2.826.0.1.3680043.2.135.1066.5.1.4.1.2.1.2

PatientRootRetrieveNKI

1.2.826.0.1.3680043.2.135.1066.5.1.4.1.2.2.2	StudyRootRetrieveNKI
1.2.826.0.1.3680043.2.135.1066.5.1.4.1.2.3.2	PatientStudyOnlyRetrieveNKI

These contexts contain sub-classed versions of the standard C-MOVE commands with the following additional optional control parameters (numbers are hexadecimal):


(9999, 0100): MaxVrSize	Do not send VRs larger than this number of bytes
(9999, 0200): MaxRowsColumns	Downsize image to maximal this #rows and columns
(9999, 0201): Frame	If given, return only the selected frame of a MF object
(9999, 0300): ConsoleText	Text will be printed to server console
(9999, 0400): Silent	If sent, console does not log transaction
	Also used internally by GUI and WEB interface
(9999, 0500): MaxSlices	Send at most # slices irrespective of query result
(9999, 0600): MaxCompression	Maximum supported compression by NKI client (default 4)
(9999, 0700): Recompression	Recompress style NKI or JPEG (default none)
(9999, 0802): VirtualServerMask	Restrict access to selected virtual servers (default all used)
(9999, 0900): Script	'ImportConverter' with which transmitted objects are processed

SECTION 3 DEFAULT QUERY/RETRIEVE INFORMATION MODEL DEFINITION


This section describes the subset of the DICOM v3.0 Patient Root, Study Root, and Patient/Study Only, Query/Retrieve Information Model Definition used by this product.

3.0 INTEROPERABILITY SCHEMA


3.0.1 PATIENT ROOT QUERY/RETRIEVE ENTITY RELATIONSHIP

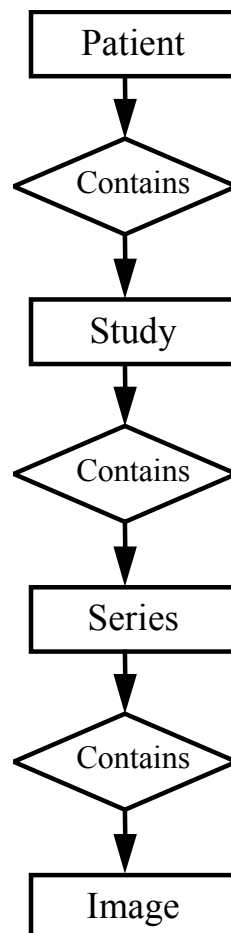

3.0.2 STUDY ROOT QUERY/RETRIEVE ENTITY RELATIONSHIP

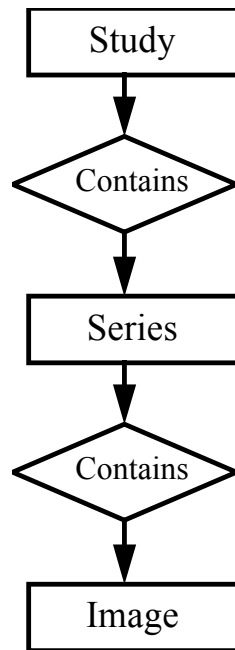

3.0.3 PATIENT/STUDY ONLY QUERY/RETRIEVE ENTITY RELATIONSHIP

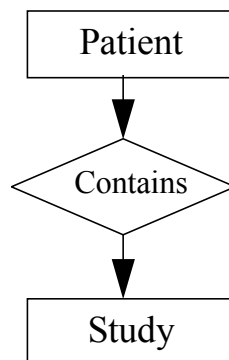

3.1 ENTITY DESCRIPTIONS

See DICOM Standard Part 4

3.2 PATIENT ROOT QUERY/RETRIEVE INFORMATION OBJECT DEFINITION

3.2.1 Patient Level Keys for Patient Root Query/Retrieve Information Model

Attribute Name	Element Tag	Type	Notes
Revision: 10 (plus computed items)			
Patient Name	(0010,0010)	R	
Patient ID	(0010,0020)	U	
Patient Birth Date	(0010,0030)	O	
Patient Sex	(0010,0040)	O	Computed* - not database
Number of Patient Related Studies	(0020,1200)	O	
Number of Patient Related Series	(0020,1202)	O	Computed* - not database
Number of Patient Related Instances	(0020,1204)	O	Computed* - not database

*requires setting EnableComputedFields = 1 in dicom.ini

3.2.2 Study Level Keys for Patient Root Query/Retrieve Information Model

Attribute Name	Element Tag	Type	Notes
Revision: 17 (plus computed items)			
Study Instance UID	(0020,000D)	U	
Study Date	(0008,0020)	R	
Study Time	(0008,0030)	R	
Study ID	(0020,0010)	R	
Study Description	(0008,1030)	O	
Accession Number	(0008,0050)	O	
Referring Physician	(0008,0090)	O	
Patients Age	(0010,1010)	O	
Patients Weight	(0010,1030)	O	
Study Modality	(0008,0061)	O	Multiple entry item, automatically updated by server Computed* - not database
Number of Study Related Series	(0020,1206)	O	
Number of Study Related Instances	(0020,1208)	O	Computed* - not database

*requires setting EnableComputedFields = 1 in dicom.ini

3.2.3 Series Level Keys for Patient Root Query/Retrieve Information Model

Attribute Name	Element Tag	Type	Notes
Revision: 17 (plus computed items)			
Series Instance UID	(0020,000E)	U	
Series Number	(0020,0011)	R	

Series Date	(0008,0021)	R	
Series Time	(0008,0031)	R	
Series Description	(0008,103E)	O	
Modality	(0008,0060)	O	
Station Name	(0008,1010)	O	
Institution	(0008,0080)	O	
Patient Position	(0018,5100)	O	
Contrast Bolus Agent	(0018,0010)	O	
Manufacturer	(0008,0070)	O	
Model Name	(0008,1090)	O	
Body Part Examined	(0018,0015)	O	
Protocol Name	(0018,1030)	O	
Frame of Reference UID	(0020,0052)	O	Computed* - not database
Number of Series	(0020,1209)	O	
Related Instances			

*requires setting EnableComputedFields = 1 in dicom.ini

3.2.4 Image Level Keys for Patient Root Query/Retrieve Information Model

Attribute Name	Element Tag	Type	Notes
Revision: 13			
SOP Instance UID	(0008,0018)	U	
SOP Class UID	(0008,0016)	O	
Image Number	(0020,0013)	O	
Image Date	(0008,0023)	O	
Image Time	(0008,0033)	O	
Echo Number	(0018,0086)	O	Multiple entry item
Number Of Frames	(0028,0008)	O	
Acq Date	(0008,0022)	O	
Acq Time	(0008,0032)	O	
Receiving Coil	(0018,1240)	O	
Acq Number	(0020,0012)	O	
Slice Location	(0020,1041)	O	
Samples Per Pixel	(0028,0002)	O	
Photometric Interpret.	(0028,0004)	O	
Rows	(0028,0010)	O	
Columns	(0028,0011)	O	
Bits Stored	(0028,0101)	O	
Image Type	(0008,0008)	O	Multiple entry item
Image ID	(0054,0400)	O	

3.3 STUDY ROOT QUERY/RETRIEVE INFORMATION OBJECT DEFINITION

3.3.1 Study Level Keys for Study Root Query/Retrieve Information Model

Attribute Name	Element Tag	Type	Notes
Revision: 17 (plus computed items)			

Patient Name	(0010,0010)	O	
Patient ID	(0010,0020)	O	
Patient Birth Date	(0010,0030)	O	
Patient Sex	(0010,0040)	O	
Study Instance UID	(0020,000D)	U	
Study Date	(0008,0020)	R	
Study Time	(0008,0030)	R	
Study ID	(0020,0010)	R	
Study Description	(0008,1030)	O	
Accession Number	(0008,0050)	O	
Referring Physician	(0009,0090)	O	
Patients Age	(0010,1010)	O	
Patients Weight	(0010,1030)	O	
Study Modality	(0008,0061)	O	Multiple entry item – automatically updated by server Computed* - not database
Number of Study Related Series	(0020,1206)	O	
Number of Study Related Instances	(0020,1208)	O	Computed* - not database

*requires setting EnableComputedFields = 1 in dicom.ini

3.3.2 Series Level Keys for Study Root Query/Retrieve Information Model

See Section 3.2.3

3.3.3 Image Level Keys for Study Root Query/Retrieve Information Model

See Section 3.2.4

3.4 PATIENT/STUDY ONLY QUERY/RETRIEVE INFORMATION OBJECT DEFINITION

3.4.1 Patient Level Keys for Patient/Study Only Query/Retrieve Information Model

See Section 3.2.1

3.4.2 Study Level Keys for Patient/Study Only Query/Retrieve Information Model

See Section 3.2.2

3.5 MODALITY WORKLIST QUERY INFORMATION OBJECT DEFINITION

3.5.1 Keys for Modality Worklist Query Information Model

Attribute Name	Element Tag	Type	Notes
Revision: 17			
Accession Number	(0008,0050)	O	Primary Key
Patient ID	(0010,0020)	R	
Patient Name	(0010,0010)	R	
Patient Birth Date	(0010,0030)	O	
Patient Sex	(0010,0040)	O	
Medical Alerts	(0010,2000)	O	
Contrast Allergies	(0010,2110)	O	
Study Instance UID	(0020,000D)	O	
Requesting Physician	(0032,1032)	O	
Requested Procedure Description	(0032,1060)	O	
Requested Procedure Code Sequence	(0032,1064)	R	1 instances of this sequence required
>Modality	(0008,0060)	R	
>Requested contrast agent	(0032,1070)	O	
>Scheduled AE	(0040,0001)	R	
>Start date	(0040,0002)	R	
>Start time	(0040,0003)	R	
>Performing Physician	(0040,0006)	R	
>Scheduled Procedure Step Description	(0040,0007)	O	
>Scheduled Procedure Step ID	(0040,0009)	O	
>Scheduled Station Name	(0040,0010)	O	
>Scheduled Procedure Step Location	(0040,0011)	O	
>Premedication	(0040,0012)	O	
>Scheduled Procedure Step Comments	(0040,0400)	O	
Requested Procedure ID	(0040,1001)	O	
Requested Procedure Priority	(0040,1003)	O	
Character set	(0008,0005)	*	Returned as “ISO_IR 100” only when WorkListReturnsISO_IR_100 is set

To code this information into a database, the contents of the sequence are unfolded to the same level as the other fields. *This field will be optionally returned whether requested or not.

4.1 FILES

Installation files (in DicomServer1416.zip):

windowsmanual.pdf	Documentation
DicomConformance_FilesLST_Changes.pdf	
conquestdicomserver.exe	User interface
dgate.exe	console mode version of MicroPACS system
dgate64.exe	Idem for 64 bits (automatically selected)
dgate.dic	DICOM dictionary for dgate.exe
cqdicom.dll	Conquest DICOM client DLL.
dgateserv.exe	NT service wrapper for dgate.exe V1.1
zipdll.dll	ZipMaster DLL zips log files (optional, else uses 7za.exe)
libmysql.dll	Mysql client dll for 5.0.22
libmysql64.dll	64 bit mysql client dll for 5.0.22
libmysql.txt	Mysql license text
libpq64.dll	64 bit client DLL for postgres (hard to find)
7za.exe	7-zip command line version
7za.txt	7-zip license text
pscp.exe	putty ssh2 copy utility
pcsp.txt	putty license text
jpeg2yuv.exe	mpegtools utility to stream a series jpg images
mpeg2enc.exe	mpegtools utility to encode a mpeg2 movie
mpegtools.txt	license file for mpegtools
sample.hl7	Example HL7 input file
webserver\ActiveFormProj1.ocx	ActiveX web viewer
webserver\conquest.jpg	conquest logo for on web page
webserver\dicom.ini	sample web page configuration
webserver\sample.cq	sample scripted web page
webserver\dgate.exe	same as above, here used as CGI application
data\HEAD_EXP_00097038\0001_002000_892665661.v2	
data\HEAD_EXP_00097038\0001_003000_892665662.v2	

Configuration files (automatically generated):

dicom.ini	Configuration file
dicom.sql	Database table definition file
acrnama.map	System/IP map file
dgatesop.lst	SOP Class definition file
USE.....	Determines database driver to be installed

Database files (automatically generated, example for SQLite):

data\dbase\conquest.db3	Database tables
data\dbase\XDicompatients.dbf	extract of database table used by browser for dbaseIII
data\dbase\XDicomstudies.dbf	extract of database table used by browser for dbaseIII
data\dbase\XDicomseries.dbf	extract of database table used by browser for dbaseIII
data\dbase\XDicomimages.dbf	extract of database table used by browser for dbaseIII
data\dbase\XDicomworklist.dbf	extract of database table (not used)
data\printer_files	temporary and printer files
data\printer_files\querycache	optional cache for repetitive queries to virtualservers
data\incoming	optional folder – monitored for data to load

Log files (automatically generated):

Installation.log	information from installation page
Maintenance.log	information from maintenance page
ServerStatus.log	information from server status page or service
PacsTrouble.log	relevant errors
PacsUser.log	all users who accessed the server
ConquestDicomServer.log	log about main actions and server start/stop
logs\logs_yyyymmdd.zip	automatically zipped set of log files

Data files (automatically generated – 5678 is the servers port #):

ExportFailures5678_0	data of failed exports for later retrying
CopyFailures5678	data of failed mirror copies for later retrying
DelayedFetchForwardFailures5678	data of failed delayed forwards
Note: The server deletes these files (or delete manual) to abort retrying	

Image file (example, for FileNameSyntax parameter set to 3) :

Dir	PatientID	seriesUID	series	slice	time	counter
Data\QA_phantom\	1.2.840.113619.2.22.287.1.19980710.202026_0001_000006_9537982720005.v2					

4.2 PRODUCT INTERACTION / DESCRIPTION

- dgate.exe:** DICOM MicroPACS AE Main. This is the principal PACS application. It handles all connections (see threading model note in section 4.4 of this part). Contained in this executable are verification, storage, print, and various query and retrieve DICOM SOP classes. This executable interfaces to the PACS database via ODBC, MySQL or Postgres drivers, or using a built-in SQLite or dBaseIII driver. In the event that disk space is low, it will optionally delete least recently entered patients. This application also contains code for installation, maintenance, and archival operations that is used by the Conquest DICOM server. For these purposes a second copy of dgate.exe is started. Type “**dgate -?**” for more information. Finally, the same executable may act as CGI server application. In some rare cases, the dgate process may still be running even though the user interface has closed (after a crash or after killing it with the task manager). The user interface, when restarted, then assumes that the dgate process belongs to a service (and displays “**user interface**” in the caption), but cannot control the running process. Killing the dgate process with the task manager or rebooting will solve this problem. All source code of dgate.exe is included in **DGATE1416.ZIP** and **DICOMLIB1416.ZIP**. To compile dgate.exe one can use total.cpp (documentation included).
- dgate64.exe** Identical application for use in a 64 bits windows OS. Dgate.exe or dgate64.exe are automatically selected. Delete dgate64.exe to force use of 32 bits in a 64 bit OS.
- dgate.dic:** DICOM dictionary used by dgate.exe.
- dgateserv.exe:** Wrapper (V1.2) for dgate.exe to make it run as a Windows service. May be used (advanced users only) to manually (even remotely) start / stop / install / uninstall service(s). Type “**dgateserv /?**” for more information. Note that DICOM server services are named after the AE of the server. Once dgate.exe is installed as a service, the DICOM server(s) starts automatically when the computer is booted. When the user interface is started it will detect that the server is already running and connect to it. Source code is not included. Service install for Windows7 requires running the server in administrator mode.
- conquestdicomserver.exe:**
This executable is the user interface and installation program and contains parts of the print server. See the installation section of this document for its functions. The database browser tries to use the Borland Database Engine. Without BDE, the server will run fine, except one filter option in the browser. Source code is not included.
- cqdicom.dll:** This is code for the Conquest DICOM client. It is used here to convert DICOM images for printing, list the image file header and provide query, move and echo actions. The source code of

this DLL is not included but it is based on the programs “TEST.EXE” and “DICOMP.EXE” that are included with source code.

Libmysql.dll: When present, provides native access to MySQL, both for GUI and the server core. Redistributed because binary compatibility is required between compiled applications and the interface DLL: i.e. this file has to be of a specific version, even if it is used to communicate with MySQL databases of other versions. Comes from <http://downloads.mysql.com/archives/mysql-5.0/mysql-noinstall-5.0.22-win32.zip>

Libmysql64.dll: provides native access to MySQL for 64 bits server core

Libpq64.dll: provides (hard to find) native access to PostGres for 64 bits server core. Comes from: <http://forum.manifold.net/forum/t49971.21>. Note: the 32-bit PostGres client DLL's are not included in the distribution and must be taken from the bin folder of your PostGres distribution.

7za.exe: 7-zip command line version: used by the server core to unzip dropped zip files with dicom files, to zip log files, and to zip dicom data sets for submission over ssh-2.

pcsp.exe: used by the submit script and server commands to submit data over a ssh-2 connection.

jpeg2yuv.exe mpegtools utility to stream a series jpg images

mpeg2enc mpegtools utility to encode a mpeg2 movie. Used by the WADO server when mpeg is requested

Web server (place these 5 files in your web server):

Cgi-bin/Dgate.exe: exactly the same dgate.exe as in the server.

Cgi-bin/Dicom.ini: specialized ini file for server running as web(cgi) client.

Cgi-bin/sample.cq: Sample of script-based web page

Htdocs/ActiveFormProj1.ocx: Optional ActiveX web viewer, to be registered manually or downloaded from your web server and installed.

Htdocs/conquest.jpg: Icon for use on the web pages

4.3 THREADING ARCHITECTURE

The MicroPACS system was designed to run in two multithreading models. The first model is “Multithreaded”, the second is “Multiprocess”. The Multithreaded model can best be described as a monolithic application. Each incoming DICOM association is allocated a separate thread of execution within the same memory and address space of the master process. These threads share the same code within the single (monolithic) executable. The Multiprocess model is similar to “inet” on UNIX operating systems. There is a controlling application (conquestdicomserver.exe) which handles incoming TCP/IP connections. For each connection, a new copy of the MicroPACS

main executable (dgate.exe) is loaded and executed. A separate process that runs in a separate address space from all other associations handles each DICOM association.

The multithreaded application is slightly faster than the multiprocess one; But the multiprocess one is more secure, and should be more reliable. Because each process runs in it's own address space, an association from one connection cannot corrupt or undermine (in any possible way) another simultaneous association. For this reason, the default installation of the MicroPACS was the multiprocess model.

However, extensive testing of the Conquest DICOM server showed that the multiprocess version sometimes failed to work correctly on Windows (some ODBC problem). For this reason, the multithreaded version is currently used with the Conquest DICOM server. Several fixes have increased its speed and made it very reliable.

Also, the built-in DBASEIII driver applies in-memory indexing and depends on the server process staying in memory to achieve a high speed. In the Linux version, the built-in DBASEIII driver, SQLite, PostGres and MySQL are the database drivers available.

APPENDIX 1.

Fixes, modifications and extensions of MicroPACS by Marcel van Herk and Lambert Zijp of the Conquest project:

DICOM.SQL (Database definition):

19980624	StudyNumber (StudyID in DICOM) is now 64 long Required for correct operation with SQL server
19990109	Made imagenumber 10 characters wide
19991222	Moved MODALITY to the series level (for DICOM-RT)
19990315	Added EchoNumber and NumberOfFrames in image table
20010830	Revision to 4, extended database significantly
20010416	Revision 5: Added patientID in series and images for more efficient querying
20010416	Revision 6: Added frame of reference UID in series table
20020519	Revision 7: Added ImageType in image table, StudyModality to 64 chars, AcqDate to SQL_C_DATE
20021028	Revision 8: Denormalized study table (add patient ID, name, birthdate) to show consistency problems. Revision 9 is a further de-normalized version that is used with the built-in DbaseIII driver.
20030319	Revision 10: Fixed width of ReceivingCoil: to 16 chars
20030907	Note: revisions 9, 11 and 12 refer to the ODBC-less version
20040402	Revision 13: Added ImageID to image database
20040905	Revision 14: Added WorkList database with HL7 tags
20080124	Revision 15: Added patient position to denormalized image table for built-in DBF driver');
20080124	Revision 16: Moved Stationname and InstitutionalDepartmentName to series table');
20080818	Revision 17: EchoNumber, ReqProcDescription to 64 characters; StudyModality, EchoNumber, ImageType to DT_MSTR; use Institution instead of InstitutionalDepartmentName');

DGATE.CPP (main of DGATE.EXE):

19980327:	Disable logfile for speed and save disk
19980327:	-! Enables logging to Personal pacs console; RunServer works; -p breaks
19980327:	All printf's to systemdebug
19980320:	Integrated dumppacs code here for convenience
19980320:	Fixed -p option; SaveToDisk returns filename, printed in stats
19980320:	Documented and added print of usage (-? option)
19980404:	Added test options, ODBC config option and command line specified console pipe / udp logging
19980409:	added FileCompressMode option; use local save to disk code instead of pdu member
19980409:	Return on failed GenerateFileName in SaveToDisk
19980409:	Cleaned up NKI private compression code and allow >15 bit differences
19980410:	Added run length encoding for equal pixels to compressor; fix 3-byte case

19980415: Cleaned up the messages; -! does not enable debug; do not enable accessupdates (use aroute)
 19980618: Some work on manual and database layout listing
 19980621: Added -z option: zap patient
 19980624: Comm test also produces systemdebug and long text output; added \n to some output
 19980625: SaveDicomDataObject in loadddo.cpp supports NKI private compression
 19980704: Added some test code for counting disk space and renaming device for patient
 19980705: Added CACHE devices in free store check; added FindPhysicalDevice (searches for file)
 19980709: Added archiving options: -as, -au, -ab, -ac, -ad
 19981217: Temporary use PDU.Save... code for testing DICOM objects (does not compress)
 19981218: Temp use Pdu.Load as well (no compression)
 19981228: Disabled ljz's code until it has been completed
 19990108: Reenabled ljz's code: Replace uncompressed by compressed pixeldata VR in SaveToDisk (it seemed to work anyway...)
 19990109: Removed some commented out code; added comments for alternative load and save routines. Added Regen of specified device
 19990110: Added n, for MAGn, as parameter in -as and -ab archiving options (default all). Use exit(1) on failure
 19990111: Fixed exit code on -ar option
 19990117: If the filename is *.dcm (depends on FileNameSyntax) use uncompressed chapter 10 format
 19990317: Added NKI-private stuff
 19990628: Temporary enabled debug in -!nnnn option
 19990630: Added MIRROR option; disabled debug in -!nnnn option
 19990707: Fixed MIRROR option; added MIRROR message; added check on FindPhysicalDevice result
 19990817: Added ability to spawn ExportConverters at the end of SaveToDisk
 19990827: Optimized read of slices: cache a DB to pass to GetFileName (10% faster)
 19991117: Added parameter FileCompressMode to in nki_private_compress call
 20000318: Added private command object 9999, 0300 = print to console
 Added private command object 9999, 0400 = force C-Echo to be 'silent' (no log). Display C-Move Destination; print time info about threads
 20000323: Serious fix: MainThread 'hangs' when several associations occur simultaneously, caused by compiler-optimization. Solution: DriverApp.Lock should be 'volatile'.
 20000329: On request by ljz: designed crash of server if silent VR receives 'DEAD'
 20000501: Compare dicom modality with UPPER-case ini-file setting (ExportConverters)
 20000621: Initialize TroubleLogFile and UserLogFile; log C-STORE and C-MOVE
 20000703: Logging of trouble now starts with '****'
 20000703: Exclude archival messages from trouble log; removed force dcm code

20001104: Added WINAPI to thread routines

20001105: Replaced malloc by new where a mix could occur (vr->data)
Made compilable for MSC and BC and WC using some #ifdefs
Changed order of verification and store; better localized
new/delete DDO more

20001106: Run-time load odbinst routine for BC and WC, use spawn...
i/s/o _spawn...

20001106: Use delete [] operation for vr->Data

20010328: Added -av: verify mirror disk

20010415: Added DeleteSeries, DeleteStudy and DeleteImage routines and
-f options

20010416: Added -f options: -feFile = enter MAGN file into DB, -fdFile =
delete file from DB -faFile = add (also makes copy) file into
server, -f?=ask UID of file, -fu = generate UID,
-fcNEWPATID,file = modify patient ID of image file (moves
file!). NOTE: -fc is irreversible in the sense that if a portion of a
study/series is changed and changed back it winds up as a
different study/series. This occurs because the UIDs must be
changed for each image modification. This problem does not
occur when a whole series or study is changed

20010417: Simplified code to change uids to allow easy extension to
referenced SOPs. Check after load on common UIDs to avoid
accepting dropped non-dicom files. NOTE: ChangeUIDinDDO
should also change sequences and referenced SOPs
(0008,1155)

20010418: Fix in above change

20010418: Changed names of database fields in UIDMODS - did not work
on MsAccess

20010419: Added -fk option: copy image file to other patient ID (as -fc but
no delete). Added version number 1.3.9 to command line help

20010429: Added -fz option: zap file from server and database

20010430: Some work on read ahead thread: note that thread must not use
any pointer copy!

20010502: Made read ahead thread working; fixed thread counter; fix odbc
option: strings too short

20020506: Fixed handle leak in read ahead thread

20020507: Fixed memory leak in read ahead thread; kbhit() test #ifdef
DO_LEAK_DETECTION

20010509: Added RenameDevice is -ae option

20010730: Replaced parameter argv[0] of ExportConverters to proper
value.

20010829: Added ExportStationName as export filter; default Export...
value set to "*"; Made export comparisons case insensitive;
added special 'forward to AE' ExportConverter; Version to
1.3.10

20010831: Added extended syntax for exportconverters with %f=filename,
%m=modality, %s=stationname, %b=file base name, %p=file
path; Also allows 'copy %f destination', but for windows NT
only (uses cmd.exe). Added ExportFilter option that uses an
SQL statement to filter.

20010901: Added a simple queue to process exportconverters
asynchronously; Create a queue for each converter; queue
mirror copy requests

20010902: Added %o = SOP; %n = newline; added the following hard coded converters: append "text" to file; write "text" to file; copy file to file; copy file to dir

20010904: Re-entered change of 20010730

20010905: Fix: After 'SaveToDisk', do not free pDDO

20011109: Fix: made ExportConverter stuff case insensitive
Note: empty filter tests for empty contents - this is not the same as '*'

20011110: Note: in release 1.3.10, the " to " in e.g., write "%f%n" to file must be lowercase

20011115: Made " to" case insensitive; Added *** before some error messages for better logging. Removed cache db: would hang the server if SQL has gone down and up. Now use one DB per thread which is passed to the read ahead thread = fast too!

20011207: Added extra errormessage when creation of datasource fails

20020109: Disabled this code for watcom or borland (instead of run-time loading an address)

20020314: Implemented ignore of C-CANCEL request for ADAC NM (was error before). Blocked out SQLInstallerError call; does not compile on clinical MS4.2

20020413: Added some notes about thread safety and db's: The serverchild is not thread safe when reading and writing is done on the same association.

20020415: Added -atDEVICE option: sanity check of images on disk; version to 1.3.11

20020416: Made UID generator configurable through UIDPrefix in dicom.ini; made time print unsigned

20020422: Sleazy fix of ModifyPatID: time(null) created same UID: now wait 1 s per changed file

20020426: Generate filename no longer gives an error: is done at higher level

20020428: Only serious error messages use *** in message

20020428: Added FAILSAFE_STORAGE: tries to save incoming files even when DB errors occur

20020429: Put SaveToDisk in myUnknownStorage: pass DB failures to caller; Disabled FAILSAFE_STORAGE: any storage failures should now be detected by client

20020508: Do not rewrite image not on MAG but it is not an error (warning is generated earlier)

20020520: Added -gSERVER,DATE option: grab images on date from remote server

20020521: fix displaying junk filename when rewriting image not on MAG

20020522: Fix in SetStringVR: strings must be padded with a space

20020529: -i and -r now have mode parameter (0=normal and default, 1=simple, e.g., for DBASEIII)

20020531: In 'QualifyOn', support case-sensitive call to 'GetACRNema', but make AE-Title uppercase after retrieving IP and port.

20020613: Better handling of Compression ON/OFF in 'SaveToDisk'
Fixed crash in DeleteImageFile (twice free of VR->Data)

20020613: compress to specified mode, check (de)compress result, decompresses data before forward; added 'export compressed to AE' export; -fa has optional ,NEWPATID syntax;

20020802: Simplified rmdir code and allow multiple subdirectories in rmdir and mkdir code. Version to 1.3.12 (unreleased)

20020804: Use PATHSEPCHAR where possible

20020807: Changed SaveToDisk order: only make patientdir when DB save succeeded

20020812: Added skeleton of printer support

20020816: Added color imagebox; Save incoming print DDO as file

20020819: UIDs of imageboxes end on rows.cols.position for easy print handling

20020819: Fix in delete nonexistent mirror files for changepatientUID

20020819: Extract pixel data from sequence in ImageBox; added sleep(10) in -fk option

20020819: Added better UID generation (but gets close to 64 chars)

20020821: Fix printer UID generation; display page orientation and film #; shorter info

20021016: Added (indexed) patid to exportfilter query and getfilename for speed; The option tests for rev5 db or higher; increase 64 strings to 66; Note: GrabImagesFromServer option does not yet pass patid to the query (i.e., will be slow with non-odbc driver)

20021017: Control NeedPack from here; added -c option: set UID counter at startup. Removed Sleep from anonymize options; do not re-create UIDMODS

20021018: Pass NoKill through GenerateFilename for interactive DGATE

20021028: Fix query on study in PatientStudyOnlyQuery (did an image query)

20021105: Revamped read-ahead thread logic to try fix rare crash where ratd->DDOOutPtr became NULL

20021115: Use Generic style retrieve classes with NKIqrsop code instead of older qrsop code. Advantages: read ahead thread used, and allows print of error messages from C-MOVE. Required changes such that ratd->pcdo can be NULL.

20021121: Disabled read-ahead thread for time being

20021212: Also loop ServerApp.Server(Port) in threaded mode server

20021213: Re-entered fix by Hanne Kooy: compressed forwarding had extra ;

20021213: Re-enabled use of generic query classes with read-ahead thread; but made read-ahead thread configurable with "EnableReadAheadThread". Made grab query faster by including date - note grab is independent of FixPhilips

20021215: Found read-ahead thread problem reported by Aaron Cann - missing Sleep in wait for slices

20021223: Version to 1.4.1 pre-release; added in dgatefn forbidden filename chars suggest by Thomas Buerer

20030113: Removed loop ServerApp.Server(Port): gave endless loop error messages

20030120: Added cleanup-code when DO_LEAK_DETECTION is defined for easy leak checking

20030205: Fix bug found by Frank-Andre Siebert: read ahead thread crashed when ID or SOP is missing. Also fix 2 other potential missing VR problems

20030228: Fixed bug reported by Jeff Babcock: series retrieve from E-film crashed (patid not passed)

20030424: Changed calls to obsolete PDU.Write(&DDO). In 'SaveToDisk' the call to PDU.SaveDICOMDataObject now has parameter DICOM_CHAPTER_10_EXPLICIT (which writes IMPLICIT files if not compressed)

20030606: Fixed incremental counter when creating UUIDs or generating filenames Fix: QueryRetrieveLevel is mandatory for a QueryResponse

20030625: Fix NGET of BasicPrinterSOPClass

20030625: Fix in above fix

20030627: Adapted for MSC4.2; Finally really fix NGET of BasicPrinterSOPClass (wrong UID used)

20030629: Added -nd, -nc, -jd and -jc options: (de)compress NKI; (de)compress JPEG (used OFFIS executables)

20030630: Started on compression for dropped, incoming, transmitted; NKI vr 0x9999,0x700=custom compression. Added 'forward compressed as xx to ...'

20030701: QualifyOn now also has compression parameter

20030703: Moved compression code to nkiqrsop; dgate -j-xx and -j*xx = test compression options. Removed "forward compressed to" because "forward compressed as .. to" is better. Note that FileCompressMode is now obsolete. KNOWN BUG: JPEG compressed images may be rewritten as V2 (when written originally as NKI) which are then not correctly read.

20030704: Made that FileCompressMode parameter still works even though is obsolete. Moved faulty delete DDO after SOPUnknownStorage.Read

20030705: Added check of Transfer Syntax next to extension for save type; ProcessDDO takes **DDO

20030706: Use dgate.dic; Attach VRType to PDU's for implicit little endian support; small fix in dump code

20030709: Printer support fix; version to 1.4.2; Added SendUpperCaseAE parameter (default 0)

20030730: Force ImplicitLittleEndian, ExplicitLittleEndian and ExplicitBigEndian objects that must be stored in Chapter10 format, to ImplicitLittleEndian.

20030815: Version to 1.4.3

20030905: Allow longer filenames

20030910: Added check on rights to write (trouble) log file and printer_files in MAG0

20030921: Allow write to any MIRROR if MAG is full; added -ff# option: delete until # MB free

20030922: Added StorageFailedErrorCode (default 0x110 = 272 decimal = failed processing)
Added ExportCallingAE and ExportCalledAE export filters (also %u=SCU, %c=calledAE in converter)

20030401: Version to 1.4.4; pass status pointer to compress routines

20040402: Delete DDOptr when SaveToDisk cannot write but does not give an error; Added "Cannot rewrite jpeg/rle image in v2 format" check; Use system DSN for ODBC (easier login)

20040406: Always use EXPLICIT syntax for jpeg, fix v2 always use DUMP

20040426: Version to 1.4.4a

20040520: Version to 1.4.4b

20040523: Added patient ID to speed GetFileName for single slice move

20040528: Set NeedPack (controls DbaseIII indexing) for regen, archive, maintenance; Version to 1.4.4c

20040528: Added built-in maintenance commands through silent VR in ECHO; pass patid in DeleteImageFile

20040530: extract (DBF only) to database like XDicomImages in same directory; Version to 1.4.5

20040601: Added deletestudy, deleteseries and packdbf maintenance commands

20040602: Fix: 'forward compressed as' always failed

20040605: packdbf maintenance command sets NeedPack to 3: forces pack

20040606: fix rewrite to not MAG (JUKEBOX) was NOT ignored

20040607: Merged fix ljz 20040602; do not allow rename and delete with empty name

20040610: Added maintenance server tasks for archival and regeneration

20040612: Added indexdbf maintenance server task

20040614: dgate command line maintenance options use NeedPack=5: non-threaded index generation. Added -am option: move(all) device data; for selected data use -amMAG0.Archiving,MAG1

20040624: Version to 1.4.5a

20040626: Pass study and series UID to getfilename; version to 1.4.5b; Fixed leak of pats array

20040713: Version to 1.4.6

20040805: Fix in C-MOVE for single file (from jlz); version to 1.4.6b

20040909: Increased length of ExportFilter (was 64, now 510)

20041020: Added mergeseries(file) and mergestudy(file) maintenance commands, silenttext to 1024

20041101: Added deletesopfromdb maintenance command; version to 1.4.7

20041103: called, e.g., AE~nj overrides IncomingCompression to nj

20041108: Added \$c test compression mode (alt-drop files with \$c as patient ID tests compression modes)

20041109: Added \$tSERVER test compressed forward; some fixes in forward compressed to (server name read wrong)
\$s tests all filename syntaxes

20041112: Added \$rN N times repeated entry test, and \$tN which enters image N time each on its own thread

20041128: Added -^file: log to file (timestamped) instead of to server console; -#file: same with debug

20041129: Added silent commands debuglog_on and log_on to set logging to UDP or file; Added silent command read_amap to re-read acrnema.map without restarting the server

20050102: Small modifications to allow compile through total.cxx

20050107: Added code to hold and retry queued forwards and mirror copies when failed: After failed export/copy, start storing entries in file ExportFailuresN (CopyFailures). If a new export/copy is requested >1 minute after last failed one, try again; if queue is empty, >100 seconds after last fail, try export/copy stored ones. Processing queued entries has priority over processing stored entries.

20050107: Version to 1.4.8; modifications for linux compile

20050108: Added server PORT# to fail files, so that multiple servers on different ports do not share files

20050109: Added configurable TCPIPTimeOut, FailHoldOff, RetryDelay and QueueSize

20050109: Adapted thread code for linux; still need to implement WaitForSingleObject

20050111: Added ctype.h (for mingw)

- 20050118: Linux: crudely implemented WaitForSingleObject; detach pthreads to avoid leaks; crudely implemented export converter apps
- 20050119: Added grabimagesfromserver, regendir and makespace server commands; dgate -- sends server commands to other running server; adapted manual
- 20050121: Small updates to manual
- 20050124: Replaced BackGroundExec with simpler system() for Linux; added query server command, e.g., dgate "-- query:DICOMpatients|patientid,patientname,patientsex|patientsex='F'|%s %s %s|t.t"; Added read_ini: server command to reread dicom.ini; changes most config except: TCPPort (busy), UseBuiltInDecompressor (cached) and FileNameSyntax (cached)
- 20050130: added get_param, get_freestore, get_amap, get_sqldef that respond through echo-response; also: get_sop, get_transfer, get_application, get_localae, get_remoteae, get_dic, forward. added: dump_header (to file), convert_to_gif (to file); adapted query (to file) where file="cgi" for stdout in cgi/mime mode, file="" to send to stdout as is; most GUI commands can now pass PatientID|SOPInstanceUID instead of filename of dicom object; added put_amap, write_amap, put_sop, put_transfer, put_application, put_localae, put_remoteae, put_param. Note: get/put_param work through disk (read_ini activates), get/put_amap through memory (write_amap writes to disk), get/put_sop etc read from memory and write to mem and disk.
- 20050204: Added delete_param, delete_amap and fix cgi out (double \n)
- 20050204: most GUI commands can now pass PatientID|? *:SOPInstanceUID; added delete sop etc commands
More fixes in cgi data passing VR vr() is mostly wrong; use VR *vr=new VR()
- 20050205: Allow up to 16 output fields in query.; repeat queried fields over all 16 outputs. Added counters for server status; start on display_status server command
- 20050206: More work at display_status, renamed testtable to xtesttable: default not opened. Access ObjectFile and DeviceName through query instead of with GetFileName up to 3 x faster!! Also fixes potential leak in read ahead thread: pats, series and studies not always freed. Allow up to 32 cgi query outputs, tidied dump layout, start on web interface (put dgate and ini in cgi-bin)
- 20050207: Added query2 option: limits number of output rows as asked; gui command not counted as association. Some more work for web interface
- 20050210: Adapted web interface queries such that they run on sql server
- 20050212: Removed need for thread local storage; automatically adapt dgate extension in cgi scripts depending in OS
- 20050213: Added more parameters to server configuration web page
- 20050214: Added update button to server config (updateconfig page to be implemented)
- 20050215: Fix in LoadForGUI: Drag 'n' drop of files in the server did not work anymore. Fixed dgate crash in case of failure in 'recompress()' (pDDO=NULL when OFFIS failes). Merged

20050327: Added rudimentary gz unzip support for dropped files (uses 7za.exe)

20050401: Start on Modality WorkList Support

20050404: List End/Start sequence db types (only supported for worklist)

20050411: Fix incorrect space after filename and device in C-MOVE internal query

20050414: Added addrecord server command: for now only option to enter worklist entries

20050603: Fix servercommand query(2) for empty where part

20050825: Fix web interface queries for sql server (were OK for dbase)

20050826: More work on web interface and added worklist there; deleterecord command

20050827: Fixes in addrecord and query commands, editworklist,saveworklist,deleteworklist

20050829: More fixes in web interface, disable caching for most/all? of it

20050831: Allow command line length of up to 4096, allow edit all worklist database fields

20050901: Removed invalid cache-control: cache; added WorkListMode:0= off, 1=optional, 2=compulsory

20050902: Read db config for worklist edit page in web interface; use generic webpages for db edit. Added HL7Tag in DumpDD

20050903: Added 'simple' loadhl7: server task implementation and HL7->DICOM translation. Also addimagefile will enter HL7 files into the server, i.e., drag and drop works

Fix some web queries for sql server with denormalized tables

20050904: Some more tweaks to hl7 code; max query formats to 48; max format to 8192

20050905: Auto split HL7 subfields and repeats without using a list

20050907: Merged change by Hans-Peter Hellemann: fixes extension problem for linux web interface for some browsers

20050908: Protect convert_to_gif when file not found

20051121: Recompute SQLResultLength because it does not work in PostGres BindField

20051123: put in #ifdef POSTGRES

20051208: put in #if 0 (fixed in odbci.cpp); start on virtual server: will grab images from other server as needed in read-ahead thread: when in db but not on disk; for now use R.A.T also for single image access to avoid code duplication

20051209: Start on virtual query: image (!) query is passed to other server, results populate db

20051210: Added "vX" incomingcompression (stores dbf but not image; forwarding disabled). Added virtual queries on all queries and moves: now works as transparant file cache. Added VirtualServerFor0..9 configuration entries: server merges own data with these AE's. Added CacheVirtualData control; if set data not deleted after use (db kept for now)

20051211: Fix: override query level in virtual query

20051212: Store virtual server # in device; saved on query, used when retrieving, stripped when storing

20051214: Use different sleep lenghts under linux - not all sleep(0); small fix in virtual server

20051215: Implemented a linux version of Sleep using nanosleep

20051217: Use new VR::ReAlloc method to replace data in VR; added quit: command

20051218: Use ReplaceVR instead of Push in anonymize etc; also changes in sequences. Version to 1.4.10; show version and exe date in startup log. Do not start mirror copy queue unless it is needed; show some thread startup info. Added -b = NOTHREAD: run server as single process, useful for debugging and profiling. Fixed doc of --regendir: command

20051219: Fixed several small leaks - as tested with Linux malloc_stats

20051222: Started on bypass server for virtual query - not functional yet (cleanup required)
Removed debugVR logic: errors should now list DCO

20051228: Fix virtual device logic for VirtualServerFor0;
Cleanup vr's and duplicates when bypassing server for virtual query, seems functional

20051229: NonDestructiveDumpDICOMObject now also list sequences; DebugLevel>0 lists DCO, >1 lists query DDO; fixed leaks in virtual query and some others

20051230: Added --debuglevel: command, small fix in NonDestructiveDumpDICOMObject

20060103: Small fix in virtual query; disabled virtual query for worklist; debuglevel 3 shows some query results

20060123: Added patient/study/seriesfinder (server parameter ignored for now, later for virtual servers etc)

20060125: Added serieslister (server parameter ignored for now, later for virtual servers etc)

20060128: Renamed to imagelister, added serieslister

20060131: Sort filenames from imagelister on ImageNumber (0020,0013)

20060226: Version to 1.4.11

20060228: Small fix in isxdigit thanks to ljz

20060311: Use ReplaceVR in virtual query - level was sent twice (thanks David Lang)
Added WEBReadOnly flag

20060317: Support calledae and callingae in filenamesyntax - for virtualquery called is always MYACRNEMA

20060324: Strip group 2 from outgoing c-stores if not compression "as" or "is"

20060328: Improved forwarder, always propose Verification and use IsAbstractSyntaxAccepted(iUID) to test if host accepts image

20060402: Fix hang condition when a file failed to read in the read-ahead thread

20060402: Do not remove transfer syntax prior to dcmdjpeg/dcmjpeg (here in test recompress only)

20060405: Tried fix for multiple moves on same assoc (thanks Andrej Savelov)

20060405: Added message when IOD fails to read

20060603: Fix crash when started as dgate -v: empty statbuf

20060607: Finally fixed multiple moves on same assoc (thanks Andrej Savelov and Fred Röhner)

20060618: Renamed dfilecopy and define ODBC_ADD_SYS_DSN if needed

20060619: Set version to 1.4.12alpha1

20060621: Added clonedb: command; transfer FULL db from other server to this one using virtualquery

20060628: AddToDatabase has JustAdd parameter: skip one unnecessary query

20060701: Speeded filter string handling

20060702: Pass DB to GenerateFilename to avoid zillions of db open and closes

20060704: Print number of records found in all queries

20060708: Set version to 1.4.12alpha

20061126: Moved servertask temporary files to MAG0/printer_files; protected query: and query2 against empty input

20061126: Experimental forwarding on same association: close unused association after 5 seconds. Clean collected sopclasses after one hour of inactivity.
Note 1: there is a problem with AReleaseRQ: do not use in this task; Note 2: maximal 20 forwarders have a remaining association

20061127: Added ForwardAssociationLevel (PATIENT, SERIES, STUDY, IMAGE, SOPCLASS; default STUDY), ForwardAssociationRefreshDelay (default 3600 s), and ForwardAssociationCloseDelay (default 5 s).
Limit maximum number of ExportConverters to MAXExportConverters=20 (for static arrays used in forwarders). Forwarders keep association open as long as UID at ForwardAssociationLevel does not change. Show filename in forward log; document where fixkodak should operate for virtual queries

20061128: Added ForwardAssociationLevel GLOBAL: use association until timeout or incompatible
Close ForwardAssociation after send when ForwardAssociationLevel is IMAGE

20061129: VirtualServerForN may be "server, FIXKODAK" to clean queries (add 0) and their response (remove 0)
Hardcoded filters for exportconverters (modality, called, calling, station) can now match e.g., RT*

20061130: Pass series and study in virtualserver outgoing c-move

20061213: Fixed nasty bug in query2 and TestFilter: Query without bindfield overwrites previous bound strings!
Maybe was problem causing crash on linux with query2: and problem with option -g reportedly stopping incorrectly

20061219: Version to 1.4.12

20061231: Added ForwardAssociationRelease to allow testing reconnect problem (default 0 = just hangup)

20070103: FixKodak blocks *xxxx wildcard virtual queries on date (not allowed in Kodak)

20070103: Version to 1.4.12b; start on ImportConverters e.g., <set 0008,1040 to "%V0008,0080">

20070104: ImportConverters %QP %QS %QE %QW; <destroy> <delete xxxx,yyyy> <save to filename>
Added %% and %i to Export and Import converters

20070105: <set 0008,1040 if "...">, <setifempty 0008,1040 to "...">, <setifempty 0008,1040 if "...">
Import/export converters only report when syntax OK and report when action is void
Added %Vxxxx,yyyy to export converters and %d(date/time) to import and export converters
An import/export converter can thus be used to extensively log incoming images

- Added import converters: <stop>, <if "string">, <ifempty "string">, <nop> (also export)
Added general purpose variables x, y, z (read as %x, %y, %z) to import converters
Definition of number of ImportConverters now unnecessary.
Note: importconverter and exportconverter strings may be changed run-time in dicom.ini
- 20070105: Some fixes in dgate --archiving options manual; fixed dgate --restoremagflags:
- 20070107: Fix web access broken by change in odbci for dbaseIII; query must now always include spaces around =
- 20070113: Do not store NKI compressed data in v2 format: decompress and give error message
Recompress to NKI format now enables StripGroup2; Tested with \$c server debug command
- 20070117: Allow multiple rules per Export or ImportConverter separated by ; and zero or more spaces
<nop> statement prints itself on console: useful for logging anything that follows
- 20070117: Now maintain multiple connections per ExportConverter line (max 20x20 forwards)
Added <stop>, <silentstop>, <if> and <ifempty> to ExportConverters; but they only affect single rule (while these would cross over to the subsequent rules in ImportConverters)
- 20070122: Now also show free space on mirror devices
- 20070123: Fixed TestFilter for patientid with '
- 20070125: Removed <if> and added <ifnotempty>, <ifequal>, <ifnotequal>, <ifmatch>, <ifnotmatch>, <ifnumequal>, <ifnumnotequal>, <ifnumgreater>, <ifnumless> as im/exportconverters
Added %xxx[first,last] substring operator; and protect it against overrange
<ifxxx> no longer crosses over importconverters; added {} block for im/exportconverters
E.g.: ifequal "%V0008,0020[0,3]", "2004"; { forward to CONQUESTSRV3; forward to CONQUESTSRV4; } nop test
Note: 1) all (also ifxxxx) statements end with ; 2) statements may begin with { or };
3) extra spaces only allowed after ;{} and single space after ,
- 20070127: Fix {} in exportconverters. Release 1.4.12b to fix more or less critical odbci error
- 20070131: Fix in TestFilter (quotes double); Version to 1.4.12c
- 20070201: Added <storage MAGn> import converter: sets device to store this (new) image to (default MAG0). Fixed importconverters stop and destroy: did not break current line
- 20070204: Added extra format starting with # to ImageFileLister: e.g. # %s:%s prints MAG0:filename
- 20070206: Added create database: dgate -esapw name user password
- 20070207: Fix query string bug on applyworklist found by capolan1
- 20070207: dgate -e name root <blank> works for mysql: root is used as admin and login
Also implemented dgate -esapw name sa sapw where sapw may be empty

- 20070210: Added CheckDuplicates to UpdateOrAddToTable; added @ in imagefilelist: replace \ by /
Start on ocx based viewer integrated in cgi interface; used dicom over http:
Small change in gif export; added * as separator in query/2
- 20070211: Added WebScriptAddress (default http://server/cgi-bin/dgate), WebMAG0Address (defaults http://server/mag0) and WebServerFor (default 127.0.0.1). Fix in cgi query parser: now loads dsize and size correctly
Version to 1.4.12d
- 20070212: Added WebCodeBase, show version in status_display, added config of which dicom server the web server connects to, added [webdefaults] section, added iconsize parameter
- 20070213: Added : as format in imagefilelist: use to list pat:sopuid
Fixed sorting problem by using adapted imagefilelist for seriesviewer
Fixed default for WebCodebase; Version to 1.4.12e;
Use correct case of sql table names in cgi interface: required by mysql on linux
Unsolved: n3 does not always decompress correctly in dview.pas see p20401977:i47, changed default to n4
- 20070215: Fixed logging and added *** Association rejected message when association is rejected
- 20070304: Added <defer> exportconverter: retry later as if connection unavailable
- 20070305: Note: n3 problem fixed in dview.pas, version to 1.4.13alpha
Allow merging of up to 1000 UIDs: set SilentText to 64000 and *uids[1000] (thanks mp)
Allow named pipes for log_on and debuglog_on
- 20070307: Work on extending extract: allow e.g. SQLite -> dbaseIII
- 20070308: Made changes for big endian and Apple's DARWIN
- 20070314: Finished extract: allow e.g. SQLite -> dbaseIII in directory MAGO dbase
- 20070315: Merged DARWIN stuff; Set TCPIPTimeOut for PDU that moves images
- 20070316: Display defer import converter; Clumsily added <compression> import converter and improved <storage> import converter; Made <storage> thread safe. Failed recompress() no longer reason to fail drag and drop; add convert_to_bmp. Use | instead of , to separate patientid from file in addimagefile:
Distributed , code to each of the server commands to make it more , safe. Note: it is NOT a good idea to use a , in a flexible filenamesyntax. Added level & window to convert_to_gif and convert_to_bmp; longer log lines for dump VR
- 20070317: <prefetch> using new prefetch_queue, mayprocess and into_queue_unique, <preforward>, <preget>
- 20070324: preget now <preretrieve> and avoids looping by checking orgmessageid!=0xfbad I.e., data collected by <preretrieve> does not go through import/exportconverters. Added delayed exportconverters <forward patient>, <forward study>, <forward series> with optional compression
Removed "forward compressed to" (without as section) as is never used. Version to 1.4.13beta; delay to 10 minutes

20070326: Fixed leak in <forward patient> etc. preretrieve also checking orgmoveae

20070330: Change by Mark Pearson: read dicom.ini from same dir as exe in unix mode; merged more Darwin stuff. Also taken over bcb's fix for the finder: studymodality was asked at patient level. Also put in a more ENxxxx as command and messageid

20070404: Added CommaInFilenameWorkAround()

20070407: Use reentrant ctime_r when available; moved static SOPPatientRootQuery etc to stack to fix reentrancy problem: concurrent outgoing c-move of CT and MRI on different associations would share PDU causing sopclass not accepted on one. Also seems to reduce linux crash problems on multiple inbound (4 or more) C-stores. Also moved CreateBasicFilmSessionRequest etc to fix potential reentrance problem for printing.

20070410: Merged CommaInFilenameWorkAround()

20070413: "extract:" will just create dbase III clone of full patient DB; added "prefetch" server command

20070415: Added Prefetcher option to dicom.ini: is set will aggressively pre-read queried patient or studies

20070416: Prefetcher prints more information; use windows calls in windows for faster pre-read

20070705: extract: uses DT_DBASEIII_NOINDEX to disable indexing and packing when extracting from e.g. sqlite to dbaseiii
Note: extract: non dbaseIII goes to MAG0\dbase, dbaseIII goes to DataSource

20070706: Added <between "9", "17"> (9:00 to 16:59) or <between "17", "9"> (17:00 to 08:59) import/exportconverter

20070709: Fixed print of clone db done

20070720: Fixed problem where waiting "forward ... to " task caused high CPU load

20070901: Added dgate --checklargestmalloc:

20071027: Added ForwardCollectDelay for delayed forwarding (default 600 sec)

20071030: Fill in called and calling AE for SaveToDisk for e.g., dropped files etc. Fix %u and %c in Export and ImportConverters: called and calling AE. Log flow control Export and ImportConverters only in debug mode. Added retrying to delayed forwards or prefetches

20071102: SetDicomErrorHandler: orderly shutdown of server on serious DICOM library errors. Fixed a bug in ForwardAssociationLevel: last UID was lost when reading INI file to szTemp. Added MaximumExportRetries and MaximumDelayedFetchForwardRetries (default=0 is no maximum)

20071103: Version to 1.4.13; defer resets queue fails -> keeps on retrying

20071104: Use local as AE for imagelister in web page instead of CONQUESTSRV1; Enable remote servers (use AE instead of 'local') in finder and lister commands; Fix crash when dgate --option gets empty filename: now lists to stdout

20071114: Put --checklargestmalloc: in dgate -?

20071118: Adapted for 64 bits (use SQLLEN for BindField)

- 20071120: Fixed KodakFixer for empty patient ID (crashed on 64 bits only). Added [modality xx] and [date yyyymmdd-yyyymmdd] clauses to export [patient|study|series] to AE
- 20071121: Added %V*GGGG,EEEE: search VR in any sequence; version to 1.4.13a; Added [now -ddd+ddd] and [age -ddd+ddd] clauses to export [patient|study|series] to AE. No retry on remote dicom error in forward xxx to: would retry zero records response
Use study date for data, age and now clauses
- 20071123: Made forward command case insensitive; added forward image; added get [patient|study|series|image] from AE. The latter option makes preretrieve obsolete
- 20071124: Added [sop xxx] clause to [forward|get]. Reorganized exportconverter code: all converters now do % substitution; reduced code duplication. Added reading of sequence items like in %V/300c,0060.0/0008,1155 or %V/300c,0060/0008,1155
Real life example (must all be one line, max 512 characters):
ifnotequal "%m", "RTPLAN"; stop;
nop "tests modality = RTPLAN";
ifnotequal "%V*300a,00b2[0,1]", "A2"; stop;
nop "test machine name = A2";
forward to XVI_A2;
nop "forwards the current recieved RTPLAN";
get study modality CT from NKIPACS;
nop "collects associated CT";
get study modality RTSTRUCT sop V/300c,0060.0/0008,1155 from NKIPACS;
nop "collects associated RTSTRUCT";
forward study modality CT imagetype *AXIAL* to XVI_A2;
nop "forwards associated CT";
forward study modality RTSTRUCT sop %V/300c,0060.0/0008,1155 to XVI_A2;
nop "forwards associated RTSTUCT";
- 20071125: Fixed syntax for %V/gggg,eeee.i/*gggg,eeee (search all groups under a specified one)
- 20071126: Start on StorageCommitment (unfinished)
- 20080103: Switched to ms7 compiler; fixed forward and get date, modality, sop filters: would not supress the UIDs
- 20080107: Fixed malformed web viewer header: worked for IE but not for other browsers. Check presence of dicom.sql in web pages that need it. Deal with spaces in patient IDs in web pages
- 20080126: Check presence of dgate.dic when starting server
- 20080129: Fix KodakFixer: wrote one byte outside VR data (pr)
- 20080129: Added [imagetype xxxx] clause to [forward|get] (pr)
- 20080130: Small fix in above
- 20080205: Added experimental PadAEWithZeros flag; version to 1.4.14alpha
- 20080210: Added experimental IgnoreOutOfMemoryErrors;
Added "delete [patient|study|series|image] [date yyyymmdd-yyyymmdd] [now -ddd+ddd] [age -ddd+ddd] [modality mm] [sop xxxx] [imagetype xxxx]" export converter
- 20080221: Added [seriesdesc xxxxx] filter and "process [patient|study|series|image] [by command]"

20080222: Small fix in imagetype filter code; process by clause not part of unique check

20080302: Fixed case of DICOM table names

20080316: Added dgate --deletestudies:date(range) and dgate --movepatient/study/studies/series

20080318: Added [series xxxx] and [study xxxxx] filters; work on web page --deleteseries/study and --moveseries/study allow patid:uid as parameter
Web interface can now push and delete on series and study level, but delete is disabled in readonly web interface
Version to 1.4.14beta

20080319: Delete in browsers fixed?; added %v(desc)desc: search VR in given sop of same patient. Example:
%V(/300c,0060/0008,1155)/3006,0010/3006,0012/3006,0014/0020,000e Gives referenced CT series UID in referenced structure set from forwarded RTPLAN

20080321: Fixes in browser; added graphic web setting; may be gif or bmp for now; Fix %V for non-existing items

20080322: Added dgate --anonymize that uses general scripting

20080323: Small fix in (local) for DcmMove: overwrote destination

20080404: Fixes in series and study filter (rethink naming)
added value* match for in match() routine

20080618: Added extra logging in IARQ()

20080620: Only show extra logging in IARQ when assoc rejected

20080816: IARQ shows which sops are accepted; and differentiates no sop from connection terminated

20080817: Small layout fixes; <process with> import converter; show PDU length in log. Check saving of file succeeded (thanks Jeff Bellamy and Alberto Smulders)

20080819: Added experimental DT_MSTR: multi-value entry, e.g. query on 'PET' matches 'PET\CT'

20080819: added --get_ini_param (same as get_param) and get_ini_num to list parameter by index

20080819: Added get_ini to list all parameters in DICOM.INI

20080819: Added code by map but kept original get_param for old web cgi client compatibility; Version to 1.4.14

20080820: Solaris 10 fixes, and general UNIX fixes (map): configfiles may not be in current dir

20080821: Default of ForwardAssociationRelease to 1; added sleep export converter

20080823: Added study UID search to PatientStudyFinder; fixed DcmMove: would call local when remote addressed; Added rudimentary query/move web pages

20080825: query/move looks better; added "thumbs" column to local series lister; config patched to readonly; added background color and display of icon (to be placed in WebCodeBase)

20080826: One more fix in DcmMove, still called wrong server sometimes
Do not use 0xfbad as message ID for DcmMove from web page; webreadonly defaults to true; CGI viewer configurable: seriesviewer, seriesviewer2, noviewer, aiviewer

20080827: Added slice number and repeat output data in ImagefileLister and SeriesUIDLister for more flexible formatting; Added flexviewer: reads code from dicom.ini in web server directory

- 20080831: Translate yyyy and yyyyymm to date range in finder; further only accept valid date and date ranges; Added dgate --get_amaps: lists all non wildcard entries from acrnema.map; Fixed handling of patient ID with spaces in ImageFileLister mode ;; All undefined cgi modes can come from the cgi dicom.ini; added special and cgi script variables; Lines can start with -- to start dgate, with # as comment, any other passed as HTML; Added source parameter: flex pages can be read from script file; Added serversideviewer: requires javascript only, nothing to install
- 20080901: Fixed some leaks in printing and --commands; fixed --get_ini: formatting
- 20080902: Added header option for web scripts; added some CGI environment variables
- 20080905: Made new changes for big endian and undid some old ones.
- 20080908: Index in filelist format is now a string (%s), and added next string that gives index+1 (is1)
Fixed manual of --modifypatid: and --anonymize;; dicom mime to application/dicom
Added modes imagelisthttp, imagelisturls and imagelistfiles; added ? format for imagelister: gives #files
- 20080909: Added studyuid option to moveseries; pass it studyuid in seriesmover call from seriesfinder
Added --modifyimage:file,script (uses import converter scripting language, where you can use ' instead of ")
- 20080910: In modifyimage only replace ' with " conditionally; version to 1.4.14a;
- 20080910: Forbid patid, studuid and seruid change in modifyimage; medged bcb Mac stuff
- 20080913: Modified and use SetStringVR to correctly send UIDs with padded 0
- 20081016: Show bitness on startup
- 20081104: Attach dictionary to DcmMove PDU
- 20081105: Fixed display of bitness in web page
- 20081116: Adapted for very large objects (use unsigned int for length); remove extra 0 byte in HTML generator; fixed CheckLargestMalloc
- 20081121: Added create database for PostGres
- 20081123: Skip export converters for dropped, modpatid, modimage, merging
- 20081124: show database type on startup; Temp version to 1.4.14a1; fix many scripting crashes
- 20081129: start on better patient ID change: process all UIDs; fix drag/drop error introduced in 1.4.14a1
- 20081129: Removed double access of SOPInstanceUID in SaveToDisk; fix: do not free ddo after SaveToDisk;
added <newuids> and <newuids except > import converters; added --genuid, --changeuid and --checksum commands
Version back to 1.4.14a; modifyimage can change patient ID and study/seriesUID if newuids used as well
set can now also work in exactly one deep sequence: set gggg,eeee/gggg,eeee to "string"
- 20081130: Added --attachrtplantortstruct:planfile,structfile command using script; also --attachanytopatient/study/series

stop now stops one importconverter- not the chain; added <call file> and <return> statements to return from file
 Added optional anonymize_script.cq: overrules fixed anonymizer; added %A = %V but then gives CRC32 of data and %E = %V but gives new UID for given UID

20081201: added --convert_to_jpg and web page use (experimental)

20081203: If jpg graphic: list options as jpg, yes, no; fix slice number display is serversideviewer
 Added frame control to convert_to_jpg etc as l/w/frame; add optional frame control to convert_to_dicom; added frame control in serversideviewer

20090203: Removed DNS reversing (you can use wildcard AE instead); work on zip loading

20090204: Finished zip loading (to be tested on linux), uses now waiting BackgroundExec

20090206: Added QueryConverter0, WorkListQueryConverter0, RetrieveConverter0: can read called (%c), calling (%u), and c-move destination for retrieve (in %s), as well as all data in data object

20090209: Made compiling under linux

20090211: Export MyDcmError; only DCM_ERROR_MEMORY is fatal

20090212: Recoded virtual server (kept old code), to provide better grouping of move requests

20090215: Added VirtualPerSeries0..9 flags: determines if virtual gets are per image (default) or per entire series

20090216: Added %Rgggg,eeee: gives restored UID for newly generated one. Prepare for RetrieveResultConverter0

20090411: Added simple wildcard matching in match(), accept *text*

20090411: Version to 1.4.15alpha

20090513: Added CompressionConverter0 .. 9: called from recompress to 's0' to 's9'; Added <fixkodak> and <unfixkodak> import converters; added <after NN> clause to <forward study> etc; Blocked out StorageCommitment

20090514: %QXgggg,eeee reads aliasfileqX.txt with lines like old<tab>new<return> to map VR to new value

20090616: Add ^/~ after % command to convert to upper/lowercase, %^ %~ %[]; fixed leak when save to sql fails

20090616 Include file stuff; fixes to Unix 7z stuff

20090618 VirtualServerPerSeries is now a limit: if set to N, more than N images go per series

20090620 Version to 1.4.15beta

20090802 Support DCM_ERROR_DEBUG (debug messages from library) that print only if debuglevel>0

20090821 Removed need to specify "delete series modality *" instead of "delete series"; Fixed "process by " command; now uses all command string upto ;

20090824 Version to 1.4.15beta1

20090921 Added extract_frames: command

20090922 Tried to fix <compression> import converter; added <reject> import converter

20090924 Restructured manual; fix crash of studyfinder without search string; Note: on vista, commands like dgate --imagelister wont give large lists (does when piping)

20090926 %t = tab in scripts; fixed failed storage error message; added RetryForwardFailed flag (default 0)

20090929 Added ImportExportDragAndDrop flag (default 0)

20090929 Version to 1.4.15

20090930 Version to 1.4.15a

20091005 Version to 1.4.15b

20091012 Optimized preprocessing of read-ahead thread by saving tt

20091108 Started on <merge study> import converter

20091120 Started on <forward> import(!) converter

20091229 Version to 1.4.15c

20091230 Merged bcb changes: DGATE_VERSION, PATH_MAX, const char, bigendian fixes

20091231 Defined PATH_MAX for Apple's Snow Leopard, endian fixes, & fixed gcc4.2 warnings (char to const char, VERSION, DEBUG and char* to program memory)
Added jpeg2k stuff (HAVE_LIBJASPER)

20100111 Merged; but keep cgi formats local (less changes)

20100113 Fixed one signed/unsigned mix in dump routines

20100119 cast %d print to (int), %ud to (unsigned int); one more const issue

20100120 Fixed two const issues detected with ms8

20100122 fixed: %ud should be %u

20100122 DcmMove uses study-root query when UID passed to fix move to osirix from web interface

20100123 Fixed rare crash in function VirtualServer2; started on DT_FL and DT_FD; version to 1.4.16alpha

20100123 Added counters for jpeg2000; put info about jpeg libraries into serverstatus display

20100124 Blocked old VirtualServer(); rephrased cancel message; changeuidback and count_frames commands
Optimized DcmError call to gpps

20100124 Use much faster MyGetPrivateProfileString: profiled as followed: Installed C:\Program Files\Microsoft SDKs\Windows\v6.1\Bin\xperf_x64.msi set _NT_SYMBOL_PATH=e:\quirt\comps\exe\dgate;srv*c:\symbols*http://msdl.microsoft.com/download/symbols; set _NT_SYMCACHE_PATH=C:\symbols; xperf -on latency -stackwalk profile; run dgate; xperf -d out4.etl; xperfview out4.etl; select time range; select trace-load symbols; right click selection; summary table (takes very long the first time - downloads all symbols!)

20100207 Added uncompress command; version to 1.4.16alpha2

20100209 Fixed another rare crash in function VirtualServer2 (thanks PR)

20100227 Fixed Scotts modifyimage problem: <http://forum.image-systems.biz/viewtopic.php?f=33&t=2143>

20100227 Added -q option to set target address of dgate -- command; version to 1.4.16alpha3; pass DCO also for non-nki clients: used in %s for RetrieveResultConverter; Added more extensive printer activity logging

20100228 Changed HAVE_LIBJASPER to HAVE_J2K so Jasper or OpenJPEG can be used

20100309 Added double parentheses (gcc4.2 Warnings)

20100309 Changed int to unsigned int, commented out unused variables and routine (gcc4.2 Warnings)

20100619 Added #ifndefs and fixed shadowed variables(gcc4.0 Warnings)

20100723 Merged: Reread all changed; agreed with all but two in checklargestmalloc and cgi; 1.4.16alpha4

20100728 Added delete vr's after ->ReplaceVR's

20100815 Merged; version to 1.4.16alpha5

20100815 Fixed bug 32 in t=2127: seriesdesc filter

20100816 Added QueryResultConverter and ModalityWorkListQueryResultConverter

20100816 Fixed bug in VirtualQuery introduced by only partly renaming shadowed variable

20100816 Allow dicom.ini and other files to be moved; fixed / as cmd line option

20100816 removed (a) from #ifndef EnterCriticalSection(a) etc

20100819 Added system export converter; added "get study" etc import converter. Allow "set" in sequence items; Allow "set" to add sequence items.

20100823 Extended virtualservermask; pass server directly to VirtualServer to override server list

20100823 Merged and simplified bcbs basedir code; use exe dir as default basedir; Fixed overlap in ImportConverter number; Made %xyz variables local; Version to 1.4.16beta

20100824 Use dFileSize to test if file is written correctly: caught 0 byte files with full disk

20100827 Fixed two typos in warning fix extract: command

20100901 Fixed empty virtual query sent if no images there

20100905 Removed file functions: now use RecompressFile; document -j* and -j-; WIP: added testmode: function to allow internal file comparison

20100914 Fixed leak in PrinterSupport

20101017 Added MergeSeriesConverter and MergeStudyConverter

20100918 Merged; version to 1.4.16beta2; testmode does not affect LoadForGUI direct filenames; Fixed that changing patid etc in importconverter breaks exportconverter data

20100919 Added <format> clause to <set> importconverter command

20100920 Fixed [,fromend] operator

20100924 Started on error handling in virtualserver

20100925 Fixed problem in <forward> IMPORT converter; only worked as first command in line; SearchDicomObject now checks on buffer overflow; ; # lines are comment in script file; note: <system> converter should run hidden; <call> now passed called and calling

20100925 Version to 1.4.16beta3

20100926 Added <mkdir> and <rm> IE converters

20100927 Added <save frame> and <save bmp/gif/jpg> IE converters, <save to> E converter

20100928 Fixed in delayed <merge>, <delete>; fixed MergeSeriesConverter and MergeStudyConverter crash

Fixed 1.4.16beta2 introduced bug: patientid was passed incorrectly into importconverter

20101003 Added seriesdesc filter and script to DcmMerge, script to DcmMove

20101003 --browsepatient command; fixed that split/merge series generated new study

20101003 Version to 1.4.16beta4; merged jd scheduletransfer
 20101003 ImageFileLister now internally also takes study and sop
 20101003 Added dgate --submit:patid,studyuid,seriesuid,sopuid,
 username@machine:folder,password; Requires 7za.exe and
 pscp.exe
 20101003 Modified dgate --Scheduletransfer:patid,studyuid,seriesuid,
 sopuid,username@machine:folder,password; Added IE
 <submit> command with target and password clause; Added
 next slice and frame buttons in serversideviewer
 20101005 Fixed ? imagefilelister (count), added \$ imagefilelister
 (#frames), started on passing frames to serversideviewer
 20101005 Started on WADO
 20101006 Enabled WADO; do not try to remove images from failed
 virtualserver request (double messages)
 20101008 Started on rudimentary WADO client (PatientStudyFinder now
 also lists images)
 20101009 Enabled rudimentary VirtualServer for WADO client:
 loadforgui will try virtualservers
 20101009 Tried to speedup query result processsing; implemented POST
 method (no multipart yet)
 20101009 WADO server and client function; upload button, not
 functional yet as requires multipart parser
 20101010 Set default WADO correct for conquest; finished (single file)
 upload fuctionality;
 20101010 Scan incoming folder once per second (thread starts when
 incoming folder exists on startup); Some fixes in
 serversideviewer; started on VirtualQueryCached
 20101013 Fix : CreateDatabase "root", used UserName as Password for
 postgres
 20101014 Fix in virtualserver for LoadForGUI
 20101016 Improved virtualserver for LoadForGUI; Cached queries start to
 work; virtual queries now return computed fields; use e.g.,
 VirtualServerFor0 = AE,CACHESERIES or
 AE,CACHESTUDIES
 20101017 Added OverlapVirtualGet parameter: controls interleaving of
 recieve and send of virtualserver (suggest: 16)
 20101018 Limited memory use of OverlapVirtualGet, tried out of order
 processing but this failed; dated folders for querycache; Submit
 and scheduletransfer call submit.cq if exists to anonymize
 20101020 Small fix in querycache: query file sometimes not stored; and
 study cache did not read date
 20101108 Added --loadanddeletedir command
 20101116 Warnings
 20101120 Fix reading of FIXKODAK in virtualserver if e.g.,
 CACHESERIES appended; merged bcb warnings (and fixed
 introduced errors); version to 1.4.16releasecandidate; Added
 warning if cached virtualserver does not provide computed
 fields; Added NONVIRTUAL flag to allow block recursive
 virtualserver connections; Do not forward transfer syntax to
 virtual queries data object
 20101121 Fixed NONVIRTUAL flag for non-cached queries if caching is
 ON; Added dgate --export: command to zip/7z/tar selected
 data; Added dgate --safequit: command and OpenThreadCount;
 prepare for ArchiveConverter0 (1900); Show in log when a file

is rewritten; Added [wadoservers] section in cgi config;
Document ul, ub, and ue in acrnema.map

20101122 Pass script to export: command; added ZIP thread that runs when the system is a service; Fix for spaces in zipfilename for export; background exe for 7za

20101124 Fix compile and 2 warnings for linux;
drop zip file extracts with paths to avoid overlapping filenames

20101209 Fixed FIXKODAK parameter detection of virtualservers

20101212 Warn if FIXKODAK paramater set in virtualquery does not match FixKodak global
Fix rewrite message for virtual server data

20101212 Correct file formats listed and accepted for 7za decompression

20101212 Version to 1.4.16

20101213 Implemented NightlyCleanThreshold here for Linux and Windows (if logging to file)

20101220 newuids will change empty uids into a new one

20101222 Fix for empty cached query that crashed

20101227 Fixed some delete NULL crashes on failures

20101227 Added org clause to forward <compressed as xx> to AE org AE

20101227 ImportConverters now also use ForwardAssociationLevel

20101228 Added \n after cloning db message

20110105 Use MakeSafeStringValue for exact match; Added MoveDeviceScript0

20110105 Added --echo and --selectseriestomove commands

20110105 Version to 1.4.16rc1

20110106 Added --moveseriestodevice; changed --selectseriestomove syntax

20110106 Todo: NightlyMoveSource, NightlyMoveDest, NightlyMoveAmount, NightlyMoveAge, NightlyTestAge

20110106 Version to 1.4.16rc2

20110110 Note: mergestudiesconverter called for series and other way around

20110111 Check for empty patient ID in virtual query and for other empty VRs

20110111 merge study converter merges series: now calls MergeSeriesConverter. Merging studies now calls MergeStudiesConverter

20110111 changed logic of converters passing study or series uid to be consistent

20110111 reject or destroy in merge script stops object from being merged

20110113 fix script clause in merge study, call also tries line in [scripts] in dicom.ini

20110113 Allow script clause without "" as well; unrecognized line attempts call

20110114 "" in script string; script command evaluates string as script

20110115 delete changed UIDs after merge study (remerge will create new series)

20110115 fixed "script" converter and substitution of ""; experimental "lua" command

20110116 fixed error message for incorrect script commands; first lua functions built-in

20110116 SearchDicomObject now supports %VName; allow setting/reading more than 10 sequence items

20110116 The set command now accepts names, e.g., set PatientID to "9999"; nop %VPatientID

20110116 Lua now allows, e.g., Data.PatientID = 'aap'; print(Data.PatientID)

20110117 Started on lua Global, Association, Command

20110118 Primary PDU is Extended to allow embedding lua state in it

20110119 Added a global PDU as script context; associations add new contexts

20110120 Fix in Association.Calling/Called; for now each thread has an isolated lua state. Fix in speed of query results (were all processed by ImportConverter). Added lua startup/command/QueryConverter0 etc; count all threads including gui threads

20110121 Added prefetcherPDU for delayed forwards and such

20110122 Fixed lua print, added lua error handling; added %f %b %p to ImportConverter

20110122 Fixed SearchDicomObject for US,SS,UL,SL tags (note: 'set to' not fixed yet)

20110124 bug fixes in new code; implemented reject in lua

20110125 Put UID changes into debuglog

20110127 Added lua getpixel and setpixel (takes 3-5 microseconds)

20110129 Added lua getrow, setrow, getcolumn, setcolumn, readdicom, writedicom; sd.ddo leaks

20110201 Lua: R/W sequence e.g., print('0 implied', Data.InstitutionCodeSequence.InstitutionName) or Data.InstitutionCodeSequence[2].InstitutionName = 'write for now max one level deep'

20110203 Lua: Unified reading and writing of Data/Command/Sequences; Allow write at any depth in existing sequences

20110204 Allow creating sequence in script and in Lua: Data.xxxx = {}

20110205 Added newdicomobject and deletedicomobject

20110206 Fix for set .. to ""; added set .. to nil for simple VR; create sequence will not overwrite existing sequence; lua web page creation: CGI(), HTML(), <?lua ?> <%= %> source=xx.lua

20110207 Added lua: get_amap(N) -- N starts at 1, dbquery(db, flds, query, sort) { returns table of tables starting at 1)

20110208 lua gc for dicomobject; lua dicomarray and dicomquery; allow virtualquery without setting level in DDO

20110208 1.4.16rc4

20110214 Fixed lua Data access; fixed "" in exportconverter; 1.4.16rc4a

20110216 Fixed calling and called in ExportConverter; fixed %f etc in ImportConverter; mkdir IE converter also creates subdirs; 1.4.16rc4b

20110228 Added Write, Read and Dump methods to lua dicomobject as closure with passed method name

20110320 Fixed Association.Calling in lua; Added RejectedImageConverter0; added t_1500 checks to all queries

20110320 1.4.16rc5

20110326 1.4.16rc6

20110328 Fix crash when setting lua Filename for import file

20110331 Added NoDICOMCheck option: if set, do not stop parsing DICOM stream on error; FlushPrivateProfileStringCache after writing to dicom.ini

DGATE.HPP (part of DGATE.EXE):

- 19980705: Added CACHEDevices and JUKEBOXDevices, FindPhysicalDevice
- 19990108: Added LoadImplicitLittleEndianFile and NKI PrivateCompression algorithms
- 19990109: Regen has extra parameter
- 19990317: Changed parameters of LoadImplicitLittleEndianFile
- 19991117: Added parameter FileCompressMode to prototype of nki_private_compress
- 20000629: Added TroubleLogFile and UserLogFile
- 20010415: Added KeepImages flag to RemoveFromPacs: clear from DB only Added SubDir parameter to regen to allow regen of one directory only
- 20010416: Added ChangeUID routine and RegenFile - to allow modification of images
- 20010429: Changed decompressor: now has extra parameter
- 20020529: InitializeTables now has mode parameter (0=normal, 1=simple)
- 20021018: GenerateFileName has NoKill option (for interactive dgate tasks)
- 20030120: Added prototype of FreeDeviceTables
- 20030701: compression parameter for amap
- 20030703: Added prototypes of recompression functions + ArchiveCompression
- 20030706: Optional filename parameter for recompress
- 20030706: Export VRType for implicit little endian support
- 20030921: Added DEVICE_TYPE_MIRROR
- 20040401: Added Changed and ActualMode flags to compress routines
- 20040626: Added study and series UID to getfilename interface
- 20040930: Adapted return type of SetString; added maxlen to BuildSearchString
- 20041013: Added MAXQUERYLENGTH
- 20041029: Added MergeUIDs
- 20041108: Added Syntax input to GenerateFileName
- 20050108: Adapted for linux compile
- 20050109: Added configurable TCPIPTimeOut
- 20050129: Added optional FILE to DumpVR, added CheckFreeStoreOnMIRRORDevice
- 20050401: Added QueryOnModalityWorkList, WorkListTableName, WorkListDB
- 20050404: Added DT_START/ENDSEQUENCE to code sequence in WorkList table
- 20050902: Made space for HL7Tag in DBEntry
- 20051229: Added iDepth to DumpVR
- 20060317: Added called and calling to GenerateFilename
- 20060324: Added StripGroup2 option to recompress
- 20060628: AddToDatabase has JustAdd parameter: skip one unnecessary query
- 20060702: Pass DB to GenerateFilename to avoid zillions of db open and closes
- 20070122: Added MIRRORDevices
- 20070201: Added DebugLevel

20070207: Added MakeTableString
 20080818: DbaseIII check now uses DB flags, not PATHSEP in
 datasource name; Added DT_MSTR
 20090620 Include file stuff
 20091231: Changed char* to const char* for gcc4.2 warnings
 20100111: Merged
 20100123: Added DT_FL and DT_FD
 20100125: Removed linux warning
 20100309: Changed SQLLength to unsigned int
 20100703: Merged
 20110119: Moved two functions to the correct place

LOADDDO.CPP (part of DGATE.EXE):

19980409: added NKI private decompression engine VR 0x7fdf,0x0010
 19980410: added run length encoding of zero differences
 19980410 Fix 16 bit absolute value decode
 19980415 Fix leak on decompress
 19980625 Added compression in SaveDicomDataObject(C10)
 19980703 Made DObjectSerialize thread save (removed statics)
 19990105 Fix: SaveDICOMDataObjectC10 created VR 0x0002 0x0013
 twice. Added comment in SaveDICOMDataObjectC10;
 CleanUp. Put the nki-routines also in dgate.hpp. Added
 LoadImplicitLittleEndianFile
 19990317 Decompression removed from LoadImplicitLittleEndianFile;
 added a VR-sizelimit of that reader
 19990827 NOTE: LoadImplicitLittleEndianFile allocates 100 MB and
 crashes when passing chapter 10 file (happens when extension
 is wrongly .v2)
 19990830 Fixed problem above: 'LoadImplicitLittleEndianFile' uses
 'PDU.LoadDICOMDataObject' in such cases.
 19991117 Added parameter FileCompressMode to in
 nki_private_compress call
 20000629 Logging of trouble now starts with '***'
 20001104 Renamed _LittleEndianUID to _LittleEndianUID_space
 20001105 Fixed where malloc and new were mixed (vr->data)
 20001106 Use delete [] operation for vr->Data
 20001128 Fix in 'Implicit length' code in LoadImplicitLittleEndian
 20010429 Use faster and safer decompressor
 20020412 Added class CBufferedIO for faster reading from DVD (stream
 functions do not seem to be caching.)
 20020415 Removed diagnostic time code (did not compile on ms4.2).
 Made buffer size dependent on iVrSizelimit (i.e, use smaller for
 regen, slightly faster)
 20030706 Attach VRType to PDU's for implicit little endian support
 20041130 Documented crash on win2000 server in code - remains to be
 fixed – rare crash related to read error of DVD
 20050107 Adapted for linux compile
 20051219 Use ReAlloc instead of new BYTE[] to fill VR data
 20070308 And little/big endian.
 20081116 Adapted for very large objects (use unsigned int for length)
 20090211 Safety checks in LoadImplicitLittleEndian to avoid crash on
 corrupted V2 file; Removed lots of dead code
 20090412 Fixed for linux compile (tell does not exist)

20091215	Fixed LoadImplicitLittleEndian() when iVrSizeLimit is used (e.g. regen) (assumption that the pixeldata would be the last VR is not correct).
20091231	Changed char* to const char* for gcc4.2 warnings
20100111	Merged
20100210	Fixed reading some old V2 files (-1 VR lenght).
20100309	Changed int to unsigned int, cast m_iNbBytesInBuffer to long (gcc4.2 Warnings)
20100309	Commented out unused (gcc4.2 Warnings)
20100619	Fix gcc4 warnings, improve speed and prevent CBufferIO from being copied.
20100717	Merged

DGATEFN.CPP (part of DGATE.EXE):

19980326:	changed file name generator to ensure correct sorting of image files by external applications
19980327:	Added FileNameSyntax option in dicom.ini; 0=original, 1=allow sorting by name
19980407:	avoid /; <> in filename; for Philips: allow up to 9999 series and 999999 slices
19980409:	do not free passed DDOPtr on failure: crashed the lot
19980415:	Error messages to operatorconsole
19980620:	Check error flag from GetDevice (for out of disk message)
19981005:	Added FileNameSyntax 2: filename starts with SeriesUID_
19981005:	Replaced size%64 by min(size, 63) for correct truncation
19981217:	FileNameSyntax 3: same as 2 but accept errors
19981220:	Fixed FileNameSyntax code
19990117:	FileNameSyntax 4: same as 3 but .dcm extension (forces ch10 format)
19990117:	Avoid attempts to rewrite image that is not on MAG store (e.g., on CD)
19990315:	Do not allow '.' in directory name. In Syntax 3 directory name is now patient ID only (better for ACRVIEW). Better unique filenames by using time + counter.
20000127:	FileNameSyntax 4 now does not truncate patid, as 3 already did
20000131:	FileNameSyntax 4 has patient ID only as directory name (same as 3)
20000701:	Added MaxFileNameLenght option: truncates from start
20010419:	FileNameSyntax 5 on request of Jason Foster - uses patientname as dir
20011109:	Made MAG name check case insensitive
20020416:	Made filename counter 4 digits for when writing >100 files/s
20020416:	Made all instances of time() unsigned
20020426:	Cannot rewrite image not on MAG store is now a message without ***
20020508:	The above message will not lead to a fail of GenerateFileName
20020802:	Added FileNameSyntaxes 6..9; cleaned FileNameSyntax code a bit
20020804:	Truncate patient ID to 32 chars for mode 6 and 8 (make sure len<255)
20021016:	Pass indexed patid to GetFilename for speed
20021018:	GenerateFileName has NoKill option (for interactive dgate tasks)

20021223: Added forbidden chars *, ?, " and : (comment from Thomas Buerer)

20030401: Overrule '.v2' filenames with '.dcm' when object is not ImplicitLittleEndian

20030522: Replaced 'static int counter' by 'unsigned int gl_iCounter'

20030606: Fixed incremental counter when creating UUIDs or generating filenames

20030730: Overrule v2 only when not ImplicitLittleEndian, ExplicitLittleEndian ExplicitBigEndian (c.f. 20020401)

20030819: Allow longer filenames

20030921: Added DEVICE_TYPE_MIRROR TO GetDevice call

20040129: Added FileNameSyntax 10 = all files in one directory

20040301: Small fix for FileNameSyntax 10

20041108: Added Syntax input to GenerateFileName (overrides ini setting)

20050107: Adapted for linux compile

20050126: FileNameSyntax 11: StudyUID\SeriesUID\SOPUID.dcm

20050131: Changed root to 'name' for FileNameSyntax 11

20051210: Clear root output string when image already in database

20060312: FileNameSyntax 12: similar to 11 but uses Modality_StudyID as dir

20060312: Checked it:
12=name_ID\Modality_StudyID\SeriesUID\SOPUID.dcm

20060314: 12=name_ID\Modality_StudyID\SeriesID\SOPUID.dcm

20060314: new filenamesyntax, e.g., %name_%id%\%modality_%studyid%\%seriesid%\%sopuid.dcm

20060317: Added calledae and callingae to GenerateFilename flexible syntax

20060325: imagenum is image number as text, fix imageid, accept all errors

20060402: Fix potential problem when patient ID not there

20060701: Speeded string handling in new filenamesyntax

20060702: Pass DB to GenerateFilename to avoid zillions of db open and closes

20070125: Fix %modality; add %studydate and substring (year = %studydate[0,3])

20071027: Extended length of FileNameSyntz parameter to 256

20071027: Stripped spaces of called and calling in flexible filenamesyntax

20071030: Small fix in the above

20071104: Added %series: 4 digit series number

20090620: Include file stuff

20090620: Include file stuff

20090926: Added %studydesc and %vxxxxx,yyyy etc

20090927: Added Cleanup ro %vxxxxx,yyyy

20091231: Changed char* to const char* for gcc4.2 warnings

20100111: Merged; but cast Time to print to (unsigned int)

20100125: GetPrivateProfileString -> MyGetPrivateProfileString

20100126: Fixed readout of studydate (typo mix with studydesc)

20100303: Changed int to unsigned int, commented out unused variables (gcc4.2 Warnings)

20100703: Merged

20100828: Update history fix

20101222: Fixed that ascii codes > 128 where replaced by zero (truncated) in Cleanup; now kept

20110201: Changed prototype of SearchDICOMObject

DEVICE.CPP (part of DGATE.EXE):

- 19980620: Setting MAGDeviceThreshHold to 0 causes out of disk error below 10 MB without cleanup
- 19980620: Implemented PanicKillOff thread here as alternative to spawning killer.exe
- 19980702: Started support functions for maintaining/determining amount of used space on device
- 19980704: NOTE: JukeBox code only TruncateFieldNames compatible for now; used mp's filecopy
- 19980709: Finished first version of archive options: prepare, undo, copy, compare, delete
- 19980721: Set default # cache and jukebox devices to 0
- 19990110: Made strings longer since got truncated, fixed some messages
- 19990110: Archive one MAG device or MAG?; fixed SQL syntax for update; speeded 0x7ffffff. Return False if SelectLRUForarchival selects no data
- 19990111: Reset archive flag reports to operator console; protected against illegal device#
- 19990113: Run up to 10 patient further to try and fill CD more exact; max patients = 10000
- 19990114: Added quick routine MakeListOfPatientsOnDevice and used instead of 0x7ffffff
- 19990117: Accept any, not only *.v2, for computing patient size, assume max file size=30 Mb
- 19990630: Added support for MIRROR devices (e.g., try MIRROR1 if MAG1 fails)
- 19990707: Fixed default MIRRORDevices (now 0)
- 19990708: NOTE: archiving should make use of mirror device if required as well!
- 19990712: Fixed that MagRampage thread was started over and over again (fixed for WIN32 only)
- 19990831: Fix: Several BOOL functions returned -1, which is TRUE !!
- 20000621: Changed layout of this update-history
- 20000629: Logging of trouble now starts with '***'
- 20001104: Fixed aDB.Close -> should be aDB.Close(); added WINAPI to MagRampage thread
- 20010328: Added verify mirror disk
- 20010329: DeleteBunchAfterBurning also deletes data on MIRROR0 if it exists; verify mirror disk keeps on going after error but does return error
- 20010312: Check if we already selected some patients for archiving from the device
- 20010509: Added RenameDevice
- 20010522: Do not call RestoreMagFlags if something goes in wrong in PrepareBunchForBurning
- 20011114: Replaced slow updates for magdevice by much faster ones. The update that now no longer has the IN part might not work for ACCESS or DBF. So: for jukebox purposes, SQL server is required
- 20011115: Replaced 'LIKE' by faster '=' in query strings; fixed previous fix
- 20020412: PanicKilloff will delete now delete 5 MB data at a time. (high water = low water + 5)

20020415	Added TestImages: sanity check of images on disk
20020416	Reversed read order for TestImages and VerifyMirrorDisk: MRU first
20020802	Allow multiple subdirectories in rmdir and mkdir code
20020804	Cleaned up a bit
20021018	GenerateFileName has NoKill option (for interactive dgate tasks); moved ALERT warning
20021020	Simplified and speeded queries for archiving using ImagePat field (requires rev5+ of db); Removed " and added [in MakeSafeString; hand-code one query for built-in dbase driver; Changed SQL construct in SelectLRUForArchival for compatibility with built-in driver
20030120	Added FreeDeviceTables
20030702	Start on ArchiveCompression; for now KB is specified -before!- compression
20030705	Replaced ProcessDDO by more correct DecompressNKI to check pixel data
20030706	Attach VRType to PDU's for implicit little endian support
20030724	Fix in GetKBUsedForPatient: missing %s in printf
20030819	Allow longer filenames
20030905	Allow longer filenames when reading device paths
20030921	Added DEVICE_TYPE_MIRROR in GetDevice; allow PanicKillOff to be called with variable threshold
20031217	Fix: Support harddisks bigger than 2 GigaByte!!! in CalcMegsOnDevice (Thanks to Clifford Sweet)
20040403	Moved ArchiveCompression to SelectLRUForArchival -> fills disks correctly
20040614	Added MoveDataToDevice
20050107	Adapted for linux compile (no threads yet, no check of disk space on device)
20050109	Added threads for linux
20050118	Detach thread after starting it to avoid leaks
20050119	GetKBUsedOnDevice is not used --> no problem to return -1 under linux
20050414	Fix CalcMegsOnDevice for relative path
20050421	Fix: MAGThreshHold is space for largest file != space for cleanup. Now set to 30 MB independent of MAGDeviceThreshHold
20050901	Fix free space check for unc paths (error counting \)
20060311	When GetDiskFreeSpaceEx fails allow storing data anyway (fix for Marius Petruc)
20061213	Fixed nasty bug in MakeListOfPatientsOnDevice: Query without bindfield overwrites previous bound strings!
20070316	Cleanup now starts at MAGDeviceThreshHold to MAGDeviceThreshHold+1000, was start on 30 to MAGDeviceThreshHold; Allow both MAG0 and MAG0.Archiving as input for PrepareBunchForBurning
20071027	Protect MakeListOfPatientsOnDevice on MAXPATIENTS; LRU selection now configurable using routine MakeListOfOldestPatientsOnDevice. E.g. if LRUSort = StudyDate, deletes or archives least recent scanned patients first. The default "" means that the original sort order on db entry date is kept
20071031	Catch out of memory errors and return error status if so

20071118 Adapted for 64 bits (use SQLLEN for BindField)

20071128 Fix by kamil.krasnik of Cache device disk space test for UNIX
Also fix Cache and Mirror for UNC paths

20080126 Fixed bug reported by Ali: second move to MAG1 of more data
if same patient would fail on delete. Added new logic: attempt
to delete file from MAG0 that is not in MAG0 but is in MAG1
is OK. Similar attempt to delete file from MAG0 that is in
MAG0 but not in MAG1 is an error

20080302 Fixed case of DICOM table names

20080317 Fixed deletebunchafterburning change (Ali)

20080604 Modified device searching order when file not found:
JUKEBOX now also searches other JUKEBOX

20080617 Fixed bug in above: second jukebox not accessed correctly

20080818 DbaseIII check now uses DB flags, not PATHSEP in
datasource name

20081016 Fix check of max patients in
MakeListofOldestPatientsOnDevice

20081120 Added UseEscapeStringConstants and DoubleBackSlashToDB

20081124 restrict PanicKillOff to 10 runs (to stop when no patients left)

20090821 Added new FileExists function that uses stat; maybe does not
try to mount tape or disk to early

20090926 MakeListofOldestPatientsOnDevice returns newest if Max<0
SelectLRUForArchival always use new code (use KB<0 to
select MRU); Default MakeListofOldestPatientsOnDevice use
AccessTime to emulate old code

20091231 Changed char* to const char* for gcc4.2 warnings

20100111 Merged

20100120 Fixed two const issues detected with ms8

20100124 Use MyGetPrivateProfileString

20100309 Added double parentheses (gcc4.2 Warnings)

20100309 Changed int to unsigned int, commented out unused variables
and routines (gcc4.2 Warnings)

20100619 Added UNUSED_ARGUMENT.

20100717 Merged

20100901 Added IgnoreMAGDeviceThreshHold

20101121 Added ArchiveConverter0 (1900)

20110105 Do not stop deleting on single error; added
MoveDeviceConverter0 (2000)

20110105 SelectSeriesForArchival: selects data older than age days
(age>0) or younger (age<0) from device

20110106 added kb limit to SelectSeriesForArchival; Added
moveseriestodevice

20110106 fixes in series code and new MakeSafeString; #if code to allow
select on AccessTime instead of SeriesDate

20110119 Pass PDU to some archiving code as script context

20110320 Added MAGDeviceFullThreshHold, system will only write to
device if more than # MB avail (def 30)

DBSQL.CPP (part of DGATE.EXE):

19980321: Changed ObjectFile and DeviceName field length to 250 for
dbf compatibility

19980321: Added define TRUNCATEFIELDNAMES for DBF
compatibility (apply in makefile)

19980322: Added run-time check on DbaseIII (should also be added in parse.cpp somehow)

19980325: Added TruncateFieldNames entry in dicom.ini to allow DBF compatibility

19980327: Made MaxFieldLength configurable

19980327: Set default event notification to UDP (for personal pacs)

19980407: Added FileCompressMode config

19980409: Check vr!=NULL in MakeSafeString, MakeSafeDate

19990630: Remove also files from MIRROR disks

19990707: Added UpdateOrAddToTable to refresh database contents without first removing

19990707: Cache Updates; and AccessTime with ~31 to allow update times to be the same

19990707: Added fix for Philips PATID; fix cleaning of MIRROR device

19990708: Added (and disabled) stress code for SQL server

19990712: Fixed FIX_PHILIPS; atoi failed sometimes because vr->data not zero terminated

19990827: Added creation of key and link indices

19991117: Added a check on ini-file value of FileCompressMode

20000621: Changed layout of this update-history. Fix: In 'SetString', type DT_UINT32 was treated as DT_UINT16 Added TroubleLogFile and UserLogFile

When updating PatientRecord, verify the new Name and BirthDate

20000629: Logging of trouble now starts with '***'

20000701: Remove *all* trailing spaces in MakeSafeString before enters database

20000703: Merged changes: made patient change verification table and record name independent and extended to other databases as well; removed trailing before comparison; show patid on all error messages

20000705: refuse inconsistent update of link: show error on operatorconsole

20000707: fixed above addition

20001104: Replaced _rmdir by equally valid rmdir

20001106: Use delete [] operation for vr->Data

20001128: Fix: protected 'GetFileName' with a critical section.

20010415: Added KeepImages flag to RemoveFromPacs: clear from DB only

20010416: Added ChangeUID routine and UIDMODS table - to allow modification of images

20010418: Changed names of database fields in UIDMODS - did not work on msaccess

20010429: Renamed genuid routine and variables to allow compile in one big file

20010713: Reduced devicename to 32 chars to space space

20010902: Improved UpdateCache to be thread safe and made it longer, no test on image db.

20011109: Made MAG check and database field consistency check case insensitive.

20011115: Replaced 'LIKE' by faster '=' when possible

20011116: Fixed error handling of db entry. It was possible that db entry failed without any message!

20011121: Create indices on ImagePat and SeriesPat if these fields exist

20020403 Do not overrrule the PatientTable with: empty data, or birthdate
xxxx0101, or sex other than MmFf.

20020409 made LastPatid non-static

20020409 Removed non-thread safe caching of LastADBE

20020409 Moved entry into UpdateCache to after succesful Update

20020412 Made NCACHE configurable and changed it from 16 to 1024
(8 MB); Set lastfield (limit of #fields) to 20 (was 16)
Added checksum code in update cache to make it faster

20020412 Made key fields unique. Reversed writing of db's. In this case
a db write fail will still result in a valid db. The db write code is
not multi-user safe. A new record may be written at the same
time by two processes -> fail

20020413 Set NCACHE to 256, enough for single thread regen
Modified another non-thread safe construct

20020415 Fix display of PatientID in 'Inconsistent' error message

20020416 Made UID generator configurable through UIDPrefix in
dicom.ini. Made all instances of time unsigned. Note:
accesstime in the db is signed int: problem around 2038

20020418 Added TimeStamp, Flags, and Spare fields on all dbs for future
use

20020430 Fix in spare fields: name was reserved in some odbc
implementations

20020515 Process all db fields; enter (1) Modality into StudyModality

20020517 Modality is merged into StudyModality without size limit
(enforced by sql server)

20020518 Skip missing VR's again; max nr fields to 30; fix reading of
NULL fields (clear s[][])

20020519 Also skip writing empty VR's (vr->Length==0); made
FixPhilips configurable (default 1). Remove / from date to
allow writing mm/dd/yyyy into sql server

20020522 Test on empty Modality vr

20020529 Added PRIMARY KEY to key DB fields (needed for
replication)

20020529 InitializeTables now has mode parameter (0=normal, 1=simple)
Do not make Index for primary keys (is done automatically)

20020609 Added compression modes 3 and 4

20020802 Simplified rmdir code and allow multiple subdirectories

20020804 Use PATHSEPCHAR where possible

20020806 Fix: cleanup entries of refused item in database

20020819 Added better UID generation

20021016 Use patid in SaveToDatabase and GetFileName query
(both test for rev5 or higher of the database)

20021017 Allow set of UIDpostfix from extern (-c option of dgate)
Removed creation of unused tables

20021020 Added [to MakeSafeString; removed "
Note: VR's containing ', ? give spurious inconsistency warning
when rewritten

20021028 Also removed dropping of unused tables; reversed table search
to image.pat in FindDBE. Made inconsistent birtdate a severe
error message (added ***)

20021110 Made date range inclusive (bug reported by Aaron Cann)

20030113 Added PatientQuerySortOrder etc overrides for sorting

20030205 Fix potential empty patient ID and SOP problems in
GetFilename

20030321	Extra check on vr!=NULL in SaveToDatabase
20031104	Added \\ to MakeSafeString
20040301	Do not double \\ when already doubled
20030425	Added DoubleBackSlashToDB control for \\
20040528	Pass patient ID in RemoveFromPacs and Enable KeyOnAll
20040529	Implemented faster delete from database
20040530	Pass a field to query in delete - obligatory for sql server
20040601	Removed enable KeyOnAll - added some more checks in NewDeleteFromDB
20040609	Check for DB revision for fast delete from database queries
20040610	Truncate too long VR's before putting into db; log to pacstrouble
20040623	Fix for non-DBASE datasources in 'UpdateOrAddToTable'
20040626	Added study and series UID to getfilename interface
20040713	Added FixKodak: replace pat 0nnnnnnnn with nnnnnnnnn
20040722	Defaults for FixKodak and FixPhilips to 0 (!)
20040930	Added multiple UID matching: UID\UID\UID in BuildSearchString (list truncates at maximum length) SetString now returns a char * which is newed (with lots of extra space for MakeSafeString which is done in-place) if Max length is passed 0. DICOM2SQLQuery has no max length anymore
20041003	Truncated printf's of SqlStatement
20041029	Added MergeUIDs routine
20041101	Added NewDeleteSopFromDB
20041220	Fix: in 'RemoveFromPACS', an even PatientID was not zero-terminated !!
20050109	Adapted for linux compile
20050111	Added ctype.h (for mingw)
20050205	Added counters for database activity
20050206	Allow empty DBE in ConstructVRFromSQL: used to get ObjectFile and Device in query
20050401	Added WorkListDB and WorkListTableName
20050404	Dicom types DT_START/ENDSEQUENCE are ignored during table creation/adding to table
20050414	Allow undefined worklist database; Delete patient will delete worklist entries as well (but needs at least one image)
20050417	Delete from DB will also delete worklist as patient goes
20050826	Added mode 2 for initializetables: clear/create worklist only
20050831	Mode 0 and 1 will now not delete worklist if it contains data
20050831	VerifyIsInDBE will not return DT_START/ENDSEQUENCE
20050901	Do not call FixImage in SaveToDataBase: is done in SaveToDisk before
20051121	Enabled SQLRealSize because SQLResultLength does not work in PostGres BindField
20051123	Fixed it, and only enable it when #ifdef POSTGRES
20051204	Disabled it, fixed postgres driver instead
20051210	GetFilename returns FALSE for an image that has no filename or device set
20051212	Clear update cache when any records deleted - drastic but safe
20051216	Small fix in above
20051217	Use new VR::ReAlloc method to replace data in VR

20051218 Fix study modality when DoubleBackSlashToDB=1; changed modality search

20051228 Truncate invalid trailing spaces (>1) from patient ID. Fiximage warns to SystemDebug

20060210 Removed order: first remove spaces and then fix the patient ID because patient ID consisting of 14 spaces caused huge troubles

20060220 Check max 50 fields when update; fix Inconsistent check for DoubleBackSlashToDB

20060607 Fix character translation for entering *?%[' into sql using MakeSafeStringValue. Fix consistency check for these characters as well (thanks Robert Chrzan)

20060607 Added "not null" to "PRIMARY KEY" to satisfy Sybase (thanks Marcin Litoborski)

20060618 Added not null to UIDmods as well, unified case of DICOMWorkList table name (may be done for other tables as well) (thanks Marcin Litoborski)

20060618 Remove " added by MakeSafeStringValue prior to inconsistency check (thanks Robert Chrzan)

20060621 Increased size of s from [50][255] to [51][256] in UpdateOrAddToTable fixes VirtualQuery crash

20060628 AddToDatabase has JustAdd parameter: skip one unnecessary query

20060706 Replaced faulty new vr construct when doing consistency check: faulted on Linux

20061213 Protect Clear_UpdateCache with critical section
Fixed nasty bug in NewDeleteFromDB. MergeUIDs, NewDeleteSopFromDB: Query without bindfield overwrites previous bound strings!

20070123 Fixed GetFileName for patientid with '
Recoded some of the inconsistency checking: now no failure on patient ID (link) with '

20070131 Fixed GetFileName for all patientid's ;->>>>

20070201 Fixed (unused) UpdateAccessTimes for weird patient IDs

20070210 Added CheckDuplicates to UpdateOrAddToTable: used on DicomStudies to catch inconsistent patientID
Enabled KeyOnAll on UpdateOrAddToTable: only delete exact records (needed if inconsistent patientID)

20070308 Changed to set patient ID to 0 for anonymized patients

20070315 Merged; also noted that changed possible use of AND at start of sql where

20071027 Also truncate too long date fields with error message; added AllowTruncate: if sql name in comma separated list: warning only else ***error

20071118 Adapted for 64 bits (use SQLLEN for BindField)

20080302 Fixed case of DICOM table names

20080319 Fixed typo in above change; fixed worklist delete for patients with space in ID

20080817 Added WorkListReturnsISO_IR_100 flag (default 1)

20080818 DbaseIII check now uses DB flags, not PATHSEP in datasource name; Added experimental DT_MSTR: multi-value entry, e.g. query on 'PET' matches 'PET\CT' if studymodality is given type DT_MSTR instead of DT_STR. Note: not implemented for DBASEIII

20080819 Added warning when querying DT_MSTR field on dbase

20080902	added EnableComputedFields (default 0)
20080910	Added create Patient ID for 0 length ID (old Picker CT)
20080910	Missing patient ID is now not filled with "0" but with PatientName + Date
20081104	Fixed crash in UpdateOrAddToTable due to missing BindField (1, ...)
20081120	Strip all trailing spaces of modality prior to merging: some scanners add more than 1; Added UseEscapeStringConstants
20081124	Blocked RemoveFromWorld when 0 images found for safety; default SQLServer to "" = black hole
20090216	Added ChangeUIDBack and ChangeUIDTo
20090616	Create index for UIDMods on OldUID; protected UIDMod functions with ChangeUIDCritical; Convert ** query field to *; old efilm version seems to create those
20090616	Include file stuff
200912308	Merged changes by Bruce (bitsltd)
20091231	Changed char* to const char* and cast time_t to int for gcc4.2 warnings
20100111	Merged
20100123	Added DT_FL and DT_FD types, added swapping in ConstructVRFromSQL, and fix DT_UINT32 and DT_UINT16 there, size is 4 and print as %u
20100123	fixed typo for Linux compile Time -> time
20100124	fixed typo in ChangeUIDBack
20100124	Use MyGetPrivateProfileString
20100309	Added double parentheses (gcc4.2 Warnings)
20100309	Changed int to unsigned int, commented out unused variables (gcc4.2 Warnings)
20100324	swap for big endian only.
20100619	Added #ifndef (gcc4.0 Warnings) and UNUSED_ARGUMENT
20100717	Merged; but still use deprecated atof as change would not compile
20100726	Added cast at atof to suppress warning
20100802	Added SUBMISSIONS table
20100816	Allow dicom.ini and other files to be moved.
20100816	removed (a) from #ifndef EnterCriticalSection(a) etc
20100818	Merged jd SUBMISSIONS table
20100823	Merged and simplified basedir logic
20101121	Do not make AccessionNumber primary key in worklist
20101212	Made FixKodak global
20101227	Added \ prior to _ in query value
20101229	Modified _ to [_] etc; presence of [now also forces LIKE
20101229	% -> [%]; _ -> [_]; [-> [[];] -> []]
20110105	Moved local routines to vrtosql: MakeSafeString, DICOM2SQLQuery, BuildSearchString, BuildColumnString
20110105	Use MakeSafeStringValues for exact match; added DICOM2SQLValue
20110105	Removed SUBMISSIONS table
20110106	Fixed regen crash due to missing vr==NULL check in MakeSafeStringValues
20110111	Extended check for vr in MakeSafeStringValues
20110115	Added DeleteUIDChanges

20110214 MakeSafeStringValue 20110111 change broke dbextract;
fixed now (return " if vr invalid or empty)

20110326 Fixed crash in SetString on findscu --key 0010,0010 --key
0010,0020 127.0.0.1 5678

VRTOSQL.CPP (part of DGATE.EXE)

20000629 Logging of trouble now starts with '***'

20010330 Added a few debug printf's

20010420 Fixed memory leaks in all query levels

20020519 Clear BindField result strings before reading (NULL does not
read)

20021028 Restructured queries to give lowest level of de-normalized
databases. Fixed sorting on PatientName in denormalized study
query

20021030 Reversed this again apart from study level because SQL server
becomes very slow

20030113 Added PatientQuerySortOrder etc overrides for sorting

20030114 Added in querycolumns (even if already present);
tested for SQL server, requires testing for other datasources

20030122 Remove unused entries from Tables string in queries

20040930 Started adapt such that query strings etc and not limited in
length. For now: query string max 8192 (here and in
odbc.cpp); sql statement max 16384 (in odbc.hpp)
Enough for query with one multiple values UID of about 100
images (list truncated if too long). Added safestrcat protection
of SearchString

20041003 Truncated debug prints, malloc strings at image level.
Analysed string lengths; limited 'records =' debug log

20041013 Used MAXQUERYLENGTH

20050107 Removed UNIX flags: solve difference in database interface

20050206 Image query can send filename and device in 0x9999,0x800 /
0x0801

20050401 Added QueryOnModalityWorkList; - todo - put selected items
into sequence 0040,0100

20050404 Take query for Scheduled Procedure Step from sequence, put
results into sequence

20050414 Made sequence unfolding for worklist more generic - accepts
any number of and nested N=1 sequences. This is correct
behavior for query input, but limited for query results

20050417 Removed unused variable

20050831 Fixes in worklist code: is ok for all OFFIS tests except 1 (empty
seq not returned) and 10 (undef non-empty seq takes one entry
from higher level) - good enough for initial release?

20050901 Fix for test 10: allow undefined sequence in query

20050907 Merged change by Hans-Peter Hellemann: fix missing max()
macro

20051229 Debug log show records of all queries. DumpVR only left in
worklistquery and shows sequence layout

20051230 Removed 2 forgotten dumpvr's

20050103 Added debug info from testing phase inside nested sequences
for modality query

20050211 Added empty required modality sequences 8,1110 and 8,1120
when not there

20050224 Fixed modality worklist query: empty result sequences were not handled correctly (Thanks Tony Tong)

20050311 Worklist change was tested OK with Agfa (thanks Frank Grogan), cleaned debug log a bit

20060607 Fix crash when coding empty sequence as last item in worklist query (thanks Francois Piette)

20061219 Small fix in layout debug log

20071118 Adapted for 64 bits (use SQLLEN for BindField)

20080817 Fixed bug found by larc: worklist sql syntax error (fields start with ,) when 1st item is sequence; Fixed bug found by Alberto Smulders: sometimes sequence level in response would not come down; The problem was that a higher levels the reported sequence level was inconsistent with the 1st level; Changed CodeSequence stop criterium and level coding for deeper levels; now works with varies sequence combination; Added WorkListReturnsISO_IR_100 flag (default 1)

20080818 DbaseIII check now uses DB flags, not PATHSEP in datasource name

20080901 Implemented 'Number of Patient Related Studies' (0x0020, 0x1200) etc

20080902 Fixed that option for VirtualServers; added EnableComputedFields

20080905 Added void* cast for deivr change

20081016 Fixed for WC compile

20081121 Fixed ISO_IR 100

20090930 Fixed ImageQuerySortOrder: crashed on C-MOVE, because sortcolumn was inserted before the filename

20091005 Fixed ComputeField: return VR was used during cleanup

20091231 Changed char* to const char* for gcc4.2 warnings

20100111 Merged

20100309 Added double parentheses (gcc4.2 Warnings)

20100309 Changed int to unsigned int, commented out unused variables (gcc4.2 Warnings)

20100706 Init Level

20100717 Merged

20100822 Delete 9999,0802 virtualservermask control VR from searches and pass it to ComputeField

20100823 Fixed compile for ms8amd64

20100901 Rephrased "Failed on VR Search...."

20101003 Delete 9999,0900 script control from queries

20101120 Delete 0002,0010 transfer syntax from queries

20101127 Added CountOnly mode to accelerate e.g., NumberOfStudyRelatedInstances queries

20110105 Moved local routines here: MakeSafeString, DICOM2SQLQuery, BuildSearchString, BuildColumnString

20110105 Pass database to MakeSafeString to allow db_type dependent processing; use LIKE only when needed

DPRINTF.CPP (part of DGATE.EXE):

20000211 Removed MajorEvent and MinorEvent instances of 'Debug'.
Cosmetics. Added UserLog and TroubleLog.
Added timestamp option.
Added CriticalSection when printing to file or stdout.

20011110	Post release 1.3.10: keep log files closed while working
20021027	Avoid crash when logfile cannot be opened
20041003	Note: max 1200 chars (use %.1000s to print long strings)
20060618	Define _SH_DENYNO if needed
20070105	Added timestamp for linux users (thanks, Mark Pearson)
20070406	Use reentrant ctime_r and localtime_r when available
20070408	Protected entire .printf() with global critical section
20070712	Closed/opened critical section where recursive call to printf would hang linux version with sqlite
20071120	Use fputs instead of fprintf at end: otherwise % removed
20080821	Fix for Solaris SUNwspro compiler
20090620	Include file stuff
20091231	Changed char* to const char* for gcc4.2 warnings
20100111	Merged
20100125	Removed linux warning
20100309	Removed Windows only variable from unix build.
20100619	Added #ifndefs made Port a local IPort (gcc4.0 Warnings)
20100717	Merged
20100816	removed (a) from #ifndef EnterCriticalSection(a) etc

DPRINTF.HPP (part of DGATE.EXE):

20000211	Removed MajorEvent and MinorEvent instances of 'Debug' Added UserLog and TroubleLog Added bAddTimeStamps member
20011110	Replaced MessagePipeName by FilePipeName Blocked out critical section
20091231	Changed char* to const char* for gcc4.2 warnings
20100111	Merged
20100619	Fix gcc4.0 warnings and prevented coping of classes with pointers.
20100717	Merged

REGEN.CPP (part of DGATE.EXE):

19980327:	put messages in one SystemDebug for UDP message layout
19980415:	changed messages to OperatorConsole
19980605:	Need to add regen for optical devices and cachedevices !!!!!
19981218:	temp use pdu.load
19990108:	Replaced PDU.Load by LoadImplicitLittleEndian in RegenToDatabase
19990109:	Set HeaderOnly flag for LoadImplicitLittleEndianFile to TRUE. Added regen of CACHE and JUKEBOX devices; NOTE: full regen imposes name restrictions on JUKEBOX and CACHE directories
19990117:	Filenames with .v2 are assumed raw; added extension .dcm for chapter 10 format
19990317:	Parameters of LoadImplicitLittleEndianFile have been changed
19990521:	.img files react same as .dcm files
20000629:	Logging of trouble now starts with '***'
20010415:	Added SubDir parameter to regen to allow regen of one directory only
20010416:	Added RegenFile - to allow modification of images
20011109:	Made file extension checking case insensitive

20020409: Experimental multithread version.
 20020413: Made NTHREADS variable and tested. Time to regen 6900 files about 3 min and 40 s.
 20020415: Tested with new loaddo code by ljz. Note: multithread works and has allowed fixing MT some errors in other modules. However, due to a problem in concurrent db writing (in UpdateOrAddToTable) it should not be used clinically.
 20020613: Removed some warnings
 20020802: Simplified regen code; traverse subdirectories recursively (allow e.g., e-film data format)
 20020804: Use PATHSEPCHAR where possible
 20021014: Exclude directory PRINTER_FILES during regen (contains slices without ID and such)
 20030120: Removed some warnings
 20030706: Attach VRType to PDU for implicit little endian support
 20030819: Allow longer filenames
 20040614: Do not regen CACHE devices; first regen JUKEBOX then MAG (e.g., in historical order)
 20050118: Adapted for LINUX compile
 20060628: AddToDatabase has JustAdd parameter
 20070210: Increased maxvrsiz for regen v2 files from 0x400 to 0x4000 -> UIDs were sometimes not found
 20070308: Ignore .DS_Store for DARWIN
 20091231: Changed char* to const char* for gcc4.2 warnings
 20100111: Merged; fixed WIN32 changes

PARSE.CPP (part of DGATE.EXE):

19980321: Added define TRUNCATEFIELDNAMES for DBF compatibility (apply in makefile)
 19980325: Added TruncateFieldNames entry in dicom.ini to allow DBF compatibility
 19980327: Added MaxFieldLength entry in dicom.ini to allow DBF compatibility
 20001105: Renamed some enums and variables to allow compile in one big chunk
 20010713: Renamed DT_UINTxx to DT_INTxx as temp fix for bug in LEX.CPP
 20010829: Undone above change as LEX.CPP has been fixed by ljz
 20050107: Added include file gpps for GetPrivateProfileString
 20050401: Added WorkList Support
 20050404: Added DT_START/ENDSEQUENCE to code sequence in WorkList table
 20050902: Added HL7 column to be used for WorkList table
 20050908: Fixed pointer error that crashed linux version
 20080818: Added DT_MSTR
 20100122: Hex2Dec incorrect for uppercase HEX numbers!
 20100123: Added FL and FD types
 20100125: GetPrivateProfileString -> MyGetPrivateProfileString
 20100309: Changed int to unsigned. (gcc4.2 Warnings)
 20100703: Merged

LEX.CPP (part of DGATE.EXE):

20000822	Created
20000823	Support cases where the input-file is empty or contains only white-space.
20010716	Fix: Allow TOKENs that are substrings of other TOKENs.
20050103	Changed include file to lower case (for linux)
20050108	Removed window.h for linux compile
20050118	GNU C++ knows BOOL etc, do not redefine
20090620	Include file stuff
20100120	Fixed const issue detected with ms8
20100309	Changed int to unsigned int, add ui as unsigned int in for loop (gcc4.2 Warnings)
20100619	Fix gcc4 warnings, improve speed and UNUSED_ARGUMENT.
20100717	Merged

LEX.HPP (part of DGATE.EXE):

20091231	Changed char* to const char* for gcc4.2 warnings
20100111	Merged
20100619	Added no-copy to the classes.
20100717	Merged

NKIQRSOP.CPP (part of DGATE.EXE):

19990317	NKI-specific code moved here
19990903	Implemented NKI MaxRowColumn VR in DCO
19990906	Improved implementation of MaxRowColumn
19991022	Changed the root of NKI-made UIDs to the official one supplied by Medical Connections(UK): 1.2.826.0.1.3680043.2.135.1066.xxxxxxxxxx
19991122	Added NKI-CompressionMode 2: safer, with CRC checks
20000131	Regardless the FileCompressMode in Dicom.ini, always send images compressed (mode=2) when communicating with an NKI application.
20001105	Fixed where malloc and new were mixed (vr->data)
20001106	Use delete [] operation for vr->Data
20001106	Temporary fix of BC leak in CompressNKI and DecompressNKI
20001120	Fix bug - vr->data was reallocated without changing vr->length
20001127	Fix: Crashes happened when more than one ServerChildThread' was active. m_pDCO is wrongly shared by all threads !!
20010424	Plugged in faster and safer decompress
20010426	Fixed leak when connection is aborted
20010429	Interleaved CRC computation with compress code (factor 2 faster) Added GetADD0 to allow read ahead within calling program
20010501	Merged liz's change of 20010426
20010502	Added extrabytes pointer to Read and RetrieveOn
20010502	Added vr(9999,0300)=SliceLimit: send at most so many slices
20010502	Changed this vr to (9999,0500)=SliceLimit
20011219	Added extra check in case other manufacturers than NKI use VR (0x7fe0, 0x0010), such as ACUSON
20020317	Replace true by TRUE

20020415 Added error handling on ProcessDDO
 20020429 fixed reporting of number of complete slices (now starts with 1)
 return cancel status if C_STORE failed
 20020609 Added compressor modes 3 and 4
 20020609 Added VR (9999,0600): MaxCompressionLevel (default 2)
 Note: This level (max of 4) is also used when downsizing
 20020613 Exported DecompressNKI and CompressNKI
 20020619 Damn! ELSCINT uses (0x7fe0, 0x0010) too (see modification
 on 20011219). Extra check on zero-length pixeldata
 20020930 Fixed SliceLimit VR
 20021016 Removed __BORLANDC__ language construct - to fix
 downsize with BC55
 20021115 Added Generic style retrieve classes; print error messages from
 c-move
 20030522 Added function ComputeCRC
 20030605 Fix in DecompressNKI: dgate crashed when DCMTK's
 'storescu.exe' wants to send an NKIcompressed image to dgate;
 it refuses to send the pixeldata!
 20030701 QualifyOn now also has compression parameter; transfer syntax
 work in progress
 20030703 Tested recompress functions
 20030704 Disabled decompression in ProcessDDO for non-nki clients; is
 done by recompress. When downsizing use
 DecompressImage(); also supports JPEG; Removed
 MaybeRecompress. Recompression for NKI clients only when
 downsizing; otherwise as specified in acrnema.map
 20030705 Moved recompression out of ProcessDDO into
 StandardRetrieveNKI::Read. Skip recompression to same level
 as incoming
 20030706 Optional filename parameter for recompress (not used yet); set
 env for offis tools
 20030706 Attach VRType to PDU's for implicit little endian support
 20030709 Removed DCMDICPATH; is not needed
 20030819 Allow longer filenames
 20030905 Note: recompressfile; DecompressImageFile and
 CompressJPEGImageFile do not allow spaces in filename
 20040401 Added Changed and ActualMode flags to compress routines;
 only give compress message if something changed
 20040401 Set maxcompression to 4 for NKI send
 20040403 Added own SaveDICOMDataObject as in dgate.cpp
 20040406 Also copy TypeCode in DecompressNKI when converting
 (0x7fdf, 0x0010) to (0x7fe0, 0x0010); Make TypeCode of
 (0x7fdf, 0x0010) OW instead of OB, because OFFIS refuses to
 compress OB
 20040406 Always use EXPLICIT syntax for jpeg, fix v2 always use
 DUMP; Use -F +ti option for dcmdjpeg for writing to v2:
 creates raw dump, implicit; Refuse to jpeg compress V2 files
 and images != 16 bits; \n is all messages; Fix error handling of
 CompressJpegImage
 20040530 Removed unused variable
 20040722 Added "nj" compression: leaves JPEG as is; else
 MAXNKICOMPRESSION
 20041101 Added built-in JPEG and RLE decompressor

20041102 Fix problem with external JPEG compression: clear transfer syntax prior to save for OFFIS tools with DICOM_CHAPTER_10_EXPLICIT

20041103 Added k1,k2,k4,k8 compressors: downsize to 1024,512,256,128 pixels if bigger to start with

20041112 Called AE SERVER~xx in C-MOVE lets xx override outgoing compression as well (e.g., call CONQUESTSRV1~k4 to always get images downsized to 256x256)

20050102 Added NOINTJPEG flag to allow compile through total.cxx

20050103 Changed include file to lower case (for linux)

20050108 Adapted for linux compile (work in progress)

20050109 Added configurable TCPIPTimeOut

20050118 replaced thread local storage under linux with variables in object

20050119 added rudimentary support to start external (de)compressors under linux

20050121 Changed filename to lower case

20050122 Improved external jpeg support under Linux: load dictionary, wait until ready

20050130 Added To8BitMonochrome and ToGif

20050204 Small fix in ToGif for non-square images; also interpret RescaleIntercept as float

20050205 Added counters for compression activity

20050206 Optimized downsizing: do not decompress if no downsizing to occur

20050206 Note: replaced thread local storage under linux with variables in object is incorrect. Under linux, the server now can also serve one at a time. Fix for color to gif; added limited auto scaling in 16 bits to gif.

20050211 Now really removed need for thread local storage

20051210 Added "vX" compression (same as as is)

20051217 Use system() to call on offis apps in linux (now works)

20051217 Use new VR::ReAlloc method to replace data in VR; fixed 2 float warnings

20051229 Fixed several leaks in (jpeg)compression code; changed alloc for nki compression

20051230 Added +sr option to dcmjpeg: compress without changing rescaleslope/intercept
This option is required for our viewers: slices must all have same rescale

20060219 Removed warning in writegif

20060222 Added extra checks (not on ID strings) to pass as NKI compressed data

20060311 dcmjpeg opts string got truncated (thanks Rainer Libowski)
fixed unused CompressJPEGImageFile (wrong exe called)
external decompress temporary files now in printer_files (thanks Rainer Libowski)

20060324 Added StripGroup2 option to recompress; strip group 2 from outgoing c-stores if not compression "as" or "is"

20060328 Use IsAbstractSyntaxAccepted(iUID) to test if host accepts image

20060402 Fix crash when recieving or dropping jpeg compressed color images. Consider not removing transfer syntax (0x0002,

0x0010) prior to dcmdjpeg/dcmcjpeg. Would fix problem with external decoding color jpeg images

20060402 Changed IsAbstractSyntaxAccepted(iUID) test back (thanks Andrej Savelov)

20060405 That was not the problem

20060618 Tested IsAbstractSyntaxAccepted - seems to work. Removed need for min()

20060619 Do not remove transfer syntax before decompression (20060402 suggestion). Maintain original object when decompression fails, changed wording of message. Only complain about internal decompressor for jpeg

20070104 Export automatic SaveDICOMDataObject

20070113 Never delete transfer syntax (was left in CompressJpegImage, solves coding error in jpeg store). Tested with \$c server debug command

20070210 Export MaybeDownsize and check on 0 for passed size

20070307 Added several debug prints in retrieve, e.g. at bail out because connection dropped

20070308 removed #include <wait.h> from DARWIN

20070314 Send 0000,1030 (MoveOriginatorAE) and 0000,1031 (MoveOriginatorMessageID) in SS->Write as result of Move

20070315 Merged bcb DARWIN stuff; Set TCPIPTimeOut for PDU that moves images

20070316 Added (color) ToBMP for convert_to_bmp; added level and window to ToGif and ToBMP

20070407 Use ~p1..9A..Z to override C-MOVE port (adds 1..36 to port number)

20070902 WriteGif now cpp

20071027 Added quotes around filenames passed to dcmdjpeg and dcmcjpeg: avoids errors with spaces in paths

20071101 Enabled JPEG compression and decompression of non-16 bit data

20071104 Added DecompressNon16BitsJpeg (default TRUE)

20071105 Restore image when dcmc/djpeg failed to start

20080103 Added mode 'uj': leave jpeg as is, otherwise uncompress
Added DecompressImage to recompress()

20080820 Solaris fix (do not use variable name SS)

20081116 Adapted for very large objects (use unsigned int for length)

20081201 Added JPG converter

20081203 Enabled internal decompressor for color, added frame extraction for jpg etc; Add frame extraction control = 0x09999,0x0201

20090209 Added QueryMoveScript callback; removed unused variable

20090325 Added RetrieveResult converter, impact on performance to be tested

20090412 Changed jpeg encode_image mode to correct BGR RGB mixup

20090513 Added modes 's0' to 's9' call script CompressionConverter0 to 9

20090518 Added modes ka, kb and kc to downsize to 64, 32 and 16

20090921 Added ExtractFrames

20090927 Stop jpg compression for PR SR and RT

20091231 Added HAVE_LIBJPEG (version 6c!) and HAVE_LIBJASPER for external library support (beta for now)
Changed char* to const char* and cast time_t as int for gcc4.2 warnings

20100111	Fixed downsize for 32 bits pixel; bcb const; cast of Time() to (int)
20100112	Merged
20100113	A few more errors in downsize for 32 bits
20100116	Fixed DecompressImage compile error if HAVE_LIBJPEG is defined
20100118	Added one missing else in CompressJPEG2K
20100119	The else was not really missing; added some const
20100120	Catch jpeg2000 configuration if HAVE_LIBJASPER not defined
20100121	Blocked out old jpeg decompressor if HAVE_LIBJPEG, removed color checks as both decompressors do color now Made lossy to lossless switch a warning not error in CompressImage; Set mode to 'l' instead of 1 after switch
20100122	Note: the HAVE_LIBJPEG code supresses RLE support
20100122	Added experimental C-CANCEL support for all C-MOVES
20100123	Added counters for jpeg2000
20100124	Added GetNumberOfFrames
20100125	GetPrivateProfileString -> MyGetPrivateProfileString
20100126	Include gpps.hpp
20100207	Fixed ExtractFrame(s) for frames packaged into pixel data without sequences
20100209	Run RetrieveResult also for non-nki clients
20100210	Fix memory leak
20100214	Merged
20100223	Combined DecompressImage and DecompressImageFile Add FUJI_FIX for evil LittleEndianImplicit jpeg compressed images.
20100224	Added libopenjpeg to use in place of Jasper, fixed more warnings (-Wall)
20100227	After 1.4.16alpha3: crash when jpeg compressing RT object
20100330	Frames to LIBJPEG, JASPER, and OPENJPEG
20100619	Many fixes for leaks, allow both Jasper and openJPEG as a dicom.ini choice (not finished).
20100703	Merged; snprintf to _snprintf
20100706	Merged, replaced _snprintf with memcpy.
20100721	Merged back; fixed a few ascii to int conversions with atoi
20100723	Merged bcb fixes in new version of ToJPG
20100728	Installed RLE support with HAVE_LIBJPEG, used new deivr Get's and ChangeVR's
20100815	Merged; do not use GETUINT8 for US type; added space =TRUE to setting PixelSpacing Simplified DecompressRLE
20100815	Fixed bug 30 in t=2127: YBR_FULL_422 images not passed to demcjpg
20100823	bcb moved that check to the right place
20100824	Pass local VariableVRs to script engine
20100826	bcb fixed lossy jpeg2000 for signed data
20100901	bcb Added fix for PlanarConfiguration=1 data
20100905	Removed file functions: now RecompressFile uses Recompress
20100905	Merged; Added UseBuiltInJPEG flag
20100914	Fixed RLE decode for frames, added Deplane, DecodePalette for regular and segmented palettes

20100918	Attempted merged (very difficult as fixes were made in older version)
20100920	Fixed warnings
20100922	Merged
20100928	Fixed ExtractFrame(s) for NKI compressed data
20101003	Added 9999,0900 script
20101004	Use updated gif codec; started on animated gif
20101009	Protected gif coder for out of memory
20101018	Swap BGR to RGB for ToJPG
20100920	Fixed j2k decompressing of j2k file instead of just the stream (used by Accusoft Pegasus).
20100923	ToJPG can be used to create MPG files using external tools
20101116	Add UL, UB, UI, and UE compression to fix big endian receive, merged, warnings
20101121	Merged
20101220	Fixed color swap for jpg and bmp; fix deplane and depalette are called when needed
20101227	Fixed 'un' in bcb 'ui' etc code in recompress
20110106	Fix ToBMP for linux 64 bits (used long)
20110119	Use ExtendedPDU_Service to pass scripting state
20110320	Fix downsize & tojpg when #bytes in image is odd
	Only apply RescaleIntercept for CT in tojpg
20110326	Increased temp string in linux ExecHidden that caused decompress crash

NKIQRSOP.HPP (part of DGATE.EXE):

19990317	NKI-specific code
20001128	Fix: Crashes happened when more than one ServerChildThread' was active. m_pDCO was wrongly shared by all threads !!
20010429	Added GETADDO to allow optional read ahead withing calling program
20010502	Added extrabytes pointer to Read and RetrieveOn
20020415	ProcessDDO now returns status (to allow compression error check)
20020613	Added prototypes for DecompressNKI and CompressNKI
20021115	Added Generic retrieve classes
20030522	Added prototype of ComputeCRC
20030701	QualifyOn now also has compression parameter
20030702	added ExtendedPDU_Service
20030704	Changed ProcessDDO parameter to **DDO (for recompress)
20050118	replaced thread local storage under linux with variables in object
20050121	Changed filename to lower case
20050211	Removed need for thread local storage
20090209	Added QueryMoveScript callback
20091231	Added HAVE_LIBJPEG (version 6c!) and HAVE_JASPER for external library support (beta for now)
	Changed char* to const char* and cast time_t as int for gcc4.2 warnings
20100111	Merged
20100703	Merged some bcb OpenJPG changes
20100706	Added support for J2K and Jasper
20100721	Merged

20110118 Derived ExtendedPDU_Service from CheckedPDU_Service
 20110118 Added lua_State to ExtendedPDU_Service
 20110119 Added *VariableVRs and ThreadNum to
 ExtendedPDU_Service
 20110122 Added ExtendedPDU_Service destructor

ODBCI.CPP (part of DGATE.EXE):

19980415 Blocked one error message
 19990628: experimental locking call
 19990628 temp defined DB_DEBUG and introduced experimental
 statement options
 19990707: Undefined DB_DEBUG and removed statement options
 19990827: Added CreateIndex
 20000629 Logging of trouble now starts with '***' */
 20001105 Run-time load ODBC routines for BC and WC
 20001128: Added InitializeCriticalSection in 'Open' and
 DeleteCriticalSection in 'Close'
 20010829: Enabled DB_DEBUG to show db errors in verbose mode
 20020412 Use SQLExecDirectWithRetry to retry in case of any error
 (deadlock=1205). Added CreateUniqueIndex function, and
 GetLastError. Ignore error from unique index to make multi-
 user safe.
 20020413: NOTE: A SINGLE DB MAY NOT BE USED BY MULTIPLE
 THREADS.
 20020417: Extended deadlock wait time to about 1 min for safety
 20021014: Added built-in dbf support for stable ODBC less-operation
 Note: requires fully de-normalized database; notes: in dbsql,
 first bind row not set; in vrtosql; debug query print with %s in
 search crashes
 20021016: Added about 1 MB of in-memory hashed index in patient ID for
 all tables; speed OK to 10.000 objects; pack db on server start
 (for speed it is wise to keep DBF files on local harddisk)
 20021017: Made NeedPack a global for control from dgate; fix base_trunc;
 fix in Updaterecords; fix first num field in AddRecord
 20021020: Made querymodes an array, so that each query field can be key,
 key% or %key% ; Use querymode also when hashing the
 entries (part of db entry); Fix process_escape for " and \[and
 also use it before dbase_write
 20021028: Set open retries shorter for normal readonly access
 20021030: Match alternative patientID fieldname such that built-in DBF
 queries work
 20021113: Added <= comparison in dbase_query (used for date
 comparison)
 20030113: Do not use () in sort order when multiple columns specified
 20030128: Added alternative field names for series
 20030321 Also handle 'write lock' for M\$Access (NativeError 1205 for
 sql-server equals -1102 in M\$Access). Other dbf-drivers???
 20030819 Allow longer filenames
 20031104 Added \\ to process_escape
 20030425 Added DoubleBackSlashToDB control
 20030520 Speedup by fix dbaseIII hashing, setting hashfield was wrong
 20030523 Implemented in-memory index to speed dbase queries; added
 IndexDBF and PackDBF settings

20040528 Fix built-in DbaseIII for > 2 GB; index fixed at 10.000.000 records; Fix problem when spaces in patient ID in index generation

20040528 Added ExtractRecords (dbase III only); fix update/delete for record 0

20040530 IndexDBF gives MB extra to allocate; skip dbf files starting with X; convert normal queries in SeriesInstanceUID or StudyInstanceUID to indexed ones

20040601 Added LongQueryDBF flag: report if query took longer than LongQueryDBF ms (default 1000); Min 20 MB index during regen; allow pack on command (NeedPack=TRUE; DB.Open) Increase query buffer size to 16384 (often reread), pack buffer to 65536

20040605 Optimize speed for huge files, extract max 100000 records; made lockstart __int64; Do not lock for read in extract (is very slow for huge files); Record lock no longer on physical bytes but on rec#; attempt to speed memory index; NeedPack==3 -> force pack

20040606 Fix lock whole file; fix index patient ID database; fix in dbcreate for test

20040609 Implemented fast in-memory locking and enabled for all shared files

20040610 on dbase_create set indexcount to 0; dbase_trunc truncs indexcount

20040612 Put index creation in a thread; server runs (but does not find non yet indexed records) during indexing; implemented indexdbf command NeedPack=4

20040614 Added NeedPack=5: create non-threaded index (for archival options)

20040615 Added warning when index is full; keep indices when creating db

20040930 Added multiple value query; protect length of copy of where string

20041003 Reused very long SQLStatement string for wheres string
For lengths see dbsql.cpp; truncated debug printf's
set cols length to 4096; malloc tmp in query at 310000;
for length calculation see dbsql.cpp

20041013 Used MAXQUERYLENGTH

20041029 Fixes in multiple query (query string truncated + mode mix problem)

20041101 DB is read only while in-memory indexing of DbaseIII active

20050102 Small modifications to allow compile through total.cxx

20050107 Export NumIndexing to tell dgate when indexing is done

20050116 Adapted for use with LINUX (built-in dbase driver only)
Notes: 1) delay poorly implemented; 2) no support for >2 GB

20050118 Detach pthreads to avoid leaks

20050119 Documented rare crash in Fill_Memory_Index

20050124 Added support for huge files under Linux (not suitable for ext2 filesystem); Allow = without spaces in query

20050205 Added counters for db activity

20050206 Small fixes to counters; allow "&" instead of " and " for CGI use

20050905 Postgres code by Piotr Filipczuk checked by mvh, will not affect other operations, but not tested yet (postgres will not compile on my linux system)

20050907 Merged change by Hans-Peter Hellemann: lseek64 not always defined, should be mapped by unistd.h

20050909 Put check for = in query below >=: date range matching failed (Thanks Paolo Marcheschi)

20050911 Removed postgres print statements

20051112 Fixed BindField for postgres - now returns actual and not reserved length

20051217 Maximum locks in dbaseIII driver from 1000 to 100 to speed up qunlock (20% regen speed)

20051218 Respond to -b = NOTHREAD: run server as single process, useful for debugging and profiling

20051219 Fixed slow leak of queryfields for built-in dbase driver

20051230 Put index of DicomWorkList on AccessionNumber

20060219 Start on native mysql driver

20060220 Some more work: get correct length in bindfield; todo: user and password

20060226 Runtime load MYSQL in WIN32

20060313 Use username and password for mysql

20060402 Start index threads with a short delay to give GUI priority to test startup

20060607 Changed escape char processing for sql values (not queries) to match dbsql.cpp change

20060618 Fix for Borland C++ builder

20060628 Moved location of disabled query logging

20060630 Speeded dbaseIII AddRecord lots and linux string handling

20060701 Added and use mysql_fetch_lengths; protect query result

20060702 Tested native mysql driver: does not like fast open/close: clonedb crashed after 4000 or so (avoided now)

20061212 Found critical error in built-in dbase driver sql parser: Adding patient with ID containing certain character sequences will immediately overwrite the whole patient database. Queries with these sequences will return too many records (less dangerous). Issue not yet really fixed but greatly reduced its likelihood

20061213 Fixed issue by replacing spaces inside strings in sql statement with non-breaking spaces. Only match on keywords and field names which include spaces; replace with normal spaces in process_escape

20070123 clear NativeError, State and Msg on db.open (e.g. for dbf that does not use them but prints)

20070126 Added missing break in non breaking space field of process_escape: failed on multiple spaces in string

20070201 DebugLevel 4 prints all sql activity\

20070203 Fixed >2GB file support in linux: use O_LARGEFILE and lseek64() - sorry Hans-Peter Hellemann

20070206 Added CreateDatabase (created db and login)

20070207 Removed creation of login from CreateDatabase for mysql and postgres

20070210 Moved debug prints to SystemDebug; removed double log in QueryDistinct for dbaseIII

20070218	Added SQLite (win32 dll version): seems to run now; very slow when sync, very fast async
20070307	Made database type flag local, added Database(type) creator
20070308	Fixed DARWIN ifs, made DARWIN/MySQL auto start. (MySQL is included with DARWIN)
20070315	Merged; changed two nested ifs that had an else that would be wrong
20070406	Use reentrant rand_r and localtime_r when available
20070507	Native mysql does not correctly return empty fields (Steini)
20070702	Fixed above fix; test version for Stephan Winter
20070703	Removed test version code; display mysql connection errors on OperatorConsole
20070705	Added DT_DBASEIINOINDEX: disables indexing and packing; #define USESQLITE_DLL to load DLL; otherwise use statically linked sqlite3.c; Extended max number of dbf files to 20
20070709	Fixed crash when mysql not started: format error in the error print
20070711	Some fixes to compile with sqlite under linux
20070715	Removed debug print statements for sqlite
20071027	Maybe fix postgres error handling: reset ntuples
20071030	More fixes to postgres error handling: missing returns in PrintLastError
20071115	Fixed SQL database creation for SQL server 2005
20071118	Adapted for 64 bits (use SQLLEN for BindField)
20071124	Removed dangling WHERE and ORDER in SQL server interface when passing "" instead of NULL
20081116	Adapted for 64 bits linux (avoid use of "long", in dbase code and when reading SQL_C_ULONG)
20081117	Allow compile with both POSTGRES and USEMYSQL
20081119	added run-time loading of postgres for WIN32
20081120	Added UseEscapeStringConstants for dbase (is needed only in one place because in other place the E in E'\\...' is ignored by the current code)
20081121	Implemented CreateDatabase for Postgres; also try load libmysql64.dll (for easy 64 bit deployment)
20081124	Added DT_NULL (black hole)
20081125	Fix critical section in DT_NULL
20081126	Todo: all realloc of dbase index when full; don't use threads to fill
20090202	Fixed problem of database fail open in GUI after install: XADICOMPatients was never closed, as connected was not set when there were no databases
20090203	Fix crash on SQLite error
20090429	Added libpq64.dll
20090616	Added OldUID as hashfield: indices UIDMods table; increased default index to 10 MB
20090620	Include file stuff
20091230	Merged bcb change
20091231	Changed char* to const char* and added blank[]="" for gcc4.2 warnings
20100111	Merged
20100123	Allow host:port syntax for hostname in MySQL and PostGres
20100125	GetPrivateProfileString -> MyGetPrivateProfileString

20100309	Cast k to unsigned int, commented out unused variables (gcc4.2 Warnings)
20100309	Added returnn FALSE to the end of CreateDatabase
20100309	Added double parentheses (gcc4.2 Warnings)
20100619	Fix gcc4 warnings, improve speed and added #ifndefs and UNUSED_ARGUMENT.
20100703	Merged; but fixed random seed change (initialize static)
20100717	Merged
20100815	Fixed linux compile for SQLite
20100816	removed (a) from #ifndef EnterCriticalSection(a) etc
20101121	use strncpy to avoid access errors in Sqlite where fields may exceed defined length
20101121	Disallow SQLite with DoubleBackSlashToDB
20101129	Filter [x] like sequences in dbaseIII driver process_escape
20110102	retry connect to postgres once; more postgres error handling
20110104	Undone [x] change of 20101129

ODBCI.HPP (part of DGATE.EXE)

19990827	Added CreateIndex
20001128	Added CRITICAL_SECTION member of 'Database'
20020415	Added error info, GetNativeError, SQLExecDirectWithRetry and CreateUniqueIndex
20021020	Added built-in dbf support (for stable operation without ODBC) Note: requires de-normalized database
20030120	Ignore Micro\$oft complaint C4200
20030321	Enlarged buffer for SqlErrorMessages (Database->Msg)
20030819	Allow longer filenames
20040528	Added ExtractRecords (dbase III only)
20040530	Added InternalQuery (for converting normal to indexed queries)
20040605	Made lockstart __int64
20040612	Made lockstart in again - now pure record#; added TotalRec, MBExtra
20041003	Statement length set to 310 kb (see dbsql.cpp for explanation)
20041013	Malloc SQLStatement instead
20050109	Adapted for linux compile
20050905	Postgres code by Piotr Filipczuk checked by mvh, will not affect other operations
20060219	Start on native mysql driver
20060220	Include all required mysql defines here
20060226	Runtime load MYSQL in WIN32
20060628	Use mysql.h proper (for datastructures) even when runtime loading (sorry for the hard include path)
20060701	Added mysql_fetch_lengths
20070206	Added CreateDatabase
20070218	Added SQLite
20070307	Made database type local, added Database(type) creator
20070705	Added DT_DBASEIIINOINDEX: disables indexing and packing; Extended max number of dbf files to 20
20071115	Static mysql use <mysql.h>
20071118	Adapted for 64 bits (use SQLLEN for BindField)
20080107	Removed double declaration of SQLLEN
20080821	Fix for SUNWsprow compiler

20081116	Adapted for 64 bits linux (avoid use of "long")
20081117	Allow compile with both POSTGRES and USEMYSQL
20081119	added run-time loading of postgres for WIN32
20081120	Finished it
20081124	Added DT_NULL
20090114	Increased MAXFIELDS to 100 for clone_xvi
20091231	Changed char* to const char* for gcc4.2 warnings
20100111	Merged
20100619	Added no-copy to the Database class.
20100717	Merged

AMAP.CPP (part of DGATE.EXE):

19990827:	added (limited) wildcard mapping in GetACRNema: AE, port or IP may end on '*'
20001105:	replaced m-alloc by new
20011109:	Made AE mapping case insensitive
20020524:	In function 'GetACRNema', first try case SENSITIVE then INSENSITIVE
20030701:	Added compression column (with this code is this column is not optional!)
20030703:	KNOWN BUG: number of columns may not be 3 (crashes)
20030709:	Solved above problem; rewritten parsing of Acnema.map
20030709:	known bug: trailing spaces lead skip of next line
20030711:	Fixed trailing spaces in acnema.map
20031118:	Fixed leak InitACRNemaAddressArray

XVGIFWR.CPP (part of DGATE.EXE):

20050129	Adapted for use in the Conquest DICOM server (see //)
20050130	Made it ansi C
20050826	Made it reentrant to avoid gif file corruption in server
20051219	Fixed leak
20070902	Made cpp
20080905	Removed malloc.h for DARWIN
20090620	Include file stuff
20091231	Changed char* to const char* for gcc4.2 warnings
20100111	Merged
20100309	Commented out unused variables (gcc4.2 Warnings)
20100619	Added UNUSED_ARGUMENT.
20100717	Merged
20101004	Taken lsp's code of avs_animatedgif into use
20101116	Warnings fixed
20101120	Merged
20101128	Fixed double fclose

JPEG_ENCODER.CPP (inactive part of DGATE.EXE):

20090215	Started on update history
20090412	Added BGRto444(14): accepts RGB instead instead of BGR (!)
20100224	Fix warning redefine INT32 with HAVE_LIBJPEG
20100309	Fixed Operation on 'x' may be undefined warnings (gcc4.2)
20100703	Merged except typedef INT32
20100706	Accepted INT32 change

20100717 Merged

WINTYPES.HPP (part of DGATE.EXE, Linux only):

20050111 added SWORD and UWORD

NPIPE.CPP (inactive part of DGATE.EXE, Linux only):

20050111 Blocked out sys/conf.h; compiles with linux but gives linker warning: fattach not implemented and will always fail
20051217 Return (unsigned) 0xFFFFFFFF instead of signed -1
20070308 Change things for BSD and DARWIN.
20070316 Some more fixes
20091229 Completely disabled call to fattach, ioctl, and isastream as per <http://www.image-systems.biz/forum/viewtopic.php?f=33&t=1666>
20100111 Some layout
20100619 Fix gcc4 warnings.
20100717 Merged

NPIPE.HPP (part of DGATE.EXE, Linux only):

20070308 Added changes for BSD and DARWIN
20100619 Added no-copy to the NamedPipe class.
20100717 Merged

GPPS.CPP (part of DGATE.EXE):

20040111 Fixes: defaults now work, allow DOS type files, case insensitive
20070330 mvh Change by Mark Pearson: also allow tabs instead of spaces around =
20090620 Include file stuff
20091231 Changed char* to const char* and added blank[]="" for gcc4.2 warnings
20100111 Merged
20100124 Fix: blank item (e.g., password =) returned default not empty
20100124 Added cache for ini items: accelerates server with cached disk data 7x!!!!
20100124 ini file is tested for newer version every second
20100309 Added double parentheses (gcc4.2 Warnings)
20100703 Merged
20100619 Added no-copy to the NamedPipe class.
20100717 Merged
20110320 Removal of trailing blanks in value also removed one character after blank
20110331 Added FlushPrivateProfileStringCache

GPPS.HPP (part of DGATE.EXE):

20091231 Changed char* to const char* for gcc4.2 warnings
20100111 Merged
20100124 Renamed to MyGetPrivateProfileString used for windows and linux

20110331 Added FlushPrivateProfileStringCache

ERRCODES.H (part of dgate.exe):

20041221 Moved errorcode to here

20050901 Renamed errorcodes.h to errcodes.h

TOTAL.CPP (part of dgate.exe for most builds)

20020822 This file compiles to TOTAL.EXE = DGATE.EXE with, e.g., BC55

20050102 Added NOINTJPEG flag: jpeg8/12/16 sources cannot be compiled at once

20050107 Changed order of compile

20050111 Added npipe.cpp and gpps.cpp for LINUX

20050118 Documented how to compile under linux and with visual c++

20050121 Changed filename to lower case

20050130 Added xvgifwr.c

20070315 bcb added DARWIN stuff

20070330 bcb added DARWIN compile example

20080820 Solaris compile example

20080910 fixed DARWIN compile example for build version of mysql

20081016 Fix link to xvgifwr.cpp; 64 bit example code

20090209 Added jpeg_encoder.cpp

20090411 Added second SUN example

20100120 removed DARWIN (PPC, 10.4, now old) compile example. JPEG lib is internal, so undef NOINTJPEG if used.

20100123 Include stddef here for windows to avoid jasper.h compile problem

20100124 Also use gpps for windows.

20100224 Added defines for OpenJPEG

20100706 Allow both libraries at once

20100721 Merged

20100815 bcb added two defines; Merged

KILLER.CPP (main of KILLER.EXE, which is not part of this release of the Conquest DICOM server):

19980619: Added some doc on command line options and modes
Fixed -L option (moved db.close)

19980620: Made code suitable for external calling; copied some of it to device.cpp. Killer.exe is not needed anymore with NKI Dgate.exe except as sleeper task because the alert task has been integrated into Dgate.exe

20030702: Adapted because of compression externals required by device.cpp

20030706: Attach VRType to PDU's for implicit little endian support

20050129: Adapted dummy for DumpVR

20071118: Adapted for 64 bits (use SQLLEN for BindField)

AAAC.CXX (part of DICOM core library):

19980327: Removed evaluation of Count without initialization in ReadDynamic

20001106: Use memcpy instead of ByteCopy
20030122: Fixed initialization of AAssociateAC
20050108: Fixed for linux compile
20080203: Added experimental ConfigPadAEWithZeros
20080313: Removed some warnings
20100619: Fix gcc4 warnings and improve speed
20100717: Merged

AARQ.CXX (part of DICOM core library):

20001106: Use memcpy instead of ByteCopy
20001128: Removed two warnings
20030122: Fixed initialization of AassociateRQ
20060618: Added definition of min()
20080203: Added experimental ConfigPadAEWithZeros
20080313: Removed some warnings
20100619: Fix gcc4 warnings and improve speed.
20100717: Merged

AARQ.HPP (part of DICOM core library):

20001106: Use memcpy instead of ByteCopy
20091231: Changed char* to const char* for gcc4.2 warnings
20100111: Merged
20100619: Fix gcc4 warnings, improve speed and added
UNUSED_ARGUMENT.
20100717: Merged; Q1 OK
20100728: Added PresentationContext(const PresentationContext&)
20100815: Merged

AARJ.CXX (part of DICOM core library):

20100619: Fix gcc4 warnings and improve speed.
20100717: Merged

CCTYPES.H (part of DICOM core library):

20001107: Fixed to allow compile with BC
20010705: Fixed to allow compile with MSVC 6 (see note below)
20040111: Fix for __GNUC__
20040117: Fix for linux
20070308: Added for DARWIN apple endian defs
20070902: Changed INT32 and UINT32 from long to int for ms7/ms8
compile
20100706: Made INT32 match libjpeg if HAVE_LIBJPEG.
20100717: Merged; fixed for WIN32 64 bits

ENDIAN.CPD (part of DICOM core library):

20001106: removed unneeded (and sometimes uncompileable)
XE_UINTX::XE_UINTX (int x)

ENDIAN.HPD (part of DICOM core library):

20001106: removed unneeded (and sometimes uncomparable)
XE_UINTX::XE_UINTX (int x)

ENDIAN.CXX (part of DICOM core library):

20070330 Merged bcb stuff
20070726 Removed some duplicates from endian.hpp

ENDIAN.HPP (part of DICOM core library):

20090215 Started an update history

DICOM.HPP (part of DICOM core library):

20071102: Define SetDicomErrorHandler etc
20080203: Added experimental ConfigPadAEWithZeros
20090211: Added DCM_ERROR_PARSE
20090802: Added DCM_ERROR_DEBUG
20091231: Changed char* to const char* for gcc4.2 warnings
20100111: Merged
20110231: DicomError returns FALSE when handler not installed - to
allow some errors to pass

RTC.CXX (part of DICOM core library):

19980330 Added an RTC memberfunction for loading the dicom
library from memory (e.g. a resource).
19980331 Added function for writing binary RTCs (function was
originally implemented in 'makebrtc.cxx')
19980414 Added RTC::GetGroupElement memberfunction, to retrieve
the
Group and Element numbers using a library-keyword.
20040930 Also return TypeCode in RTC.GetGroupElement()
20071128 Do not remove existing TypeCode if new one is unknown
20090620 Always include stdlib.h
20091028 pack(4) for __Binary_RTC_Header
20091029 More adjustments for 64-bits (reading/writing binary RTC)
20100309 Changed int to unsigned int, commented out unused variables
(gcc4.2 Warnings)
20100717 Merged

RTC.HPP (part of DICOM core library):

19980331 Added prototype for writing binary RTCs
20091029 Adjustments for 64-bits
20100619 Added no-copy to the RTC class.
20100717 Merged

TRNSYN.CXX (part of DICOM core library):

19980409: Fixed possible transition between presence and absence
of TypeCodes in 'Dynamic_ParseRawVRIntoDCM'
19990904: Treat type UN same as OB,OW and SQ when reading Explicit
files.

20001106: Replaced == operator by strcmp
 20010905: Several fixes: UCDCM should NOW be able to read ExplicitBigEndian files
 20011220: Fixes in Sequence and Item delimiters
 20030122 Fixed small leak in Dynamic_ParseRawVRIntoDCM
 20030205 Support new datatype 'UT' (unlimited text). Thanx to William Peterson
 20030424 Changed prototype of ParseDCMIntoRawVR. Fixed huge bug in sending of ExplicitLittleEndian sequences
 20030627 Fix for MSC 4.2
 20030725 Keep GroupLength for group 0x00 (Radworks 6.0 was choking in this)
 20040406 Make TypeCode 'UN' if not known, and conversion to Explicit is wanted
 20050112 Fixed huge bug in Explicit_ParseDCMIntoRawVR, causing data-loss. Thanx to Goran Peros.
 20051217 Use new VR::ReAlloc method to replace data in VR
 20051218 Use memcpy to enter UID (extra 0 will be available: end of string)
 20051219 Use ReAlloc instead of new BYTE[] to fill VR data
 20051229 Moved Push in ParseRawVRIntoDCM so that not done when VR exists
 20070308 Added big endian stuff
 20071031 Removed some commented out code
 20071108 In Dynamic_ParseDCMIntoRawVR(), let group 0x08 behave the same as all other groups greater than 0x02.
 20071118 NOTE: previous change is not in release 1.4.13
 20071128 Add TypeCodes if needed using AttachedRTC->RunTimeClass
 20080913 Fix in bigendian input: interpret group 0 as littleendian
 20090211 Added many checks on corrupt dicom data during parsing; removed blocked out code
 20090212 Fixed a problem in the check code if some (but not all) group lengths are missing
 20090215 Improved formatting of lenght error message
 20090620 Added <stdlib.h>
 20090824 Resume parsing after invalid element length if length was not too big
 20100102 Merged bcb changes for bigendian
 20100111 bcb moved setting 0002,0010 to work for all transfers in ParseRawVRIntoDCM: fix for osiris; Added JASPER stuff; const
 20100118 Always allow jp2k
 20100119 Put HAVE_LIBJASPER back in (unfortunately that creates a dependency between dicomlib and dgate)
 20100119 Changed Dynamic_ParseRawVRIntoDCM and ParseImagePixelDataRawToDCM to read NovaRad files and remove the errors.
 20100121 Merged but put into NOVARAD_FIX define; not bcb alsp prints which parser reports errors
 20100125 Linux warnings.
 20100224 Changed HAVE_LIBJASPER to HAVE_J2K for OpenJPEG and fixed more warnings (-Wall)
 20100309 Added double parentheses (gcc4.2 Warnings)

20100309	Changed int to UINT, commented out unused variables (gcc4.2 Warnings)
20100419	Fixed Dynamic_ParseRawVRIntoDCM big endian
20100619	Made VRBuffer local IVRBuffer.
20100707	Merged
20100718	Looked at and rejected the date change: you can use importconverters for this job
20100722	iorgmode change breaks the reader for me!
20100724	Put back date change with DATE_FIX inclosing it.
20101001	Allow CurrentGroupLength==0 as warning
20101018	Fixed big endian in Dynamic_ParseRawVRIntoDCM.
20101020	Merged.
20101116	Added to/from big endian and used it to fixed big endian receive.
20101120	Merged
20110231	allow some parsing errors when DicomError handler not installed

DEIVR.CXX (part of DICOM core library):

19990318	Added 'DICOMObject::DeleteVR'
20001106	Used delete [] operator for vr->Data
20010426	Fixed small leaks in 'DICOMObject::DeleteVR'
20010730	Added 'DICOMObject::ReplaceVR'
20010802	Added 'VR::GetSpecialValueCodeOfVR'
20011121	Added VR_SPECIAL_RGB_TO_MONO. Used by a conversion-tool to change ultrasound Kretz RGB images to monochrome that VariSeed (brachytherapy) understands.
20011121	Added VR_SPECIAL_RGB_TO_MONO_PLUS. Same as above, but tries to preserve colored drawings.
20051217	Added 'VR::ReAlloc'
20051217	Use malloc/free instead of new char[] and delete []
20051229	Removed unnecessary reset() in DeleteVR
20071031	Temp printf memory error messages
20071102	Send allocation errors to DicomError
20080313	Removed some warnings
20080910	Added VR::VR(UINT16 g, UINT16 e, UINT32 l, LE_UINT16*d) & VR::VR(UINT16 g, UINT16 e, UINT32 l, LE_UINT16 *d, BOOL mFlag) for big endian.
20081016	Fix for WC: new of 0 gives NULL (check Length in error message)
20090620	Added <stdlib.h>
20091230	Merged bcb bigendian change
20100110	And const change
20100309	Added double parentheses (gcc4.2 Warnings)
20100619	Fix gcc4 warnings and improve speed and added questions.
20100717	Merged except keep default ReleaseMemory to TRUE and check data when adjustin length
20100728	Removed unused AutoMakeDO, added Get's to VR and DICOMObject. Added ChangeVR's. Fixed LastVR & LastVRG?
20100815	Merged; fix VR constructors for failed malloc; added max parameter to GetCString function; Moved defaults from implementation to header files

20100826	bcv moved another default, size_t
20100901	bcv changed alloc of US to 2
20110231	DicomError returns FALSE when handler not installed - to allow some errors to pass

DEIVR.HPP (part of DICOM core library):

19990318	Added 'DICOMObject::DeleteVR'
20010730	Added 'DICOMObject::ReplaceVR'
20010802	Added 'VR::GetSpecialValueCodeOfVR'
20051217	Added 'VR::ReAlloc'
20080910	Added VR::VR(UINT16 g, UINT16 e, UINT32 l, LE_UINT16 *d) & VR::VR(UINT16 g, UINT16 e, UINT32 l, LE_UINT16 *d, BOOL mFlag) for big endian.
20100619	Fix gcc4 warnings and improve speed, and prevented coping of classes with pointers.
20100717	Merged
20100728	Removed unused AutoMakeDO, added Get's to VR and DICOMObject. Added ChangeVR's.
20100815	Merged; added max to GetCString; Moved defaults from implementation to header files
20100823	bcv moved default parameter here

SOCKET.CXX (part of DICOM core library):

20010426	Added 'setsockopt' in 'Listen' and 'Open' Call 'shutdown' before closing a socket
20010720	Changed 'print' to 'fprintf(stderr,...)'
20050116	Adapted for LINUX compile
20050119	Blocked out SOCKET END ERROR message
20070406	Use reentrant gethostbyname_r when available
20070415	Small fix there for windows
20090620	Added unistd.h for UNIX
20090824	Debug output of adress connecting to socket
20091231	Changed char* to const char* for gcc4.2 warnings
20100111	Merged
20100122	Gethostbyname: fixed bug reported by Arash 2station would not be found
20100122	Added Poll() method: returns TRUE if data available
20100125	Linux warning
20100309	Commented out or init'd to fix unused variable warning (gcc4.2 Warnings)
20100619	Fix gcc4 warnings, improve speed and made TimeOut local.
20100707	Merged

SOCKET.HPP (part of DICOM core library):

20091231	Changed char* to const char* for gcc4.2 warnings
20100111	Merged
20100122	Added Poll function
20100619	Added no-copy to the Socket class.
20100707	Merged

SAFEMEM.CXX (inactive part of DICOM core library):

20010720 Changed 'printf' to 'fprintf(stderr,...)'

DIMSEN.CXX (part of DICOM core library):

19990415 Removed static on TranslateUIDToVR
20020822 Use instead of DIMSEN.CPP
20071118 Removed faulty overloaded 'NGetRQ::Write'
20080905 Removed (void*) cast for deivr change
20100309 Added double parentheses (gcc4.2 Warnings)
20100619 Added UNUSED(x)
20100717 Merged

DIMSEN.HPP (part of DICOM core library)

20020822 Cleanup layout
20100619 Added virtual destructors
20100717 Merged

DIMSEC.CXX (part of DICOM core library):

20030219 Removed MessageID and Priority from the 'write' functions of
CStoreRSP, CFindRSP and CMoveRSP
20030606 Added parameter 'QueryRetrieveLevel' to CFindRSP::Write
20030811 Do not send DDO in CMoveRSP::Write (DicomEye and OFFIS
do not like that).
20050129 Added Write with extra vr to CEchoRSP
20051222 Do not add query level if already exists
20070308 Made changes for big endian
20070314 Allow send of 0000,1030 (MoveOriginatorAE) and 0000,1031
(MoveOriginatorMessageID) in CStoreRQ
20070330 Merged bcb's Darwin and Endian stuff
20080203 Added experimental ConfigPadAEWithZeros
20080910 Removed big endian changes from here and moved to deivr
20100309 Added double parentheses (gcc4.2 Warnings)
20100619 Added UNUSED_ARGUMENT.
20100717 Merged

DIMSEC.HPP (part of DICOM core library):

20030606 Added parameter 'QueryRetrieveLevel' to CFindRSP::Write
20050129 Added Write with extra vr to CEchoRSP
20070314 Allow send of 0000,1030 (MoveOriginatorAE) and 0000,1031
(MoveOriginatorMessageID) in CStoreRQ
20100619 Added virtual destructors (gcc4.0 Warnings)
20100717 Merged

UNIQ.CXX (part of DICOM core library)

19990511 Fix: All routines wrongly used '%' instead of '&'

MAKEBRTC.CXX (part of DICOM core library)

19990415 Note: requires changelog

PDU.CXX (part of DICOM core library):

19980322 Blocked one remaining printf statement "comparing ..."
19990202 Added PDU memberfunction 'IsAbstractSyntaxAccepted'
19990415 Merged both versions
20000322 Fix in search for PresentationContext when identical
AbstractSyntaxes occur (e.g. GECT-scanner). Do not break out
of the inner loop in PDU_Service::InterrogateAAssociateAC.
20000322 Fix of PresentationContextID in PDV: in routines
PDU_Service::Write(DICOMObject*, UID*), set
PresentationContextID to TempByte, not to TempByte1.
20000911 Removed two warnings
20020316 Added GetLocalAddress and GetRemoteAddress
20030306 Fix: Huge bug in 'InterrogateAAssociateRQ'
20030424 Blocked out obsolete PDU.Write(&DDO)
Some work on offering other TransferSyntaxes than
ImplicitLittleEndian when running in SCU mode
(Note: when running in SCP mode, 'dgatesop.lst' is used)
Fix: CommandObjects are always ImplicitLittleEndian
20030627 Fix for MSC4.2; re-enabled obsolete write calls for test.cxx
20030710 Adjusted test.cxx, and disabled obsolete calls again
20080816 ValidPresContexts is now a global: how many incoming
connection can handle
20090802 Added DCM_ERROR_DEBUG statements to debug failed
connect
20100303 Commented out unused variables (gcc4.2 Warnings)
20100619 Fix gcc4 warnings, improve speed.
20100717 Merged
20100726 bcb documented the mess in function AddAbstractSyntaxAlias
20100728 May have fixed AddAbstractSyntaxAlias, but I have no way to
test.
20100815 Merged
20100914 Cleaned up AddAbstractSyntaxAlias problems (double release)
20100918 Merged

PDU.HPP (part of DICOM core library):

19990202 Added PDU memberfunction 'IsAbstractSyntaxAccepted'
20020316 Added GetLocalAddress and GetRemoteAddress
20030424 Changed prototype of ParseDCMIntoRawVR
20030627 Fix for MSC4.2
20030905 Removed spurious ;
20080816 ValidPresContexts is now public: how many incoming
connection can handle
20080819 CanYouHandleTransferSyntax is now public; use BOOL not
bool
20091231 Changed char* to const char* for gcc4.2 warnings
20100111 Merged
20100619 Fix gcc4 warnings and improve speed,
and prevented copying of the PDU_Service class with pointers.
20100723 Merged
20101120 bcb added parameter
20110320 Rollback of dicom decoder (parameter) to 1.4.16beta4 version

20110326 Put it back again

FILEPDU.CPP (part of DICOM core library):

20000420 Fix in MakeChapter10: type of AffectedSOPClassUID and RequestedSOPClassUID
20030205 Fixed FileMetaInformationVersion for big-endian machines
20030424 Put the correct TransferSyntaxUID in chapter10 files
20041108 Fix strdup memory access problem (found using \$c drop file test mode)
20051229 Fixed LEAK of 68 bytes when 0002,0010 not defined
20060618 Added definition of _SH_DENYNO
20090211 Check result of Dynamic_ParseRawVRIntoDCM
20091231 Changed char* to const char* for gcc4.2 warnings
20100111 Merged
20100309 Added double parentheses (gcc4.2 Warnings)
20100309 Cast multichar as UINT16, commented out unused variables (gcc4.2 Warnings)
20100706 Fixed TransferSyntax in SaveDICOMDataObject(ACRNEMA_VR_DUMP) for testcompress.
20100726 Merged
20100823 Merged small comment fix

BUFFER.CXX (part of DICOM core library):

20001106: Use memcpy instead of ByteCopy
20071031: Temp printf allocation errors
20071102: Send allocation errors to DicomError
20081016: Default buffer size from 1024 to 10240
20100619: Fix gcc4 warnings and improve speed.
20100717: Merged

BUFFER.THH (part of DICOM core library):

20100309 Changed BufferSize, InSize and OutSize from INT to UINT
20100621 Added no-copy to the BufferSpace classes.
20100707 Merged

VERSION.H (part of DICOM core library):

200107017 Changed ImplementationClassUID to a valid UID: the Agfa dicom validation tool protested against the original 'UID'.
Changed ImplementationVersion from "0.1B/WIN32" to "1.01/WIN32"
20020416 Changed IMPLEMENTATION_VERSION_STRING to 1.3.11/WIN32
20021018 Changed IMPLEMENTATION_VERSION_STRING to 1.4.0/WIN32
20020208 changed IMPLEMENTATION_VERSION_STRING to 1.4.1/WIN32
20030424 changed IMPLEMENTATION_VERSION_STRING to 2.0.1/WIN32

20030709	changed IMPLEMENTATION_VERSION_STRING to 1.4.2/ WIN32
20030815	changed IMPLEMENTATION_VERSION_STRING to 1.4.3/ WIN32
20030402	changed IMPLEMENTATION_VERSION_STRING to 1.4.4/ WIN32
20040530	changed IMPLEMENTATION_VERSION_STRING to 1.4.5/ WIN32
20040713	changed IMPLEMENTATION_VERSION_STRING to 1.4.6/ WIN32
20041101	changed IMPLEMENTATION_VERSION_STRING to 1.4.7/ WIN32
20050901	changed IMPLEMENTATION_VERSION_STRING to 1.4.9/ WIN32
20060103	changed IMPLEMENTATION_VERSION_STRING to 1.4.10/WIN32
20060311	changed IMPLEMENTATION_VERSION_STRING to 1.4.11/WIN32
20060708	changed IMPLEMENTATION_VERSION_STRING to 1.4.12/ WIN32
20070307	changed IMPLEMENTATION_VERSION_STRING to 1.4.13/ WIN32
20080831	changed IMPLEMENTATION_VERSION_STRING to 1.4.14/ WIN32
20100713	changed IMPLEMENTATION_VERSION_STRING to 1.4.16/ WIN32; added osx

FLPDU.CXX (part of DICOM core library):

20050130	Allocate 1 item longer lists to allow configuration from GUI
20050213	Fix to allow windows created file as input under linux
20060618	Added definition of _SH_DENYNO
20090802	Added DCM_ERROR_DEBUG statement to debug failed connect
20100309	Commented out unused variable (gcc4.2 Warnings)
20100619	Fix gcc4 warnings and improve speed
20100717	Merged
20110118	Added unchecked CheckedPDU_Service () using NULL default

FLPDU.HPP (part of DICOM core library):

20050129	Made tables public for config
20080819	Made CanYouHandleTransferSyntax public
20100619	Added no-copy to the CheckedPDU_Service class.
20100713	Merged
20110118	Added unchecked CheckedPDU_Service () using NULL default

VERIFY.CXX (part of DICOM core library):

20050129	Added ReadRequest and WriteResponse, allows extra response vr
----------	--

VERIFY.HPP (part of DICOM core library):

20050129 Added ReadRequest and WriteResponse

QRSOP.CXX (part of DICOM core library):

20020429 Fixed reporting of number of complete slices (now starts with 1); return cancel status if C_STORE failed
20030606 Added parameter 'QueryRetrieveLevel' to CFindRSP::Write
20050401 Added ModalityWorkListQuery
20051222 Added pADDO to Query::Write method - answer appended to pADDO if set
20080820 Solaris fix (do not use variable name SS)
20090205 Added QueryMoveScript callbacks
20100122 Added support for C-CANCEL: poll the socket for any data and cancel if so
20100309 Added double parentheses (gcc4.2 Warnings)
20100619 Made Port local (lPort).
20100707 Merged
20100816 Added QueryResultScript

QRSOP.HPP (part of DICOM core library):

20030606 Added member-variable 'QueryRetrieveLevel' to StandardQuery
20050401 Added ModalityWorkListQuery
20051222 Added pADDO to Query::Write method - answer appended to pADDO if set
20090205 Added QueryMoveScript
20100816 Added QueryResultScript

STORAGE.CXX (part of DICOM core library):

20020429 check status of C-STORE RSP and return FALSE if failed
20030922 Added PDU_Service to CheckObject call
20070307 Added more debug error messages to retrieve
20070314 Allow send of 0000,1030 (MoveOriginatorAE) and 0000,1031 (MoveOriginatorMessageID) in StandardStorage : Write
20070315 Blocked out printf statements
20100309 Commented out unused variable (gcc4.2 Warnings)
20100619 Added #ifndefs (gcc4.0 Warnings) and UNUSED_ARGUMENT
20100717 Merged

STORAGE.HPP (part of DICOM core library):

20030922 Added PDU_Service to CheckObject call
20050108 Fixed for linux compile
20070314 Allow send of 0000,1030 (MoveOriginatorAE) and 0000,1031 (MoveOriginatorMessageID) in StandardStorage : write

PRINTSOP.CXX (part of DICOM core library):

20020814 Added some comments and block-out mvh's debug code
20020816 Fix in 'AbstractPrintManagementMeta::Write'

CQSERVER.CPP (inactive part of conquest interface DLL):

1998xxxx Creation. This file contains DicomServer code. A 'DicomServer' is an application that receives images, and is called a server because it 'provides' disk-space!? All code is reached by the routine 'ReceiveThread' at the end of this file. The ReceiveThread is launched by the exported function 'DcmMoveHere' in CqDicom.cxx. Communication with the mainthread is done by using global variables (sorry).

19990318 Added support for MultiFrame modalities

19990407 Swapped priority of VRs 0020-0050 and 0020-1041; now the slicelocation is preferably deduced from 0020-1041.

19990915 FillSliceInfo now also indicates whether pixeldata are 'unsigned'.

19991028 Fix (in cases that went wrong anyhow): changed 'free' to 'DllFree'.

19991029 Added 'Signed_compute' to convert unsigned short pixeldata

19991122 nki_private_decompress now supports Mode=2

20000131 Fix: Replaced an instance of free by DllFree

20000321 FillSliceInfo now also retrieves 'BitsStored'

20000405 Added CheckRangeOfCT, to Calibrated CT-values to Hounsfield + 1024, and to cut the values below 0.

20000418 Fix: crash on (INVALID !!) Siemens demo-data on RSNA 1994(CR images of RXDpat1-1994). PixelRepresentation tag was empty!

20000530 Use different pixelspacing tag for RTIMAGE

20010123 Moved code to GetSliceCoord; added static keywords

20010410 Improved communication between MainThread and ServerThread

20010411 Do not 'sleep' the ServerThread, but let it terminate

20010426 Again improved communication between MainThread and ServerThread

20010508 Replaced nki_private_decompress by a faster version

20010717 Added 'CheckRTDoseScaling'

20010720 Made Dummy::printf do nothing

20010905 Added 'CheckModalityLUT'

20020227 Support US pixelspacing (hiding in SequenceOfUltrasoundRegions)

20020425 Added exported funtion 'ComputeCRC'

20020609 Pasted extended decompressor from NKIQRSOP.CXX

20020613 Moved MkTypedDicomObject to here

20020620 Guess SliceLocation in multi-frame images.

20021028 Support multiframe dose-grid in 'CheckRTDoseScaling', and slice-positions in 'GridFrameOffsetVector'
NOTE: Varian multiplies and Helax divides by the DoseGridScalingFactor !!!

20021113 Guess SliceLocation in multi-frame images (SliceThickness).

20021213 Adjusted field-coordinates of RTIMAGE

20030122 Support various Ultrasound pixelspacings

20030127 FillSliceInfo now also retrieves 'PhotometricInterpretation'

20030128 Support for 3D TimeSequences, resulting in 4D 4-space output scans

20030213 Fixed bug in change above

20030624 Workaround for Varian Vision 7.1 multiframe dosegrids

	Support for integer dosegrids and very small DoseGridScaling
20030904	Improved workaround for Helax DoseGridScaling
20041119	Added support for 4D object in slices
20051219	Use vr->ReAlloc
20060515	Also apply RescaleSlope and NEXT_VOXEL_DOUBLE to modality PT
20060704	Don't apply yet...
20060913	Work on Philips MR: get PixelSpacing and PatientPosition from deep inside sequences. Replaced GetFloatsFromKey call by GetFloatsFromVR calls
20070404	Heavily optimized InsertSlice: first transpose then insert biggest gain (10 s) Uncompress during read gains 2 s
20070506	Added optional prealloc data pointer to CreateOutputField
	If LargeScan (>256 MB) decompress later to save memory
20070522	In CheckRTDoseScaling() take PLATO negative dose into account
20070704	Fixed InsertSlice(): non-square slices went totally wrong
20070730	Dirty fix for CMS RTDOSE
20071127	Fixed bug in AddToOutputField (affects multiframe slice positions)
20080313	Removed some warnings
20080410	Fix: In CheckRangeOfCT(), unsigned CT pixels were ill-treated.
20080818	Added support for philips 3D US (200d, 3303)
20081014	Fixed dose reading: broken by above change
20090131	Exported tranpose routines for CqCreateDcm
20090602	Added sensible Time-coordinate (AcquisitionTime) for 4D slice-based images. Works for our PET and MR scanner...

CQDICOM.CXX (part of conquest interface DLL):

19980330	Creation
19980331	Added exported DcmMkBinaryLib
19980414	Restricted the HeaderDump to max 4096 data, and the number of bytes on one line to 256.
19980416	Added sequence support in HeaderDump
19980420	Fixed last written line in multiple line sequence dump. Added functions DcmFind, DcmMove and DcmMoveHere
19980526	Added DcmCreateBitmap
19980610	Changed BGR to RGB in DcmCreateBitmap. Added DcmMoveHeaderHere
19980629	Fix: DimX and DimY were reversed. Improved efficiency when loading one slice.
19980630	Fix: last change caused crashes
19980707	Removed some checks to allow non-standard Philips images. Added DcmSetProgressHook
19980708	Added DcmReplaceInHeader, which can be used for anonymization of dicom images.
19980709	Added C-CANCEL command object when a ProgressHook stops or aborts a DcmMove operation. Added support for NkiPrivateCompressed images in DcmMoveHere.
19980826	Added #pragma pack(1) in Field.h
19981105	Added DcmReadHeader
19981127	Added DcmGetLastXfms

19990108	Updated the dictionary for DICOM-RT support
19990114	Slices are sorted on coord. When duplicate coords exist, the last received slice eliminate previous ones. Included changed SOP-list by MvH
19990318	Added three xxxRetrieveNKI PresentationContextSOPs in the function DcmMoveHere
19990407	Swapped priority of VRs 0020-0050 and 0020-1041; now the slicelocation is preferably deduced from 0020-1041.
19990511	Added DcmSetLocal(char* pszLocalAE, char* pszLocalPort)
19990628	Fix: DcmCreateBitmap did not like MultiFrame dicom files
19990826	Added DcmSetFreeHook and DcmMoveFilesHere
19990906	Added DcmSetExtraParams to support (amongst others in the future) transmission of downsized Dicom images between NKI stations.
19990913	Private VRs of unknown type, will possibly be shown in the headerdump as character strings.
19990915	Fix in ParseLine of headerdump. Added support for unsigned pixeldata in DcmCreateBitmap.
19991019	Allow multiple slices with the same or without a coordinate, if this is the case for ALL slices (e.g. multiple scout-views in one series)
19991022	Changed the root of NKI-made UIDs to the official one supplied by Medical Connections(UK): 1.2.826.0.1.3680043.2.135.1066.xxxxxxxxxx
19991028	Converted unsigned short pixels to signed int pixels
19991122	Added DcmStoreFiles
20000126	Fix in WorldXfm (bug introduced on 19991019)
20000128	Fake slice-coords when a '3D volume' consists of e.g. multiple scout-views (Siemens MR produces such series). Prevents viewer-crash when loading a second volume into it.
20000321	Convert unsigned short to integer ONLY if all 16 bits of the pixeldata are used. Better contrast-stretch in DcmCreateBitmap.
20000327	Added C-ECHO command
20000405	Calibrated CT-values to Hounsfield + 1024. Cut-off below 0.
20000707	Added DcmSetCacheAddress
20000829	Added an ErrorMessage
20001128	Replaced 'Oh shit!' errormessage by 'Cannot find related scan'.
20010123	Added static keywords. Moved code to GetSliceCoord; this routine checks besides tags (0x0020,0x1041) and (0x0020,0x0050) also (0x0020, 0x0032). This last tag (ImagePositionPatient) had to be used for Philips MR scanners.
20010302	Fix: improved temporary filename
20010410	Improved communication between MainThread and ServerThread. Again improved temporary filename...
20010411	Kill ServerThread only if it is not responding
20010413	Fix: allways close the threadhandle
20010426	Again improved communication between MainThread and ServerThread
20010508	Adjusted call of nki_private_decompress to a faster version
20010619	Fix in GetFloatsFromKey (crashed on PatID 1234567890 NuclearMedicin testpatterns)
20010717	Adjusted DcmMoveHere for RTDose post-processing

20010802	DcmReplaceInHeader now also supports replacements inside sequences; Added export DcmVersion
20010808	Fix: SetStringVR appended a NULL byte to strings with uneven characters. This is correct for UIDs, all other strings should have a space appended.
20010824	Updated 'soplist.txt' resource to DICOM 1999 final text.
20010905	Fix: Headerdump was incomplete when a VR consists of more than one short or int; Start implementation of ModalityLUT
20011121	Replaced 'SetStringVR' by 'SetAnyTypeVR' (Used by DcmReplaceInHeader)
20011218	Added DcmWriteKretzFile: extracts a Kretz 3D file from a dicomfile
20020225	Fix for Helax-TMS: when determining XFMs and (0x20, 0x1041) and (0x20, 0x50) are both absent, use (0x20, 0x32)
20020403	Fixed bug when UIDs are 64 bytes long. Some code butifying.
20020411	Added DcmWriteVRToFile
20020502	Added cacheing properties
20020522	Replaced ad hoc field allocation code by 'AVSdata_alloc'
20020523	Do not give warning for slice-order (DCM_E_SLICEORDER)
20020609	Pass MaxCompressionMode that is set to 4
20020620	Adjustment to CalcXfm for multi-frame images.
20020814	Added two Print error-messages
20020930	Added MAXSLICES to optional extraparams
20021028	Exported GetFloatsFromKey. Try te get XFM's of multiframe objects right.
20021105	Get rid of scout-views when loading a volume (in 'SortSliceDDOs')
20021118	Fix in multiframe XFM containing 'GridFrameOffsetVector'
20030122	JPEG pixeldata support; Fixed memory leak
20030128	When SliceCoords are not provided, sort dicom-objects on imagenumber
20030130	Support for 3D private Kretz ultrasound
20030424	Changed all calls to obsolete PDU.Write(&DDO) to PDU.Write(&DDO, uid). Fix in handling of JPEG compressed images. Version to 2.0
20030519	Fix XFMs when ImagePositionPatient contradicts SliceLocation. Now the slicecoord in ImagePositionPatient is preferred (Marconi-fix).
20030523	Fix: above change frustrated coronal and sagittal MRI
20030716	Workaround for sloppy Adac NM multiframe reconstructions (@#\$\$%^&*)
20041116	Start on time series recognition, now return unsorted
20041117	'proper' time series (with exact N copies of each slice with given coordinate) are sorted on derived frame# (order of image#) then coords; but still returned as 3D. MATPEL and WRLDMATC know how to make 4D data out of such image data (CONQUEST does not yet). This option should read 4D CT (not tested) as well as 4D MRI/DTI (tested) as long as all frames are sent as one series.
20041119	Adjustment in CalcXfm
20041119	Added support for 4D sliced object, now sent out as 4D field
20041124	Merged fix ljz 20041119
20041201	Improved detection of 4D data, now allow fringes, e.g., if slices at certain locations occur NMMMMKL times, N slices at start

and K+L slices at end are removed. In this way, both Philips and GE DTI and perfusion data (without fringes, e.g., MMMMMM) and Siemens RCCT data is supported. Note 1: Siemens writes % of cycle in 0018,0022 as 'TP100PC5576', where 100 is the %. Not used for now, 4D data is just ordered based on slice number instead, with coordinates 0, 1, .. Note 2: some localizer series are now returned as 4D as well.

- 20041221 Fix in 4D reader for out-of order images; first sort on image#, later in index
- 20050301 Fix in dubious check on 'Proper4D'
- 20050308 One more check on 'Proper4D': at least half of the slices should be Proper4D; Fix in last check
- 20050418 Fix in WriteBmp when all pixelvalues are equal
- 20050804 Fix in check on 'Proper4D'
- 20050905 Added SOP 31 for query: modality worklist query
- 20060531 Moved FreeOutputField to field.cpp and field.h
- 20060701 Added warning on order of 4D data - MessageBox needs to be replaced
- 20060913 Replaced GetFloatsFromKey call by GetFloatsFromVR calls
- 20061020 Fixed bug in GetFloatsFromVR
- 20061030 SortSliceDDOs crashed on MR scans from Lucas hospital
- 20070112 Added exported function DcmReadJpeg
- 20070119 Using DcmSetExtraParams SORTMODE=1, slices will be sorted on ascending coordinate, SORTMODE=2 means descending. Fixed bug in SortSliceDDOs.
- 20070220 Fixed bug in SetAnyTypeVR; Added an errormessage
- 20070220 Attempt to fix frame reversal - debug code temp off
- 20070506 prealloc data pointer during receive to allocate large block first: allows loading larger scans
- 20070730 Dirty fix for CMS RTDOSE
- 20070902 DcmWriteVRToFile will now also write sequence item to file as raw VR dump; takes Index as extra par
- 20071127 Support for multiframe-objects where ImagePositionPatient is located in a DetectorInformationSequence
- 20080313 Fix: In SortSliceDDOs(), long ScanOptions could lead to crashes or other undesired behaviour; Removed some warnings
- 20080318 In CreateLastPropertiesField(), a few properties that resided deep in sequences for RTDOSE were added.
- 20080320 ...and put in a separate function
- 20080717 Support of Pinnacle DOSE in CalcXfm()
- 20080812 Support of Varian's Eclipse DOSE in CalcXfm()
- 20080908 Implemented &Philips to load embedded 4D ultrasound
- 20081007 Compute location of philips subvolumes by length/N
- 20081014 Also accept QueryIni-fields that contain Header/Key/Value instead of Key/Value
- 20090605 Fixed crash in DcmMoveHere() when GetDDOarray() returns OK, but no data were received
- 20090608 Fix: Due to different AcquisitionTime the warning 'Inconsistent Slices' was given too often
- 20090610 Added "Upi is NOT unique: DANGER !!" error-message
- 20091029 Adapted for 64-bits version
- 20100427 Added workaround for illegal (incomplete) query response from Kodak

20100811	Always return at least one record in DcmFind(), containing the query tags; Recoded the Kodak workaround. Now only matches to the query tags are returned, not any additional info that the server might provide. In parameter pszNKIcommand1 of DcmEcho() starts with '#', then the EchoResponse sent by the server in (0x9999,0x0401) will be dumped to file.
20100827	Fixed memory leak in ProcessVRs(), updated documentation
20101130	In DicomMoverHeaderHere, translate 'Unknown' tags that ARE present in the library back to (gggg,eeee)
20101208	Fix: Prealloc did not allocate enough in case SamplesPerPixel is unequal to one

CQDICOM.H (part of conquest interface DLL):

19980330	Creation
19980331	Added exported DcmMkBinaryLib
19980414	Restricted the HeaderDump to max 4096 data, and the number of bytes on one line to 256.
19980416	Added sequence support in HeaderDump
19980420	Fixed last written line in multiple line sequence dump Added functions DcmFind, DcmMove and DcmMoveHere
19980526	Added DcmCreateBitmap
19980610	Changed BGR to RGB in DcmCreateBitmap. Added DcmMoveHeaderHere
19980629	Fix: DimX and DimY were reversed. Improved efficiency when loading one slice.
19980630	Fix: last change caused crashes
19980707	Removed some checks to allow non-standard Philips images Added DcmSetProgressHook
19980708	Added DcmReplaceInHeader, which can be used for anonymization of dicom images.
19980709	Added C-CANCEL command object when a ProgressHook stops or aborts a DcmMove operation. Added support for NkiPrivateCompressed images in DcmMoveHere.
19980826	Added #pragma pack(1) in Field.h
19981105	Added DcmReadHeader
19981127	Added DcmGetLastXfms
19990108	Updated the dictionary for DICOM-RT support
19990114	Slices are sorted on coord. When duplicate coords exist, the last received slice eliminate previous ones. Included changed SOP-list by MvH
19981127	Added DcmTree, DcmPatientList and DcmSetLocal
19990825	Added extended patientlist export: DcmPatientList2
19990826	Added DcmMoveFilesHere
20000824	Added DcmFileGetRtStructNames and DcmFileGetRtStruct
20020425	Added DcmVersion and DcmCreateRtStructFile

CQCREATEDCM.CPP (inactive part of conquest interface DLL):

20020417	Created. All created DICOM stuff should go in here
20020502	DcmCreateRtStructFile seems to work
20041119	Work on DcmCreateSliceFiles
20041128	Tested write new scan on grid (and properties) of existing scan: needs ScanToDicom matrix to correctly compute

ImagePositionPatient; writes too many \\ in tags;
 SliceLocation not written; Now write abs slice thickness;
 Added missing *10 in Z coordinate of ImagePositionPatient;
 Added a minus to the Z coordinate of ImagePositionPatient:
 this is a kludge that works only for scans read with CQDICOM
 without rotations in scanner. Added creation date and time in
 default series description

20041223 Added XFM parameter to DcmCreateSliceFiles
 Added DcmCreateRtPlanFileLight
 Fix: also incorporate ImageOrientationPatient,
 FrameOfReference, StudyID

20050906 Some fixes in DcmCreateSliceFiles
 Added 'DcmCreateRtPlanFileLight()'

20051018 Several fixes in DcmCreateSliceFiles:
 - Required tags KVP and PositionReferenceIndicator were
 missing - Element numbers of RescaleSlope and
 RescaleIntercept were wrong; Set all MU's to 0 in
 DcmCreateRtPlanFileLight; added OperatorsName

20060704 Increased stringbuffer in GetContouData() (crashed ridiculously
 detailed on ESTRO data)

20061018 In DcmCreateSliceFiles, don't insert sequence items.

20070220 Moved UID_ROOT to somewhere else

20070423 In DcmCreateSliceFiles, use scan or extra properties for
 ImagePositionPatient

20070510 Made GetInterpretedType() more aware of markers.

20070705 Fixed bug in DcmCreateSliceFiles()

20070726 In DcmCreateSliceFiles, don't insert sequence items of
 ExtraProperties either!

20071018 In DcmCreateRtStructFile(), the created UID is stored in a
 global for later use

20080416 DcmCreateSliceFiles() also supports 4D and non-CT data

20080603 Few adjustments in DcmCreateSliceFiles()

20080708 Made PixelSize always positive in DcmCreateSliceFiles()

20080717 The filename parameter of DcmCreateSliceFiles() may also
 contain RemoteAE;RemoteIP;RemotePort for immediate
 pushing of created slices

20080930 Fixed bug in last change

20081001 Adapted for BCB's VR() change

20081002 Moved filename containing RemoteAE;RemoteIP;RemotePort
 to separate function

20081027 In DcmCreateUID(), the last digits are now time related, and
 not the IP address of the caller (adjustment to XVI
 requirements).

20090127 Had a go in implementing xfm in DcmCreateSliceFiles
 Fixed access to ReferencingPhysician in DcmCreateSliceFiles
 Fixed read only problem of pszFileName in
 DcmCreateSliceFiles, and allow AE;IP;PORT without trailing ;

20090128 Revamped xfm in DcmCreateSliceFiles:xfm is now
 ScanToDicomPatient

20090130 Fixed it! And documented code to write as read

20090131 Created ad-hoc, dicom and siddon modes (passed in
 iNewSTudy)

20090210	In DcmCreateSliceFiles, Series and Image Date/Time can be set in pExtraProps when it is NOT a new Study. SoftwareVersion can also be set in pExtraProps.
20090513	Support private and short tags in DcmCreateSliceFiles()
20090523	Allow RTDOSE export through DcmCreateSliceFiles: expects dose always in 0.01 Gy;Max number of slices to 2000; added missing test on presence of pXfm
20090524	Reverted a line of test code
20090602	Fixed handling private tags in IsGroupElement()
20100303	Added DcmCreateSC (Secondary Capture, Monochrome or color image); Fix : DcmCreateSliceFiles messed up non-square slices
20100409	DcmCreateSliceFiles() now uses SeriesNumber and StudyID from ExtraProperties
20100512	Kodak did not like our SC files without PlanarConfiguration (0x0028, 0x0006)
20110104	Do not override StudyTime when no new study Take AccessionNumber from extraprops or props when present Take AcquisitionNumber from extraprops when present

CQGLOBAL.H (part of conquest interface DLL):

19990318	Some cosmetics
19990511	Added a few warnings and prototype of MoveDDOHere
19990915	Added PixelRepresentation (0028,0103) in SLICE_INFO.
19991028	Added prototype of Signed_compute()
19991122	Added errorcode of failed C-STORE-RSP
20000317	Added BitsStored (0028,0101) in SLICE_INFO.
20000405	Added CheckRangeOfCT prototypes Added RescaleIntercept (0028,1052) in SLICE_INFO. Added RescaleSlope (0028,1053) in SLICE_INFO.
20000530	Added ImagePlanePixelSpacing (3002,0011)
20000707	Added gl_ppvCache
20000710	Added prototype of CacheInitFromSelectionStrings
20000824	Added prototype of CacheSetReferencedStruct
20000828	Added prototype of Dx_fm_compute
20000829	Added an ErrorCode
20010123	Added prototype of GetSliceCoord
20010410	Added gl_hevEndReceive
20010426	Renamed gl_hevEndReceive to gl_hReceiveThreadIsListening
20010508	Changed prototype of nki_private_decompress()
20010629	Added DoseGridScaling (3004,000e) in SLICE_INFO.
20010717	Added gl_pszExtraParams
20010831	Added prototype of CheckModalityLUT
20020418	Added prototypes for cache-ing properties
20020424	Added declaration of field gl_pLastProperties
20020613	Added prototype of MkTypedDicomObject
20020814	Added two Print error-codes
20021028	Added prototype of GetFloatsFromKey
20021213	Added RtImagePosition to SLICE_INFO structure
20030122	Added prototype of DecompressJPEG
20030128	Added 'iNbTimeSequences' and szPhotometricInterpretation to the SLICE_INFO structure
20030129	Added prototype of READ_KRETZ_compute

20041119	Added support for 4D sliced object
20050203	Added prototype of CacheCleanupRtObjectFiles
20050906	Added prototype of CacheGetReferencedProperties
20060913	Added prototype of GetFloatsFromVR
20070130	Added ec DCM_E_INVALID_RTPLAN
20070220	Sdded a few prototypes and an errorcode
20070506	Added optional prealloc data pointer to CreateOutputField
20070522	Added szSoftwareVersions to SLICE_INFO structure
20080717	Added prototype of DcmStoreFiles
20081002	Added prototype of FileNameOrDicomNode
20090204	Added prototypes for transpose_Xbytes_inplace()
20090602	Added fAcquisitionTime to SLICE_INFO, for coodinates of 4th dimension
20090610	Added DCM_E_MULTIPLE_RTPLAN, used when 'UPI' is not unique

CQRTOBJECTS.CPP (inactive part of conquest interface DLL):

19990407	Created. All DICOM RT stuff should go in here
19990404	Added RTSTRUCT
19990511	Added RTPLAN. Still under construction !!!
19990825	Some changes in RTPLAN; still under construction
20000619	Added GetStructInfo to be used by TREE
20000707	Lots of work on RTSTRUCT and RTPLAN
20000828	Structure and beams has wrong XFM when patient is prone
20000829	Added GetRtBeamFromRtImage
20010302	Fix: improved temporary filename
20010410	Again improved temporary filename...
20010619	Added some comments on RTPlan organization.
20010717	Store szReferencedFrameOfReference in STRUCT_TREE_INFO; To do: in 'CalcBeamXfms' the collimator angle is not treated right when the patient is prone.
20000824	Added DcmFileGetRtStructNames and DcmFileGetRtStructFixed two small memory leaks.
20020225	Fixed free-ing twice in 'GetRtStruct'
20020502	Removed some warnings
20020613	Moved MkTypedDicomObject to CqServer.cpp
20030122	Some work on GetRtBeamFromRtImage: still under construction due to world-wide lack of good data!!!
20030520	In GetRtStruct, fail with DCM_E_INTERNAL when there is no cached underlying scan.
20050125	Made beam field output contain short data instead of char data GetRtBeam has an optional string parameter to load a specific segment (= controlpoint); Added GetPlanInfo for TREE building purposes
20050131	Moved MkTypedDicomObject() in GetRtBeam to StoreBeamPropertiesInCache
20050201	Fix: do not delete pDDO in GetRtBeam anymore
20050202	Fix: When settings of a controlpoint do not change, they may be defined only at controlpoint 0.
20050921	Change in CalcBeamXfms for FeetFirst and Prone patient-positions (This change is already in use for two months, and has been approved by Peter Remeijer)

20070705	Gave individual contours instead of organs a unique number (solves the problem of more than one contour in a slice)
20080313	Removed some warnings
20080717	Removed empty structures from the PatientTree in GetStructInfo()
20091028	Adapted for 64-bits version

CQRTOBJECTS.H (inactive part of conquest interface DLL):

19990407	Created
20000619	Added prototype of GetStructInfo
20000707	Added more RtObjectXXX prototypes
20000824	Added szReferencedStruct to BEAM_TREE_INFO. To be used in order to get the XFM of a beam IFF RTPlanGeometry equals 'PATIENT'. In the other case (when RTPlanGeometry equals 'TREATMENT_DEVICE'), the the XFM is defined by the 'IEC fixed coordinate system'; this case is not handled in CqDicom.
20000829	Added prototype of GetRtBeamFromRtImage
20010717	Added szReferencedFrameOfReference in STRUCT_TREE_INFO To do: in 'CalcBeamXfms' the collimator angle is not treated right when the patient is prone.
20050126	Moved DCM_PLAN and DCM_STRUCT typedefs to here; added a few strings to DCM_PLAN; Added prototype of GetPlanInfo; Added parameter to GetRtBeam

CQTREE.CPP (inactive part of conquest interface DLL):

19990504	Creation: Moved Tree and PatientList from Avs_dcm to here
19990511	Adjustments concerning DcmSetLocal
19990825	Added extended patientlist export: DcmPatientList2
19990826	Implemented DcmPatientList2
19990830	Fix: ParseSelInfo did not handle an 'extra' Key/Value pair in a selection-string.
19990906	Improved Alias-names of Dicom Tree Interface
19991019	Besides CT and MR, the images of modalities US, DR and DS belonging to the same series are grouped into a single volume.
19991026	Use SeriesDescription in tree if no SeriesNumber/SeriesTime available
19991028	MultiFrame images will be treated as '.scan' instead of '.image'.
19991029	Added support for multiple multi-frame images in one series When MovingHere only one image, the C-FIND operation is skipped. Fix: EchoNumber was discarded when retrieving header-info
20000126	Fix: EchoNumber was used even when value is '(empty)', only in cases where EchoNumbers were needed in other parts of the tree.
20000128	Fix: three memory leaks
20000426	Adjustment for difference in response of Borland ODBC and Microsoft SQL: an empty string-field is returned as "" and " " respectively.
20000707	Put structures in the tree under a scan
20000710	Fix: Update cache when TREE has been bypassed
20000717	Fix: XFMs of multiframe-scans were not returned

20000808: Extra check: SOPInstanceUID must be passed in selection for RT objects;

20000823 Fix: Crash in building of TREE when the move-response of an RT-object was OK, but no DDO was transmitted.

20000828 Fix: Structure and beams has wrong XFM when patient is prone

20000829 Added RTIMAGE properties: 'refmeam', 'refbeamwordxfm' and 'refbeamgantryxfm'

20010420 Added support for PatientID containing '/' or '\\'.
20010619 Start implementation of RTDose objects.

20010717 When RTStructs do not reference a Series, use ReferencedFrameOfReference if possible. Caller may use 'IMAGESONLY=1' in DcmSetExtraParams when grouping of images into scans is not desired.

20011214 Damn! It now seems best that US images are not grouped together anymore...

20020403 Fixed bug when UIDs are 64 bytes long. Got rid of two #defs

20020501 Added cacheing properties

20020521 When modality equals PT, combine slices to a volume.

20020613 Always use SeriesDescription in tree

20020619 Split date parts with dots; display studymodality (0008,0061) in study alias

20020620 Merged 20020613 with 20020619. Do not split dates.

20020815 In 'ParseDicomInfo' allow also ':' as a separator

20021011 Fix: When bypassing the TREE, cache was not initialized when retrieving a single image or a multi-frame volume

20021011 Added other query modes (4..9) for DICOM_LOAD_PATIENTLIST (CreatePatientList)

20021014 Merged the two versions

20021028 Try to get XFM's of multiframe objects right.

20030128 Support for 3D TimeSequences, resulting in 4D 4-space output scans. Make check on QueryResponse case-insensitive (choked on PatientID Test vs test)

20030130 Support for 3D private Kretz ultrasound

20030213 Above change crashed when in a FIND operation there is US present, and the queried dicomserver did not supply the 'Manufacturer' key.

20030624 Support for multiframe dosegrids

20030710 Additional sorting of C-FIND result on SOPInstanceUID

20040525 Also send PatientID when query/retrieving RT objects

20050126 Made better ALIAS in TREE for RTPLAN
Made ALIAS in TREE of a series '(empty)' if no good ALIAS can be constructed
Implemented interface to load individual segments in a beam
Moved DCM_PLAN and DCM_STRUCT typedefs to "CqRtObjects.h"

20050131 Check empty sequences in 'StoreBeamPropertiesInCache', and made DDO a TypedDDO

20050201 Handle possibility of retrieval of beam properties before retrieval of outline in ReadKeys()

20050203 Function GetRtDDO now caches DicomRT files. When a NEW PatientTree is constructed, these files (of a different patient) are deleted. This speeds up things enormously

20051018 Support for Corvus in FixControlPointProperties

20060103 Studydate queries on val instead of *val* (kodak could not handle it)

20060531 Added exported function 'DcmGetPlanPois'

20061018 Apply rotations to DcmGetPlanPois when the patient is not HFS

20061101 Recoded 'DcmGetPlanPois' (pretty complicated now...)

20061109 ROI and POI RTStructs from Pinnacle do NOT have exactly the same timestamp

20070112 Extra consistency check in DcmGetPlanPois

20070220 Adjusted DcmGetPlanPois so it can handle 'derived' plans
Added DcmShiftPlanIsoc

20070419 Fix in PatientTree in presence of long RTPlanName

20070423 Added outputstring pszCreatedUID to DcmShiftPlanIsoc

20070508 Fix: date query on empty mask failed

20070522 Temporary workaround for XVI-upi-bug in DcmShiftPlanIsoc()

20070530 Added DcmGetPlanStructUIDs()

20070530 Fixed bug in DcmGetPlanPois() when more than 1 isoc defined in RTSTRUCT

20071018 Added DcmGetLastCreatedStructUID and DcmSetLastCreatedStructUID. This is a mechanism so that a newly created RTStruct can be referenced in a shifted RTPlan.

20071031 In DcmGetPlanPois(), support 'QUIRT CqDicom' structuresets too. In CreateTree(), treat a marker as a structure

20080313 Support markers; treat them as structures; In DcmGetPlanPois(), the consistency check on isoc in plan and structureset is omitted. We take the isoc from the plan.

20080318 Store RTPLAN properties also in cache, and added 'NumberOfFractionsPlanned'

20080320 In ReadKeys(), also use SOPInstanceUID for RTDOSE

20080717 Moved prototype of DcmStoreFiles to CqGlobal.h

20080818 Made to compile with MS7 (fabs on integer)

20080908 ix xfm2 for &Kretz, and implemented those for &Philips

20081002 Adjustment in the Tree for Philips ultrasound (possible 3D scan in private VR)

20081124 Fix matrix for Philips 4D US (AMC patient 0321)

20090202 Fix display of study modality in patient list (is now in dictionary)

20090610 Added safety check in DcmGetPlanPois(), DcmShiftPlanIsoc() and DcmGetPlanStructUIDs() in case our so called UPI is NOT unique

20091028 Adapted for 64-bits version

20091103 Adjusted DcmGetPlanPois() to new Pinnacle version; should be backwards compatible.

20100629 Fixed FixControlPointProperties() for Nucletron RTPlans

20100827 Caller may use "EXCLUDERTOBJECTS=1" in DcmSetExtraParams when loading of RT Objects is not desired (e.g. for faster calls to CreateTree())

20101214 Fix: GetRtDDO() now checks on whether something was actually received; Work on DcmGetDoseFromPlanURL()

CQCACHE.CPP (inactive part of conquest interface DLL):

20000710 Added CacheInitFromSelectionStrings

20000808 Fix in comparison of selection with cache

20000824	Added ReferencedStruct to cache (used for beams)
20000829	Also store 'image' in cache
20010717	Some work on RTDose objects
20020418	Added cache-ing of properties
20040616	In CacheInit (Called from PatientTree), keep the cache if the tree is from the same patient
20050203	Added CacheCleanupRtObjectFiles. This is done just before a PatientTree of a different patient than last patient is being constructed
20050906	Added 'CacheGetReferencedProperties()'
20050919	Increased MAX_SEL_CACHE from 200 to 2000
20061109	Increased MAX_SEL_CACHE from 2000 to 10000
20080318	RTPLAN properties are now cached too
20091028	Adapted for 64-bits version

CQPRINT.CPP (part of conquest interface DLL):

20020814	Added exported functions 'DcmPrintScan' and 'DcmPrintFile'; Altered 'PDU.AttachRTC' in 'PrintGrayScaleImages'; Tested with OFFIS and Mallinckrodt
20020816	Padded uneven string VR's with space Support of byte images. (RGB does not work yet)
20020823	DcmPrintFile accepts multiple filenames (separated by ;) and decompresses NKI data
20020825	Added DllProgress calls during print (used by server to ensure printing without loosing socket data)
20051217	Use nicer method to replace compressed by uncompressed data
20080313	Removed some warnings
20080818	Made to compile with MS7
20091028	Adapted for 64-bits version

CQINI.CPP (inactive part of conquest interface DLL):

20020814	Added exported functions 'DcmPrintScan' and 'DcmPrintFile';
20081002	Created
20081014	Fixed parsing of backslashes in KeyValueToVR
20081015	Fixed translating hex values in KeyValueToVR
20081029	When the data of a VR exceeds MAX_DATA_LENGTH, then a '*' is appended to the Key in DcmReadIniObject() The '*' is ignored (stripped) in DcmWriteIniObject()
20081106	In DcmSetField() and DcmSetString(), also accept a new VR. For now, this is only implemented when the VR is not inside a sequence.
20081111	Fixed errorhandling in above functions
20090602	Added DcmPlanIniToPrescription.
20100218	Two fixes in GetVRFromDDO. DcmGetString and DcmSetString crashed when parameter pszKey is sequence item AND a literal string passed from Delphi. When the sequence item is out of range, now DCM_E_MISSING_VR is returned instead of DCM_E_INVALID_VR.
20100701	Recoded DcmSetString()
20100810	Recoded DcmSetField()

CQJPEG8.CPP (inactive part of conquest interface DLL and dgate.exe):

CQJPEG12.CPP

CQJPEG16.CPP

20020301 Created
20030415 Better exception handling
20030417 Support for multi-frame images
20041221 Changed include files
20050215 Fixed for fragmented frames in a sequence
20050901 Renamed errorcodes.h to errcodes.h
20070112 Return Width, Height and BitDepth of the image
20071001 Output of JPEG error messages to OperatorConsole in server
20080819 Suppress large amounts of jpeg errors; added \n
20081116 Adapt for very large objects (use unsigned int lenght)

CQJPEG.CPP (part of conquest interface DLL):

20030123 Created
20030128 Start work on RLE decompression
20030415 Fix in determining RLE decompressed size\
20080313 Removed some warnings

DICOMP.CXX (sample print program)

20020816 Fix in (unfinished) main server code
20020822 Made this sample program part of code base
20080113 Fixed for MS7 compiler
20100120 Fixed for MS8 compiler

TEST.CXX (sample client program)

19980322: played around; added comments in query client
19980322: Added command line interface for c-find
19980323: Added command line interface for c-move
20000318: Added optional NKI command vr's to echoclient
20020315: Started on -fe: forwarding server; implemented echo, working
 on move and find
20020316: Added cstoreclient -t; load dd.txt as dicom dictionary;
 -fd=diagnostic mode (logs) Added IARQ; note: cast of PDU to
 dump() seems to crash sometimes. Forwarder works between
 QUIRT/conquest apps and conquest dicom server.
20020317: Made forwarder more general; works between EFilm and
 Conquest
20020811: Fixed forwarder such that it works for dicom-print
 Fix: Forwarder did not worked with OFFIS because OFFIS
 checks message command/data pdv bit 6[1] that is was not set
 correctly by forwarder (command/data object)
20020822: Made this sample program part of code base
20030630: Fixed crash in cast of Dump(PDU) (c.f. 20020316)
 Fixed passing of cmds to Find and Move
 Fixed continuous CechoServer
 Removed calls to obsolete PDU.Write()
20030721: Fixed SetStringVR (pad uneven strings with a space)
 C-Find and C-Move wildcards "*" seem not to be valid
20040711: Fixed forwarder: status=0xff00 loops responses; status==0
 keeps association open

- 20040712: Added auto-returner to forwarder - functions with NKI viewers (assumes viewerAE of X%ip or XX%ip).
You need to set AE and port of return forwarder in your PACS though; see gatewayname, gatewayport, returndump and viewerport for configuration
- 20040713: Attempt to make it multithreaded; however return connection only starts once (supports single client only!)
- 20040713: Split into modes -fp: for printer; -fb: forwards and returns; -fe normal forward; -ff fixing forwards
- 20051217: Use new ReAlloc method to replace data in VR
- 20060618: Allow (and disabled) looping of c-move to test repeated c-move problem in read-ahead thread
- 20061127: Allow (and disabled) looping alternating c-move and c-find to test server
- 20070902: Fixed for ms7 and ms8
- 20090411: mode -fp adds more transfer syntaxes uid2..uid8 (experimental)

DGATESERV.DPR (part of dgateserv.exe)

- 20000316 Created
- 20000326 Close handle on terminate
Dgate runs via new SlaveProcess mechanism of Uservice

USERVICE.PAS (part of dgateserv.exe)

- 19990316 Fixed English of some messages
Added account and password as optional parameters for install
- 19990325 Added option to start/stop/install/uninstall on remote machine
Works fine with start / stop / uninstall
Remote install works except for wrong exe path
Therefore added ExeName property and command line switch
- 19990326 Added service option: makes it a generic installer!
Made ServiceName and ServiceDisplayName read/write
Added AllowSlaveProcess: command line can set slave process
Added SlaveProcess and SlaveProcessArgs: executable to start
Fixed arguments passing when /start is followed by more /
Fixed ServiceName: is read as first arg in DoCmdLineStart
Note: SlaveProcess is killed when service stops
- 20100827 WaitForStop also tests the SlaveProcess: service stops when slave crashes; Version in help text: now V1.1
- 20100901 Avoid RaiseLastWin32Error for common errors: it now shows a crash window
- 20101111 Avoid RaiseLastWin32Error for failure to start, use ShowMessage
- 20101113 Fix for vista - stop does not reach service
- 20101116 Stop before uninstall fixes uninstall for vista

SERVERDRIVER.PAS (part of conquestdicomserver.exe)

- 19980405 Created
- 19980407 Added dgatesop.lst file creation; max AE to 16 chars
- 19980407 Sorted image list on SOPINSTANC to get correct order
- 19980412 Added DuplicateHandle code; but still deadlocks

19980413 Threaded mode avoids TCP/IP component (seems to fix deadlock)

19980414 Restart after switching server mode does not always work;
Fix application to threaded mode for now.
Added compression configuration.

19980415 Some fixes for the first installation code

19980505 Made length of directory field in MAGDEVIC 250 chars

19980608 Replaced TQuirtViewer with TImage => no need to load ntpars.dll

19980614 Merged code with use FPiette tcp components

19980616 All OK with new fpiette code; actually I had to fix my own bug

19980617 Save as bitmap and dump header in popup menu

19980618 Query and Move options for CQDICOM.DLL

19980619 Added header lister; default MagDeviceThreshHold to 10 MB

19980620 MagDeviceThreshHold is now configurable

19980621 Renamed table to MagDevice; use full table names
Nicely coded DBF/Access choice; but fixed to DBF
Delete patient and refresh database menu options

19980622 Double click query memo to make new query
Show on bitmap: patient is being deleted/file not found

19980623 Save bookmarks of tables; move to next record after delete

19980624 DICOM.SQL: StudyNumber (StudyID in DICOM) is now 64 long (Microsoft SQL server does not allow truncation)
Update bitmap after loading for fast display
Text strings in UDP input to 1024 (for long SQL errors)
Default password and user name to 'conquest'
PAGETIMEOUT to 600 for better database performance
Made DataSource name more or less configurable
Made all port numbers more or less configurable
Installing a second server on one PC while one server is running works

19980702 Added KBUSED field in MAGDEVICE table

19980703 Some changes in static texts and hints

19980704 Found a handle leak in RunDgate; not fixed yet

19980708 Make anonymous; show progress for move

19980710 Added experimental archive option; fix stopcopying option

19980711 Fix double click query on empty form; use tray icon

19980713 Add Name of server in tooltip of tray icon; fix DiskFree

19980717 Fix crash if directory does not exist

19980721 Clicking tray icon restores window

19981219 Improved error handling display;

19981220 List header of undisplayable object in static
Read and save FileNameSyntax; default=3

19981221 Supress display of large files (>1 MB)

19981222 Use FieldByName, added combobox selectors
Disable study navigator if only 1 available
Moved modality from the study to the series level

19981229 Changed order of TableN.close to avoid side effects

19990104 Fixed MAGDEVICE creation under SQL server (KBused optional)

19990109 Made ImageNumber 10 bytes wide (for GE AdvantageWin)

19990110 Combobox1 is now patient filter and or selector
Added access to cache and jukebox in browser but with limited support: up to 10 devices (one character)

19990112 Added archiving page; fixed most handle leaks
 Finished first archiving page; -hide command line option
 Added clear database and regen single device buttons
 Clear combobox for each page select
 NOTES: Archive page is configured by editing
 Restore original configuration or restart to load it
 Archiving allows up to 10 Jukeboxes, but only the
 selected one is used for writing. Only the drive
 with MAG0 is checked for disk space for automatic
 archiving. Only device CACHE0 is passed correctly
 to the burn program. This means: stick to 1 MAG
 device and 1 CACHE device for now.

19990113 Speeded archiving by selecting MAG0 if there is only 1
 Renamed entry to BurnThresholdMB, made BurnProgramList
 Note: to do - keep log file of archiving operations.
 - add RT objects to default sop file.

19990114 Added date and time formats for burnprograms
 19990115 label format burnprograms; no write empty ini
 Sync magdevice0 with DataDir; %% and %p formats

19990315 Added EchoNumber in image table - used to separate PD,T2
 Completed dgatesop.lst with Philips, GE, RT
 Added NumberOfFrames in image table - to find multiframe
 objects

19990413 truncate QUERY request depending of level
 19990510 Allow AEs starting with a digit (Siemens MRI)
 19990527 Revision 3 of database layout (birthday, sex, bolus)
 19990902 Added load/store of mirror devices and export converters
 19990905 Switched word wrap off for memo's
 Added DcmMoveFilesHere on right click of copy; but seems to
 crash; Problem: if we pass DcmMoveFilesHere servers
 AE/PORT saving slices from own server will fail. Now pass
 'test;1234'

20000127 Hide archiving tabsheet in new install; Add NKI sops to install;
 20000128 Added extra GE sop; compressmode 2; disabled movefiles here
 20000316 version 1.3.5: Integrates dgate as service operation
 New install does not start dgate right away

20000317 Truncate patient list if too long
 20000318 '\$ RESTART' as console message kills and restarts server
 '& DGATE xx' as console message runs dgate with args xx

20000323 Added query sop choices through level dropdown list
 20000326 Service supported for more than one server:
 servicename=AE; port passed as /process dgate.exe -!port
 Removed need to enter password for SQL access
 Save and restore user and password in dicom.ini
 Added 'ping' option for remote check of archiving:
 Deletes ConquestDICOMServer.Ping and makes
 ConquestDICOMServer.response with archive status
 Allow up to four servers at the same time, also as services
 Note: only the first server will start correctly on boot!
 (on boot, the /process parameter is not passed and the
 default IP message port will be used -!1111); Presence of file
 USESQLSERVER allows install on SQL server
 Presence of file USEMSACCESS allows install using access db

20000328 Added tests of local server at key moments

20000402 added -noconfirmquit option; show VERSION in caption
 Install service with startup at right socket
 Read service socket from registry on startup
 20000403 Improved socket handling; fixed space in path when not
 service; space in path blocks service install.
 Now recognizes installed but not running service
 Number of servers on one PC is now unlimited.
 20000417 Fixed MSACCESS ODBC creation
 20000501 ExportModalityx should be UPPER-case
 20000528 Added "force display of large image", "remove image from
 database", "run external viewer" and demoviewer option
 Removed '.' as workdir from runprogram etc (allow bat files ?)
 Added democopy option: copy incoming to dir + CallingAE.v2
 20000529 Merged two changes
 20000529 Release to 1.3.6
 20000616 Attempt to create NewInstallForm database selector
 20000625 Added NewInstallForm database selector; default to
 MSACCESS
 Added Send ... To menu items in popupmenu
 Removed '*' items from Dicom system selectors
 Added Send Selected Images for more flexibility
 Added animated display menu options
 20000626 Added Send Selected Images selector window
 20000701 Some adaptations of Send Selected Images;
 MaxFileNameLength
 TrimRight strings in query - trailing spaces depend on db
 20000707 Added Edit Database by Hand menu option
 20000808 Added ,0 code in timed archiving
 20000915 Added ListenSocket.Addr (0,0,0,0) in multi-app mode
 20001107 Added CheckBoxViewIncoming for demo display of
 incoming images
 20001108 Display originating server on incoming image;
 Disable menus when the ViewIncoming option is selected
 20001129 Added fix so that DICOM.INI may contain UNC path
 Added menu choice to change ID of selected series
 Version to 1.3.7
 20010312 Trimmed remote IP so that hostnames can be resolved in
 CqDicom.dll
 20010318 Trim servername and ip;
 alt-rightclick the service buttons installs service 4 times
 e.g., at ports 5678, 5679, 5680 and 5681 (all with same data)
 This may be used for efficiency and to limit the effect of
 crashes; Version to 1.3.8
 20010327 Always pass curdir to runprogramxxx and startprogram -
 required because directoryview changes current directory!
 20010328 Uppercase account for service; colors for archive status
 Added verify mirror disk button; burn only once per time
 Show date and time for archival steps; added KeepAlive
 Added MirrorVerifyTime (verifies whole MAG0)
 Added conquestdicomserver.log with major messages
 20010405 Added comma-separated list option for patid in query page
 20010406 Ctrl-dblclick adds patid to comma-separated list
 20010406 All status memo's are now limited in length and save to
 separate log files

20010409 Added (and disabled) simple zip code for log files per day
 20010409 Do not log query page
 20010409 Added Tape backup page; replace few GetCurrentDir by CurDir
 20010410 Added timeouts (10s and 5 hours) to tape backup
 20010412 Separate tape backup button - some fixes
 20010416 Improved anonymize and change patient ID using new dgate -f options; Added drag and drop interface to add files to server
 20010418 Changed description of forced backup set; refuse dropped directories; Version to 1.3.9
 20010418 Update maintenance memo during drag and drop
 20010418 Fix: timer4 was enabled if no KeepAlive specified in dicom.ini
 Hide tape backup page in new installation
 Added default install button in firstinstall
 20010419 Added anonymize and change patient ID for study
 20010502 Added DeleteThisStudy/Series
 20010830 revision 4 of database definition (extended)
 Added ExportStationName in dicom.ini
 Updated dgatesop.lst to conform to DICOM 1999 final text
 Version to 1.3.10
 20010903 Suggest .username as account
 20010904 Fix: Query was freed twice when a DicomMove failed
 20011108 Added automatic zipping of log files
 Make sure that tape backup and archive pages are not shown
 Added save/restore of ExportFilter settings
 20011109 Small fixes:
 do not write empty configuration strings (empty<>default)
 keep some settings with original case (see sorg)
 use StrToIntDef to avoid exceptions on invalid settings
 Added ZipTime as configurable
 20011127 Added SeriesPat and ImagePat fields to DICOM.SQL
 Surpress fail message box when using SQL server
 Version to 1.3.11
 20020402 Adapted controls.pas: exception handling in finddragtarget and findvclwindow
 20020409 DB rev 6: added frame of reference UID to series
 20020416 Store/save UIDPrefix for in dicom.ini
 When creating dicom.ini, create unique UIDPrefix:
 '1.2.826.0.1.3680043.2.135.Date.Time
 Added configurable TestReadTime: regular check of MAG0
 20020417 Fix: QueryResult has not necessarily 64 char
 Renamed some data structures
 20020422 Do not save empty UIDPrefix; fix change patid for study
 20020424 Added simple weekly checks page; mail ALL failures
 20020426 Fix in mail sender list; use first e-mail as 'to' address
 20020428 All recipients show in to: header
 Only serious error messages have *** in string
 Added MailCollectTime and MailWaitTime
 Avoid starting check more than once
 20020510 Added CheckDicomSubmits (send mail when users forgot to send images to the PACS)
 20020513 Checks: empty time runs all day; added log header/footer
 Updated CONTROLS.PAS: more exception handling
 Fixed reading of weeklychecks; display CD num in header

20020515 Fix warnings and patient ID checks in CheckDicomSubmits
 Added ImageType to database (rev 7)
 Added find missing patients button;
 allow more than one weekly check time (; separated, not adjacent)

20020515 Fix missing pats when query empty
 Added grab options to weekly checks

20020517 Changed StudyModality to 64 chars (e.g., fits :
 CT\MR\PT\XA\ST\MG\MS\RT\IMAGE\RTDOSE\RTSTRUCT\RTPLAN\RTRECORD)

20020518 Fix in weeklychecks tag: too many with tag 0

20020519 Fix in rev 7 db definition: AcqDate is now SQL_C_DATE
 Added FixPhilips in DICOM.INI

20020520 Use DGATE.EXE for grabbing

20020522 Fix reading of entries with an '=' in e.g., dicom.ini

20020524 Fix acrnema.map problem with new install

20020529 Fixed duplicate mailing of messages, added WriteMemo
 Added archive.log; extra parameter for regen for DBASEII
 Removed MessageBoxes

20020609 Read and write FileCompressMode, default remains 2

20020613 Added alt-drop code: enter with new patient ID

20020819 Basic printer support; view incoming shows printer data

20020819 Process messages as files are dropped
 Print incoming imageboxes on default printer

20020821 Printer accepts landscape and portrait any pictures per page
 Progressbar shows printer queue activity
 Note: printer queue is single user (but may be fixed, using film#)!

20020823 Added print options in browser - print to local dicom server

20020825 Headerlister is modeless - added CTRL-A, CTRL-C, CTRL-F,
 CTRL-S, F1, F3 end ESC
 Printer has progress hook - required to avoid socket data loss
 Selection of print page format; edited info text
 Added all 1.3.11 fields to edit database by hand

20021015 Start deleting patient started delete "=ALL when no patient in
 list!!!!
 Protect all pop-up functions for no selection

20021016 Better support for copying large #patients, even if one or more
 missing
 Added denormalized database option (for DBF without ODBC)
 Removed obsolete magdevice button

20021017 Added "" around dropped filenames; use filter instead
 of master-detail (requires index) for built-in dbf support
 Version to 1.4.0; added dropping of directories
 Added UID counter passed to dgate to avoid UID problems

20021018 Fixed browser fail and missing dbase dir after non-odbc install
 Limit dbase list in browser (alternative to image) to 100 lines

20021020 Added Kill and Restart the Server weekly check (packs
 built-in dbf); Fix reading of database name if not default

20021021 Small fix in maintenance memo for very long strings

20021028 Also denormalized series database; default de-normalize study
 (rev 8); Show denormalized study entries in database editor

20021029 Fixed query keys transmittal (sent sop when not needed)

20021213 Added EnableReadAheadThread as configurable parameter
 Disabled GECTStorage amd GEMRStorage in dgamesop.lst

20030113 Version to 1.4.1; Fixed keepalive option (was never enabled)
Added PatientQuerySortOrder and such

20030128 Fix update of headerlist when no image shown
Make DBF table links case insensitive

20030319 Fix: ImageDB entry 'ReceivingCoil' is 16 bytes (not 12)

20030324 Start on 1.4.2: db rev 10: made ReceivingCoil 16 wide

20030402 Merged above two changes

20030409 Handle DicomPrint error; several path fixes in tape-handling

20030613 Catch WM_QUERYENDSESSION: shutdown disables close dialog

20030628 Fix that more than one dropped directory will be loaded

20030706 Prepare for release 1.4.2; new dicom.ini; kill server
Compression column in acrnema.map

20030707 Added manufacturer to denormalized db (rev 11)
Write registry key
HKEY_LOCAL_MACHINE\SOFTWARE\NKI AVL\Conquest\Dicomserver\SERVERNAME
with values: directory=path without trailing backslash and
laststarted=yyyymmdd. For use by Dicom-works only.
Note: servername changes are updated on GUI startup only.

20030708 Made JPEG compression configurable (affects dgatesop.lst)

20030709 Write version into registry

20030710 Install service as system account (no user required)
Fix attempt to mail when not wanted
----- release 1.4.2 -----

20030717 Fix problems when registry access fails

20030809 Will automatically use ADO or MDBF when BDE not installed
Set version to 1.4.3

20030810 Log used DB engine; Added partial ID matching for MDBF
Default enable uncompressed button before install
Force user to push 'Save config button'
Added some text to install and maintenance pages
Do not display images when on jukebox (use force display)

20030819 Made max drop filename length longer (256->1024)

20030905 Date of zip file is one day before if zip before 8 in the morning
Hide external viewer menu directly after install
Optional configuration of SendUpperCaseAE added
Removed obsolete flush button
Do not allow install in directory with spaces in its name
Removed faulty extra text from install page

20030907 Fix ODBC generation for directory with spaces
Just warn against directory with spaces

20030909 Test write access to log files prior to startup

20030920 Mail messages that contain 'fail' instead of 'failed'
Added tape full warning (that will be mailed)
Added enables for timed archival (to allow retry/timer disable)
Changed order of burn CD message / tape backup
Added NextSlotToUse (slot n) next to NextCDToBurn
(JUKEBOX0.n); % in burn parameters %n = NextSlotToUse,
%c = NextCDToBurn; Manual archive buttons control
checkboxes as well

20030921 CTRL-Double click "automatic archival label" starts archival
CTRL-Double click "weekly checks time" starts check
Added NightlyCleanThreshold parameter (clean # MB at
01:00)

20040401 Added ImageID to database; version to 1.4.4
Show ImageID in database editor and browser

20040402 Allow anonymize etc on MAG0..MAG9;
changed datasource name (now system DSN)

20040425 Added DoubleBackSlashToDB to dicom.ini

20040426 Version to 1.4.4a

20040528 Use built-in servertask commands for maintenance and browser

20040530 Browser for built-in DBASEIII driver uses extract server
command; Database editor to read only for built-in DBASEIII
driver; These changes allow efficient use of in-memory index
for dbaseIII; OK for 10⁷ images when patientID, studyUID or
seriesUID passed in Queries; Added PackDBF and IndexDBF
to dicom.ini; Reworded some in popup menu; wait for patient
deletion to finish; Fix refresh for sql server

20040601 Added LongQueryDBF flag

20040605 Fix extract when no patients; kill server stops loop of server
tasks; start server on activate (after windows shows)
Pack the Database as Weekly Check Action

20040606 Fix in formactivate; display @ startup

20040610 use servertask to clear index and rebuild it in built-in dbase
mode

20040614 small fix in keep server alive; added nightlymove option

20040615 small fix in nightlymove option; also reindex after regen device

20040713 Added FixKodak; version to 1.4.6

20040722 Installation defaults for FixPhilips and FixKodak are now 0

20040804 Added *.lst to zipped log files; added bug report option
Version to 1.4.7

20041029 Added merge and split series options;
Process messages during long GUI actions if possible (no DB)

20041030 Modified GUI of merge and split, process messages

20041101 Check for MAG storage for split/merge
Delete from DB does not use image (for orphaned DB)
USEDATABASEIIIWITHOUTODBC replaced by Pos('\',
Datasource)>0 while running

20041102 Added nj compression mode

20041108 Added warning when alt-dropping file(s) with \$.. as Patid
(debug)

20041128 Fixed wording of delete messages: works on all mag devices
Fixed problem in DeleteThisSeries, Anonymize, and change
Patient ID: last series could not be modified/deleted

20041129 Made font size of server status window selectable; Made debug
log switchable; service will continue to log to file even if
user interface is closed (but with different date/time layout)
Now can save and use acrnema.map without server restart
Fix startup problem when disk 'c' not available

20041201 Backup fail 1..5 files temp logged as ALMOST OK ->
continues

20050106 Fixed install as four services, fixed nightlymove amount
Is now nightlymovethreshhold - free space: i.e.,
attempts to make nightlymovethreshhold MB free space
Fixed nightlymove: forgot to pass ',0' for MAG0
Also, try move even if select failed to avoid stall
after partial failed select

20050109 Made backup of and warn when dicom.sql is overwritten

Added TCPIPTimeOut, FailHoldOff, RetryDelay, QueueSize
Version to 1.4.8

20050114 On startup ask to delete ExportFailures*, CopyFailures*

20050422 Added BurnTime2 (2 dvds per day) and BurnWeekend (all weekend through)

20050429 db rev 14: added worklist support

20050829 recurse directories when dropping files to ease load dicom CD

20050831 Added find modality worklist to dgatesop.lst

20050901 Added clear worklist button, version to 1.4.9, WorkListMode

20050905 Modality worklist query, auto append worklist to dicom.sql

20050907 Avoid dicom.sql message in new install, v* and s* in acrnema.map

20050912 Small changes to dicom.ini, dicom.sql, dgatesop.lst; added SQLHost. Small change to mdbf: extra refresh avoids browser failures

20051024 Version to 1.4.9a: recompiled with patched controls.pas

20051230 Version to 1.4.10

Moved clear worklist button and add pack database button

Added warning when worklist is added to dicom.sql: clear worklist required

Added settings: DebugLevel, CacheVirtualData, LargeFileSizeKB, VirtualServerFor#

enable study controls prior to return if same (none) image shown

20060102 Update of hints

20060103 1.4.10 Release

20060201 Added K-Pacs/EZDICOM based viewer

20060312 Keep case of VirtualServerFor; Message buffers from 1024 to 8192; Open k-pacs viewer on correct series; version to 1.4.11

20060314 Added experimental native Mysql access; installs when libmysql.dll is found - uses mysql version 4.1.14

20060315 Keep case of FileNameSyntax

20060328 Updated hints and file version for release 1.4.11

20060618 Set default user for mySQL to root (thanks Chris Muller)

Seems to then also work with mySQL 5.0.22, version to 1.4.12

20060707 Bug reported by Paolo Marchesi: save config loses DBF

Now only set to default when previous does not exist

Use same zipmaster as in previous versions (zipmaster\org)

20061101 Added SeriesDescription to query page

20061129 Added control for debuglevel, and allow queries on UID

Double click queries labels to change mode

20061130 Added new dgate parameters; consolidated debuglevel control and param; Made application forms resizable and set size constraints; Small font control works on all status pages; Double click image to force display it

20061220 Pass Tempfile to kpacs viewer form

20070123 Added processfilter to allow browsing of patients with ' in patientid

20070126 Importconverters stuff and ForwardAssociationRelease in dicom.ini; Version to 1.4.12b

20070201 Version to 1.4.12c

20070204 Fix in generation of UIDPrefix: was not automatically generated on startup

20070206 ODBC buttons will also create database for SQL server

20070207 ODBC button creates database for native mysql
 20070211 Native MYSQL did not create indices during install
 Fixed selection of DBF (with odbc)
 20070221 Started to SQLite (not yet fully functional)
 20070305 Version to 1.4.13alpha
 20070316 Use | in patid instead of , in drag and drop to allow dropping
 files with a , in them while changing patientif (not backwards
 compatible)
 20070415 Back to , in patid in drag and drop: ljz fixed it in dgate.exe
 Added Prefetcher flag in dicom.ini (default 0)
 20070520 Fixed missing completed message in regen single device
 20070705 Added BrowseThroughDBF: unified browser used for SQLite
 Works for other datasources but not configured as such
 Small fixes in completed message in several regen functions
 Fixed view/edit database second time round (enable)
 20070706 Auto create mysql registry settings for native mysql
 Disallow anonymize and change to empty name
 Message on end of odbc or db creation
 Version to 1.4.13beta
 20070708 Patched forms.pas for Chad Moser's crash problem.
 Added BrowseThroughDBF setting in dicom.ini
 Default 1 for SQLite, 0 for others
 Use extract: also for dbaseIII: to avoid multi-user problems
 when browsing and writing at same time
 20070709 Small fix in dbf+bde browsing
 Auto register webserver\ActiveFormProj1.ocx if found
 Do not use extract: for dbaseIII: browser does not work
 while indexing
 20071030 Also delete DelayedFetchForwardFailures* on startup
 20071103 Added LRUSort, AllowTruncate, ForwardCollectDelay,
 MaximumExportRetries,
 MaximumDelayedFetchForwardRetries
 Version to 1.4.13
 20071104 Added DecompressNon16BitsJpeg, UseBuiltInDecompressor
 Made IncomingCompression editable (double click label)
 Made FileNameSyntax editable (double click label) and
 allow choice of V2 and DCM through radiobuttons
 20071114 Save root password while installing for mysql with password
 20071115 Disabled multi-process mode in RunInThreadedModeClick to
 allow multiple saves of ini file during install (for above
 password)
 20071115 Save password and username for SQL server install as well
 20071116 Longer default password for sql server 2005
 20080124 Added mailer settings in dicom.ini
 Disable LittleEndianExplicit on install
 Added configuration for mode UJ
 Database revision 16: Moved Stationname and
 InstitutionalDepartmentName to series table
 Added extra warning when revision 16 not detected
 moved Stationname and InstitutionalDepartmentName in view/
 edit database
 Version to 1.4.13a
 20080126 Automatic add backslash to read MAGDEVICES etc and give a
 warning message

20080322	Version to 1.4.14beta
20080322	Fix first install bug in test dicom.sql; fix send image crash in UID mode; display selected UIDs in query/move page
	added PadAEWithZeros and IgnoreOutOfMemoryErrors
20080818	Database revision 17: extended size of EchoNumber and ReqProcDescription to 64; EchoNumber, StudyModality and ImageType are DT_MSTR: multiple values; use Institution (8,80) instead of InstitutionDepartmentname (8, 1040)
	Added WorkListReturnsISO_IR_100
20080819	Version to 1.4.14
20080831	Default ForwardAssociationRelease to 1
20080902	mvh Added EnableComputedFields, default 0
20081104	Fixed install as four NT services: trim IP port before convert to int; Fixed save of database name to dicom.ini for mysql install
	version to 1.4.14a
20081121	Added Postgres support; attempt fix MySQL support
	Automatically choose the 64 bit dgate if possible
20081123	Retry server test for 4 sec after db install
20090203	Trim data read from dicom.ini to remove trailing spaces
20090205	Added MailPort option
20090411	version to 1.4.15alpha; Added PrintSquareLandscape, max converters to 19, VirtualPerSeries0..9, RetrieveResultConverter0, QueryConverter0, WorkListQueryConverter0, RetrieveConverter0
	Let OCX registration fail without warning (for vista)
20090927	Added RetryForwardFailed
20090927	Optionally use 7za.exe instead of zipmaster (when zipdll.dll deleted); Added RetryForwardFailed
20090928	Default sql password to conquest1415;
20090929	Added ImportExportDragAndDrop flag (default 0)
20090929	Version to 1.4.15
20100202	version to 1.4.16alpha; check other GUI running; jpeg2000 stuff; LossyQuality
20100206	use convert_to_bmp in browser to allow JPEG2000
20100207	export server task; added spinedit for frames; UseKpacsDecompression flag
20100207	version to 1.4.16alpha2; made it a const; removed warnings
	write zips to folder logs if exists; create logs folder on install
20100227	version to 1.4.16alpha3; added DICOM print to selected AE
20100827	version to 1.4.16beta; Fixed display of jpeg2000 config in GUI
	LargeFileSizeKB to 4096; note: jpeg2000 requires HAVE_J2K in cqdicom.dll; Fixed viewincoming; fixed enable NJ mode
	choice logic; fixed service detection to warn for existing but not running service
20100828	Automatically start installed but not running service when GUI comes up; Added QueryResultConverter and ModalityWorkListQueryResultConverter
20100905	Added UseBuiltInJPEG and IgnoreMagdevicethreshold flags
20101003	Added browsepatient command; show record counts in browser hints; Fixed frame number passed to BMP generator
	Improved query page text selection when brackets in text
20101003	Version to 1.4.16beta4
20101110	Typo in reading UseBuiltInJPEG (save will always write 1)
20101111	Version to 1.4.16beta6; added tests to service control

	Issue: uninstall often fails (and leaves dgate.exe running)
20101116	Fixed uninstall in dgameserv, changed messages
	More checks in service code
20101122	Add option to log to file only with gui running
	Added export to zip file options
20101123	Default EnableComputedFields to 1, added OverlapVirtualGet, ArchiveConverter0, update acrnema.map header with ui, ul, ue
20101212	Right lock new checkbox; version to 1.4.16
20101212	Disabled BigEndianExplicit for new; default file type to DCM
20101212	Removed nop export script
20101213	Enable delete even if no file found;
	stop zipping and cleanup (dgate will do it) if only log to file enabled
20101213	Fixed update of grid hints for totals: note count = unfiltered
20101220	Block spinedit for 1 frame image
20101227	Reset CheckBoxOnlyLogToFile when restarting server etc
20110105	version to 1.4.16rc1; default BrowseThroughDBF=1 for SQLSERVER
20110123	version to 1.4.16rc3; other support for other sections in dicom.ini
20110326	version to 1.4.16rc6; Also show incoming 'Rewritten' images
20110329	Version to 1.4.16; added MAGDeviceFullThreshHold
20110331	Added NoDICOMCheck; release date

SERVERABOUT.PAS (part of conquestdicomserver.exe)

19980405	Created
19980618	Added reference to Fpiette; use form also as file lister
19980624	Removed line about backdoor
19980721	Removed line about Mark Oskin going away
19990527	Release to 1.3.3
20000128	Release to 1.3.4
20000316	Release to 1.3.5
20000529	Release to 1.3.6
20001129	Release to 1.3.7
20010318	Release to 1.3.8
20010318	Release to 1.3.9; fixed dbaseIII text
20010830	Release to 1.3.10;
20011127	Release to 1.3.11;
20020816	Release to 1.3.12;
20020825	Made Lambert's e-mail address more prominent
20021018	Version to 1.4.0; added release date
20021020	Update release date
20021028	Update release date
20021215	Update release date
20030113	Version to 1.4.1; release date
20030128	new release date
20030303	new release date
20030710	Version to 1.4.2; release date
20030809	Version to 1.4.3
20030810	Added reference to offis, zipmaster, and mitec
20030922	Release date
20040402	Version to 1.4.4
20040406	Release date

20040426	Version to 1.4.4a; Release date
20040615	Version to 1.4.5; Release date
20040722	Version to 1.4.6; Release date
20040805	Version to 1.4.7; Release date
20041129	Release date
20050106	Version to 1.4.7a; Release date
20050109	Version to 1.4.8; Release date
20050303	Release date
20050912	Version to 1.4.9; Release date; worklist info
20051024	Version to 1.4.9a; Release date
20060103	Version to 1.4.10; Release date
20060314	Version to 1.4.11; Release date; new acknowledgements
20060402	Changed fpiette's website; Release date
20060708	Version to 1.4.12alpha; Release date
20061222	Version to 1.4.12; release date
20070126	Version to 1.4.12b; release date
20070201	Version to 1.4.12c; release date
20070706	Version to 1.4.13beta; release date; sqlite reference
20071116	Version to 1.4.13; release date
20080322	Version to 1.4.14beta; release date
20080831	Version to 1.4.14; release date
20081121	Version to 1.4.14a; release date
20090411	Version to 1.4.15alpha
20090927	Version to 1.4.15
20100202	Version to 1.4.16alpha
20100207	Version comes from define in the serverdriver

SENDSELECTED.PAS (part of conquestdicomserver.exe)

HEADERLISTER.PAS (part of conquestdicomserver.exe)

20020825: added keyboard interface (see serverdriver.pas)

EDITDATABASEFORM.PAS (part of conquestdicomserver.exe)

ABOUTNEW.PAS (part of conquestdicomserver.exe)

20000625	Added NewInstallForm database selector; default to MS ACCESS
20010418	Changed text; added default install button
20010830	Version to 1.3.10
20011127	Version to 1.3.11
20020816	Version to 1.3.12
20021016	Added DBF without ODBC
20021017	Version to 1.4.0
20021018	Default to USEDBASEIIIWITOUTODBC
20030324	Version to 1.4.2
20030810	Version to 1.4.3 - removed ODBC DBASEIII choice
20040102	Version to 1.4.4
20040426	Version to 1.4.4a
20040530	Version to 1.4.5
20040713	Version to 1.4.6
20041029	Version to 1.4.7
20050106	Version to 1.4.7a

20050901	Version to 1.4.9
20051024	Version to 1.4.9a
20051230	Version to 1.4.10
20060313	Added experimental native Mysql install when libmysql.dll is found
20060618	Version to 1.4.12
20070126	Version to 1.4.12b
20070201	Version to 1.4.12c
20070305	Version to 1.4.13alpha
20070305	Added experimental (not fully functional) native SQLite install when sqlite3.dll is found
20070706	Version to 1.4.13beta; sqlite always enabled (driver linked in)
20071103	Version to 1.4.13
20080322	Version to 1.4.14beta
20080819	version to 1.4.14
20081121	Added postgres; mysql unconditional; version to 1.4.14a
20090411	version to 1.4.15alpha
20090927	version to 1.4.15
20100202	Default to SQLITE; version to 1.4.16alpha
20100207	version comes from define in the serverdriver

DVIEW.PAS (k-pacs viewer part of conquestdicomserver.exe)

20000625	Added NewInstallForm database selector; default to MS
20060116	Adapted for Delphi 5 compatibility Do not use TFormatSettings; Do not use DelimitedText
20060116	Included NKI decompression (without CRC check)
20060117	Reset NKICompress flag if normal pixel data found
20060117	Fixed AddDelimitedText; fixed for multiframe data
20060117	Fix NKI compression when DCMloadMultipleFiles is true
20060117	Fix Get_ZDimension when DCMloadMultipleFiles is true
20060117	Accept conquest .V2 files as dicom in DCMloadMultipleFiles
20060120	DcmFilename now accepts multiple filenames separated by "*"
20060120	Added new dcmTool kSlice to stack through an image
20060120	New global boolean gNoOffsetList to bypass gOffsetList array
20060121	Reimplemented tiled display (mosaic), set with procedure DCMSetMosaic
20060123	Restored layout to allow windiff, lossy jpeg now gives correct error
20060126	Added support of value representation "UT"
20060126	Improved scaling of text overlay and measurements
20060129	Added option to save image in RAW format
20060129	New OnMouseEnter and OnMouseLeave events and custom cursors for several gTool's
20060129	Added overlay text color option
20060129	Fixed JPEG bug in multifile mode
20060130	Changed mosaic slice# place
20060130	Fixed nki_private_decompress: error for large pixel values (>32768)
20060131	Fixed reading of ELSCINT and ACUSON images that use NKI private tag 7fdf:0010
20060201	Improved real-time image zooming through bypassing the SetDimension procedure

20060202 Fixed display of slice numbers in empty fields in multview mode, fixed mosaic mode for multiframe dicom images
 20060204 Enabled saving to TIFF using W. Krug's Bmp2Tiff unit
 20060205 Added Tool kShutter
 20060223 Changed rescale method to rescale buffer directly after reading pixeldata
 20060223 Reverse NKI test: if nki compression mode >4 it is not made by NKI
 20060304 Fixed NKI compression of 8-bit and color images; todo: 1966 and some PT images level&window wrong
 20060420 Added orientation indicators to text overlay
 20060421 Implementation of TdcmOverlay record to describe custom overlay
 20060425 Added TRuler class to show spacing
 20060502 Implementation of basic VOI_LUT support. Currently only 16bit VOI LUTs are supported.
 20060607 Merged changes mvh 20060223 and 20060304
 20060608 Need CONQUEST define to enable NKI compression, fix crash in TdView.AdjustOrientChars
 20060609 Fixed VOI Lut crash when encoded explicit
 20060611 Added PixelPaddingValue and PixelRepresentation to DicomData and fixed an unwanted pixel value correction if pixel padding is out of SS range
 20060618 Merged again; added 20060223 and 20060304 changes back in
 20060712 fixed wrong orientation lable display when ImageOrientation and ImageOrientationPatient string is available
 20060728 Added support for non-square Pixel Aspect Ration
 20060801 Bitmap overlay support implemented
 20060830 3D Cursor function implemented using open source vector math units
 20061107 ?
 20061220 Made compileable again in conquest: stripped html support in conquest mode
 20061220 Found endless recursion on double click: IFNDEF conquered
 20061222 Tried to restore layout
 20070204 Added (dumb) access to http: files
 20070216 Important bug fix: allow switch to implicitVR: I found a NKI compressed image with VR length 03544C = 'L', 'T', 3 and LittleEndianImplicit --> length was seen as LT code causing reader to go out of sync see: { mvh 20070216 }
 20070216 Fix that magnifier would never return its cursor by setting Screen.Cursor to default when changing modes
 20070305 Put downloaded dicom file in TempPath, document download code: note not safe when using multiple controls side by side.
 20100207 Added uncompress mode (uses dgate.exe, not for OCX); delete temp before using it

UVIEWER.PAS (wraps k-pacs viewer in conquestdicomserver.exe)

20060130 Replaced Japansese fonts (never create forms on a japanese laptop ;->>>)
 20060304 Set limit to 2000 studies
 20060312 Slice with + and -: keep other keys for series browser
 Added OpenSeries; disallow hor scrollbar in stringrid

20060618	Removed unused variable; moved slice controls into toolbar
20061220	Replaced viewertemp.\$\$\$ by passed tempfile
20061222	Added some logic for better control of zoom and l&w
20070305	Made mosaic global
20071103	Use DLL way of accessing server to avoid 'server not running' error
20080108	Fixed hard-coded IP port 5678 for K-PACS viewer
20080901	Added flip left-right button; OCX define
20100207	Added uncompress option (if set uncompression done by dgate)

UViewerActiveFormImpl1.PAS (wraps k-pacs viewer in activeX control)

20070305	Created update history
----------	------------------------

SQLITE.PAS (TDataset control for GUI access to SQLITE3)

20070221	Adapted for SQLite3, but not yet fully functional
20070305	Created update history

forms.pas (part of Delphi core library)

20020101:	added two extra nil checks to avoid windows xp multi-monitor crash
20090819:	better documented changes

controls.pas (part of Delphi core library)

20020402:	added exception handling in finddragtarget and findvclwindow
20020513:	extended exception handling
20090819:	Better documented changes

dgate.dic (dictionary file)

19990915	Moved all uneven (private) group-codes to dd-private.txt
20030706	Add NKI Pixel Data
20081104	Add 0008,0061
20100214	Added compression info
20101227	Fixed 0020,1209 etc; added Conquest private commands