

Toolbar



Opens any selected studies in the viewer.



Modifies DICOM header information.



Transfers selected studies to a PACS.



Used to burn selected studies on a CD or other portable media.



Used for importing non-DICOM data to a study (ex. from scanner, jpg, tiff, etc.).

Drag existing study to the button to prepopulate study data.

Search

Use a combination of criteria to search for Patients or Studies. The more criteria entered, the more narrow the search will be.

Begins the Search process

Shows Today's studies

Shows Yesterday's studies

Clears any search criteria

Filters by specific modalities

Query Box

Queries any PACS checked in the list. Multiple PACS can be queried at once.

Queries and displays studies residing on the local machine.

Used for importing DICOM images into iQ-VIEW (ex. from CD, USB or folder)

Study Browser


Use the switch to display studies grouped by Patient name.

Column headings in the study browser can be used to sort by the column name.

Select multiple studies by checking the box next to each study.

Use the + to drill into the study and view the series or images contained within.

Modality	Status	Patient name	Patient ID	Date of birth	Study date
<input type="checkbox"/> OT		12bit ICPLUT	Demo263		2/2/2002
<input type="checkbox"/> NM		Bone, , ,	Demo1105517	5/28/1942	6/17/2008
<input type="checkbox"/> CR		Breast, Chloe	Demo003	6/12/1973	1/14/2009
<input type="checkbox"/> MR		Cerebellar, Atrophy	Demo004	6/14/1994	10/25/2002
<input type="checkbox"/> CR		Corner, Martin, Nana	Demo068	6/8/2005	3/30/2006
<input type="checkbox"/> CT		Culare, Navi	Demo008	6/22/2006	12/12/2005
<input type="checkbox"/> NM		Demo Esophagus, Christel	Demo22138	12/15/1934	5/19/2008

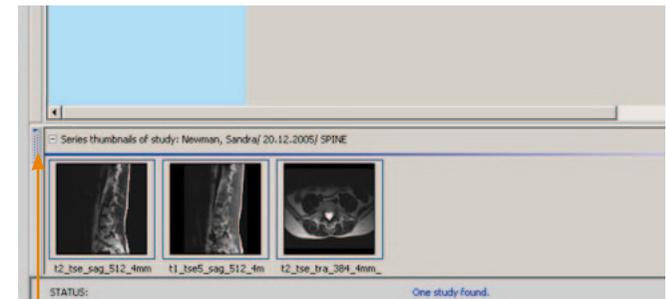

Use the switch to show/hide thumbnails of the selected study.

iQ-VIEW 2.8 QUICK GUIDE - THE VIEWER

Magnifier button queries PACS for previous studies of the current patient

Drag series thumbnails into the panes (views) below for viewing

Double-click an image to view in a single pane (view). Double-click again to return to tiling.

Switch can be used to show/hide the series preview bar

Blue border denotes the active image


Tabs show the various studies loaded

Use to display two studies side by side when multiple studies are loaded.

Click box to select study - turns red when selected

Side Toolbar

- Synchronizes different series at the current position
- Use to scroll through all series without switching views
- Determines if changes made to an image apply to the series or just the one image
- Scoutlines function
- Turns off side toolbar functions

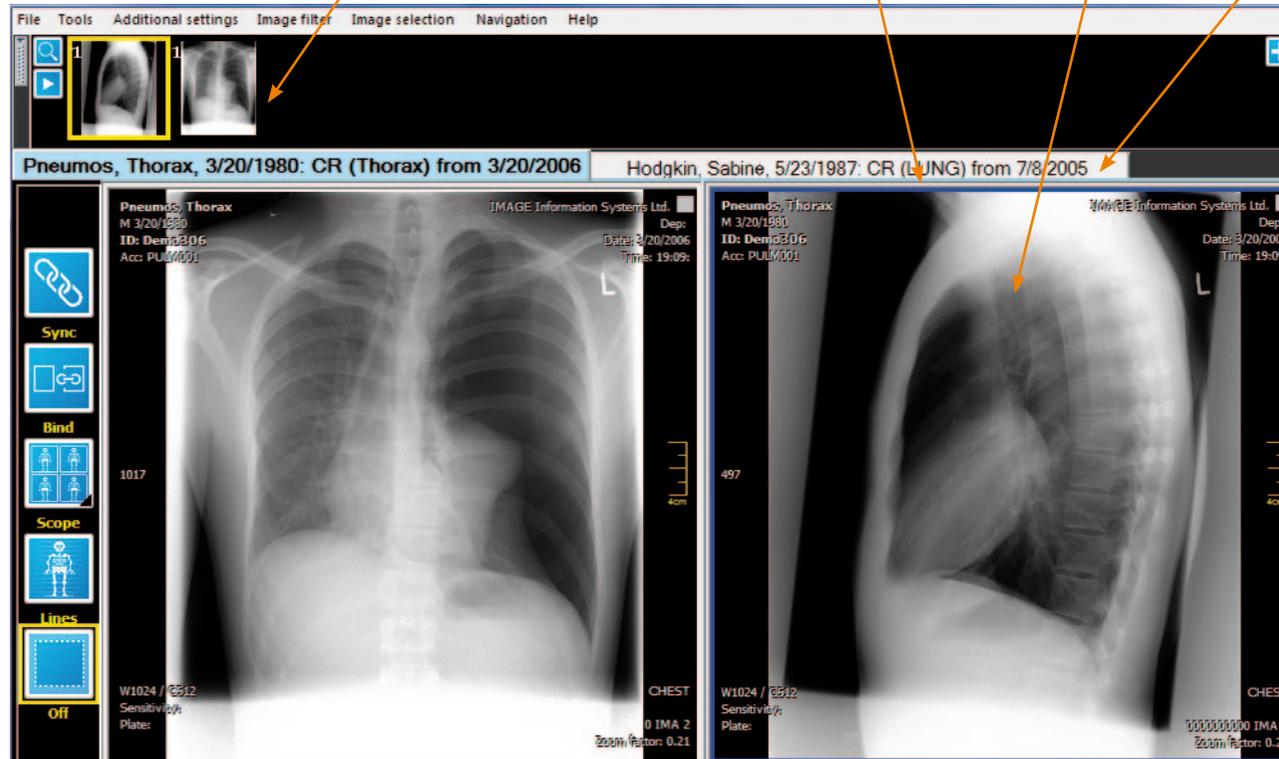

- To save measurements and annotations
- CTRL + SPACE will create a secondary capture and append it to the study
- SHIFT + SPACE creates a new secondary capture series

Bottom Toolbar

Buttons with a black triangle have other tools underneath. Click the triangle to see them.

Closes the viewer

Center/Windowing - Left click in the view and drag up/down for brightness or left/right for contrast

Cine mode - right-click for options

Color Remapping Scheme

Measurement tools

Magnifier

Zoom/Pan - Point to border of image for tool to zoom, middle of image for panning

Flip/Rotate

Tile screen

Structured Report Editor

Toggles the Text Overlay on and off

Resets the image

Link to iQ-3D, Orthoview and iQ-STITCH. Select images before launching specific program.